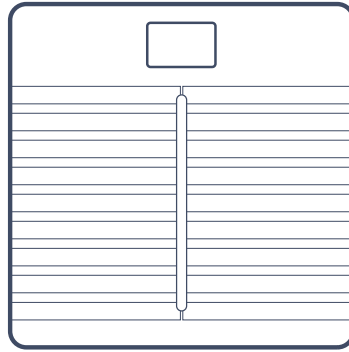
Withings Body Cardio

QUICK INSTALLATION GUIDE

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EN | JP | KR | 简 | 繁

THANK YOU FOR CHOOSING THE WITHINGS BODY CARDIO

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
感谢您选择 WITHINGS BODY CARDIO

感謝你選擇 WITHINGS BODY CARDIO 電子磅


This guide is available in additional languages at: www.withings.com/guides

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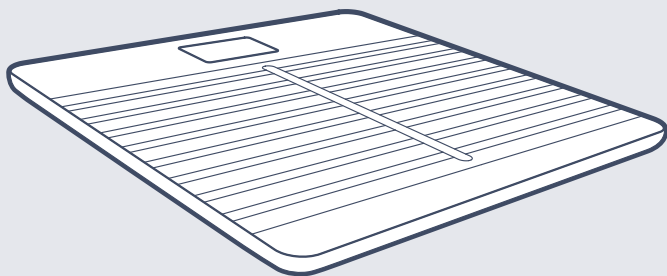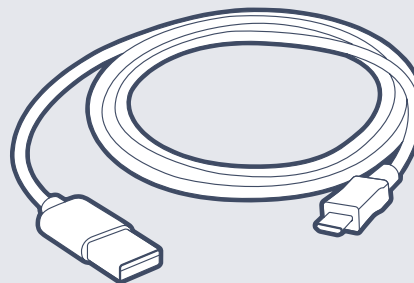Connect the cable to a power source to charge your Body Cardio. Unplug the cable to weigh yourself again.

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1

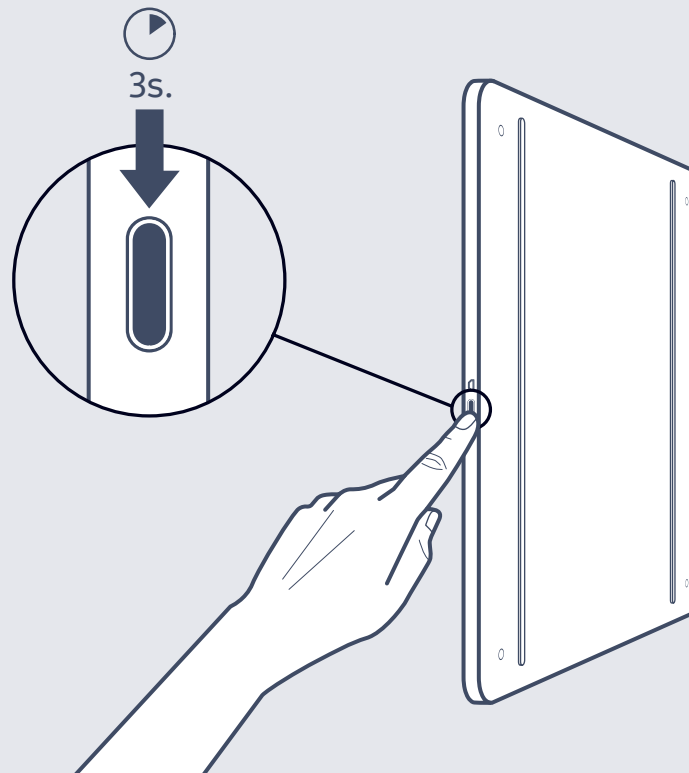
Press and hold the side button for a few seconds.

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2

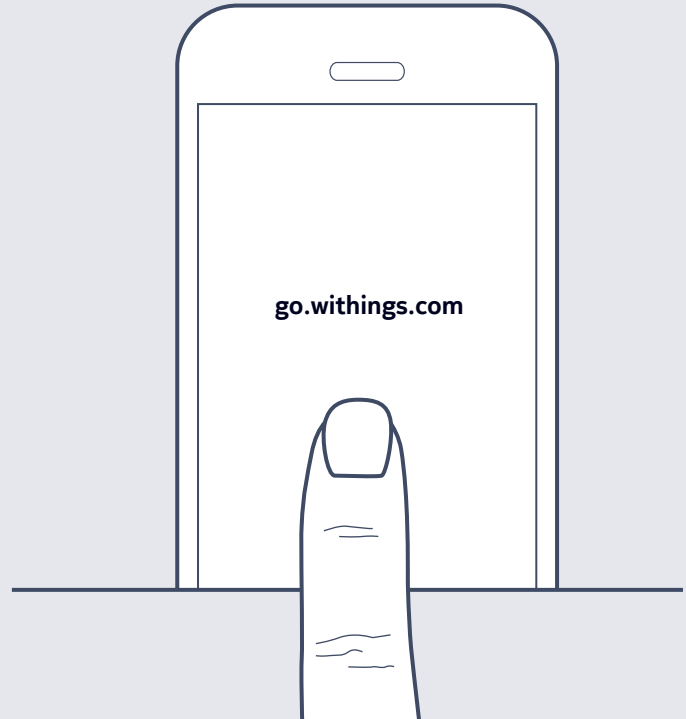
Visit go.withings.com to get the app and follow the instructions.

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Analyze your Body Composition, Heart Rate and be notified of Weather Forecast for the day, while weighing.

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
Change the order of the screens and choose what matters most to you.

If you use a Withings watch, activity tracker or track your steps with the Health Mate™ app, activate the Steps screen to know how much you walked the previous day!

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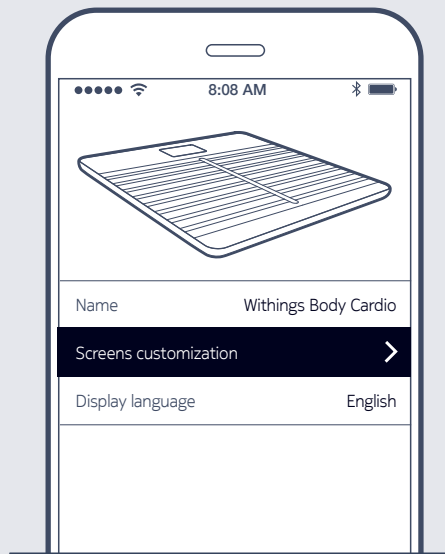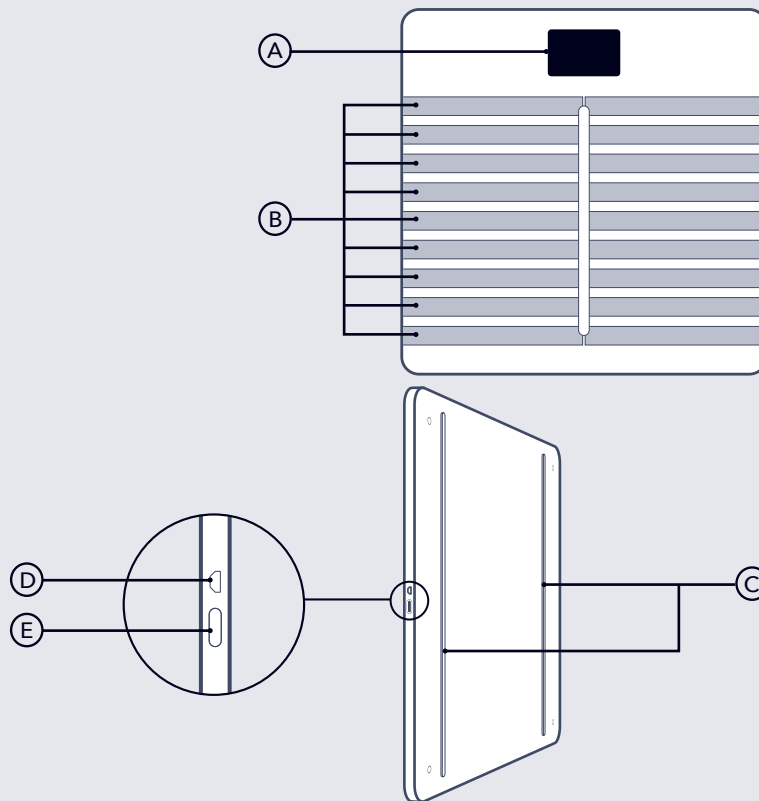
! 11

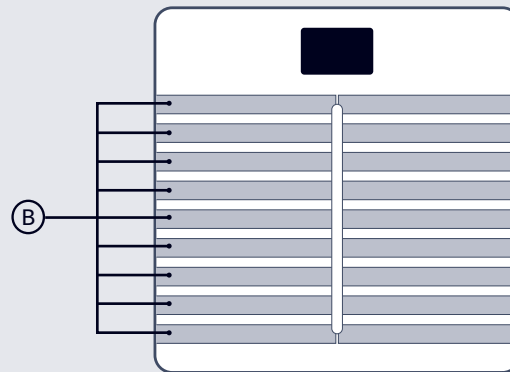
Use only a cloth dampened with water to clean your scale. The use of any detergents or cleaners may cause irreversible damage to the coating that protects the electrodes (B) used to measure your body fat and heart rate.

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This product is not suitable for people wearing a pacemaker or any other internal medical device.

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Compliance

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EN This information applies only to Withings SA Electrical and Electronic Products sold in the People's Republic of China.

Parts	Hazardous Substance					
	Pb	Hg	Cd	Cr6+	PBB	PBDE
Printed Wiring Board Assemblies	x	o	o	o	o	o
Glass parts	o	o	o	o	o	o
Metal parts	o	o	o	o	o	o
Cables & cable assemblies	o	o	o	o	o	o
Plastic and Polymeric parts	o	o	o	o	o	o
Internal battery	o	o	o	o	o	o

This table is formulated in accordance with SJ/T 11364

o : Indicates that the concentration of the hazardous substance in all homogeneous materials in the parts is below the relevant threshold of the GB/T 26572 standard.

x : Indicates that the concentration of the hazardous substance of at least one of all homogeneous materials in the parts is above the relevant threshold of the GB/T 26572 standard. The "x" in the above table indicates that some exemptions of EU RoHS are applied in the parts.

The Environment- Friendly Use Period (EFUP) for all enclosed products and their parts are per the symbol shown here, unless otherwise marked. Certain parts may have a different EFUP (for example, battery modules) and so are marked to reflect such. The Environment- Friendly Use Period is valid only when the product is operated under the conditions defined in the product manual. Manufacturing date: 02/20/2017

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14

EN Withings™ Two (2) Years Limited Warranty - Withings Body Cardio

Withings SA, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, (“Withings”) warrants the Withings-branded hardware product (“Withings Product”) against defects in materials and workmanship when used normally in accordance with Withings’ published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser («Warranty Period»). Withings published guidelines include but are not limited to information contained in technical specifications, safety instructions or quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product’s use.

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<http://gp.supportweb.jp/>

CMIIT ID: 2016DJ2373

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

Withings SA. hereby declares that the device Withings Body Cardio is in conformity with the essential requirements and other relevant requirements of Directive 2014/53/EU. The declaration of conformity can be found at: withings.com/compliance

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withings

Withings Body Cardio | Heart health & Body Composition Wi-Fi Scale

© 2018 Withings SA. All rights reserved.