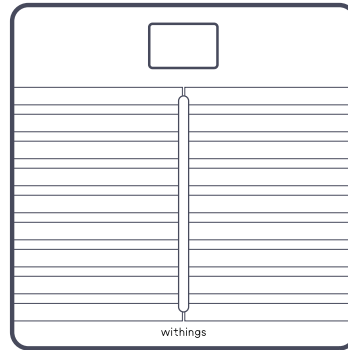
# Withings Body Cardio

## QUICK INSTALLATION GUIDE

---

GUIDE D'INSTALLATION | INSTALLATIONSANLEITUNG | GUÍA DE INSTALACIÓN | GUIDA ALL'INSTALLAZIONE

EN | FR | DE | ES | IT

# THANK YOU FOR CHOOSING THE WITHINGS BODY CARDIO

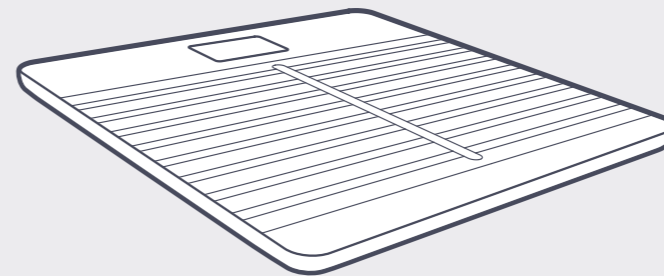
MERCI D'AVOIR CHOISI WITHINGS BODY CARDIO  
WIR BEDANKEN UNS FÜR IHRE WAHL ZUR WITHINGS BODY CARDIO  
TE AGRADECEMOS QUE HAYAS ELEGIDO WITHINGS BODY CARDIO  
GRAZIE PER AVER SCELTO WITHINGS BODY CARDIO

This guide is available in additional languages at: [www.withings.com/guides](http://www.withings.com/guides)

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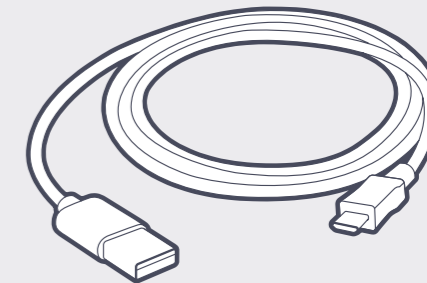
### USB Cable

Câble USB | USB-Kabel | Cable USB | Cavo USB

**Connect the cable to a power source to charge your Body Cardio. Unplug the cable to weigh yourself again.**

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4-5

1

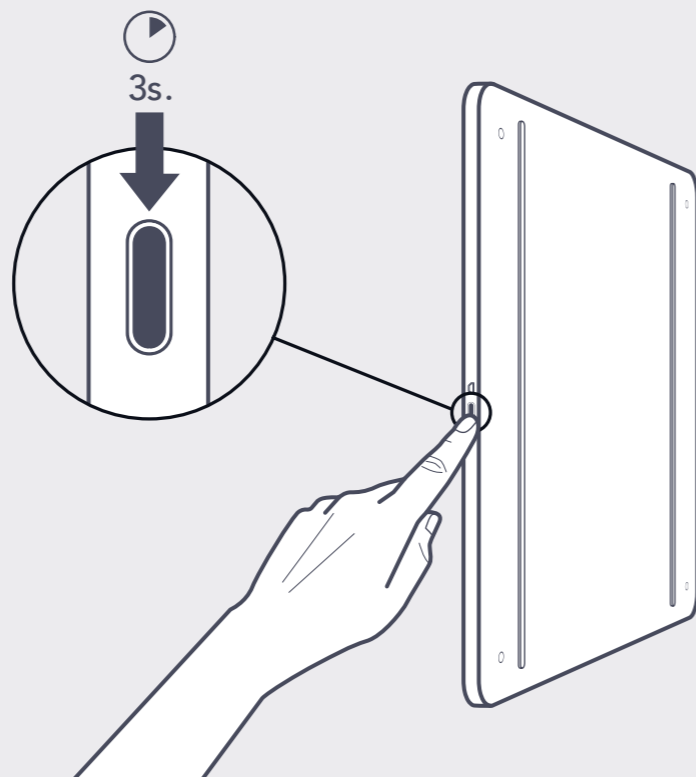
Press and hold the side button for a few seconds.

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2

Visit [go.withings.com](https://go.withings.com) to get the app and follow the instructions.

FR Obtenez l'appli via [go.withings.com](https://go.withings.com) et suivez les instructions.

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6-7

Analyzes body composition and delivers standing heart rate, pulse wave velocity, and a daily local weather\* forecast.

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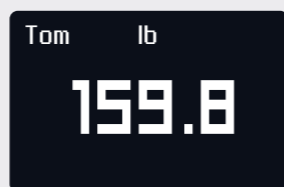
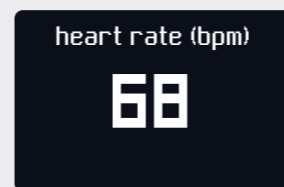
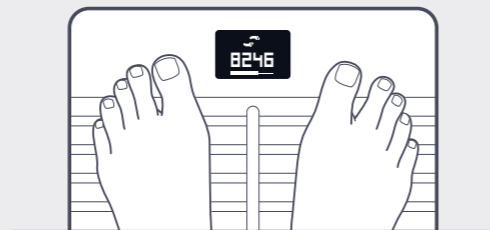
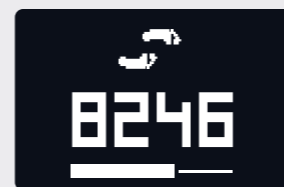
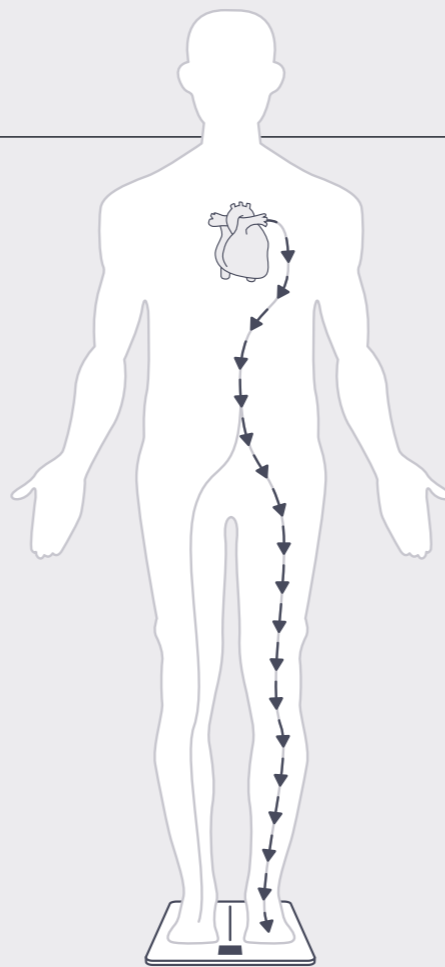
## Cardiovascular insights via Pulse Wave velocity estimation

Body Cardio gives you an indication of your overall cardiovascular health via Pulse Wave Velocity estimation, which is a medically-accepted measurement pointing to the speed of arterial blood flow.

Lifestyle changes such as a healthy diet and exercise, controlling blood pressure, diabetes, and managing cholesterol levels can help improve pulse wave velocity measurements, and consequently, your overall cardiovascular health.

Body Cardio does not make a medical diagnosis: it will tell you if the PWV estimation is normal for your age. As ever, if you have any concerns about your heart health, you should always consult with a physician.

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10-11

## Change the order of the screens and choose what matters most to you.

If you use a Withings watch, activity tracker or track your steps with the Health Mate™ app, activate the Steps screen to know how much you walked the previous day!

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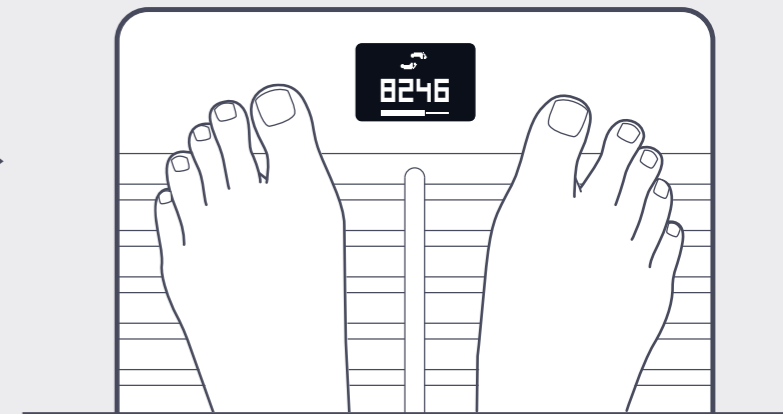
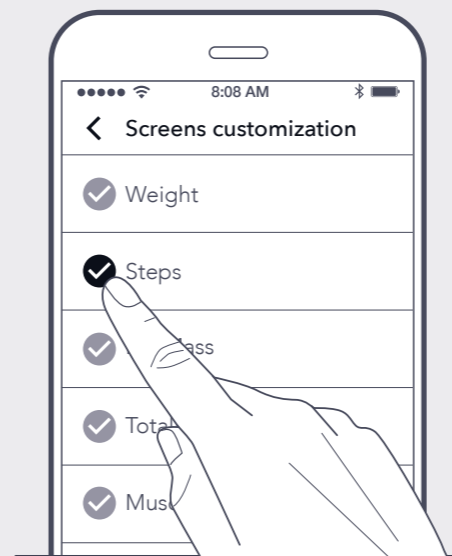
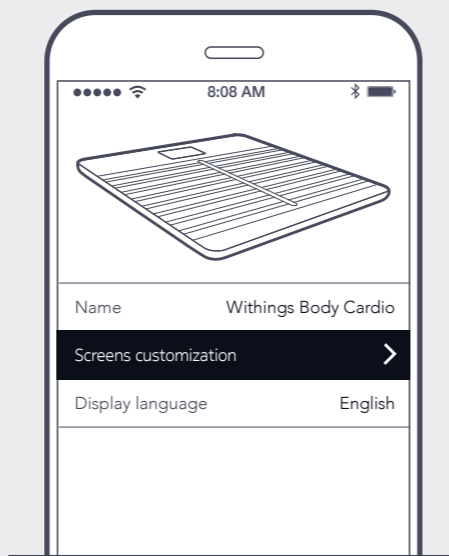
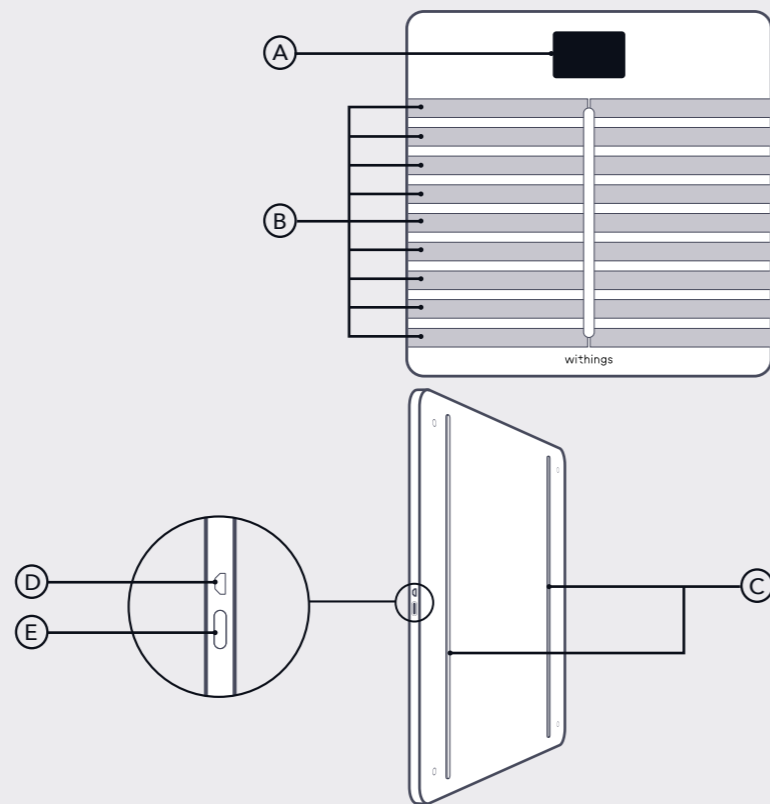
! 13

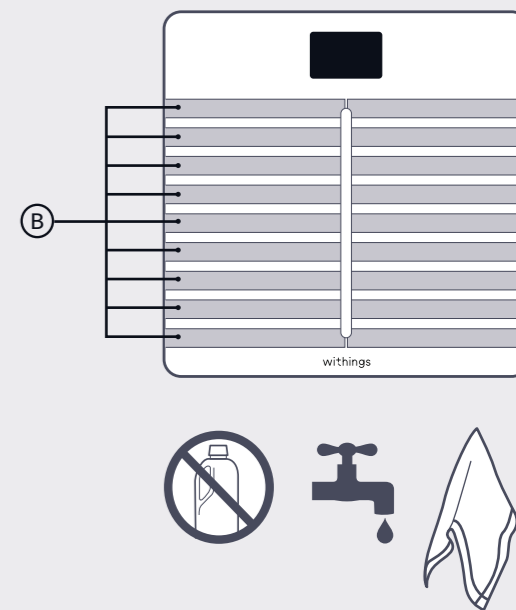
Use only a cloth dampened with water to clean your scale. The use of any detergents or cleaners may cause irreversible damage to the coating that protects the electrodes (B) used to measure your body fat and heart rate.

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**This product is not suitable for people wearing a pacemaker or any other internal medical device.**

- FR** N'utilisez pas ce produit si vous portez un stimulateur cardiaque ou autre appareil médical.
- DE** Benutzen Sie dieses Produkt nicht, wenn Sie einen Herzschrittmacher oder ein anderes medizinisches Gerät tragen.
- ES** Este producto no es adecuado para personas que llevan un marcapasos u otro aparato médico implantado.
- IT** Non utilizzare questo prodotto se porti uno stimolatore cardiaco o ogni altro dispositivo medico interno.

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15

## IC ID: 11411A-WBS04

This device complies with Industry Canada's licence exempt RSSs. Operation is subject to the following two conditions:

- (1) This device may not cause interference; and
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

## Radiation Exposure Statement

The product complies with the Canada portable RF exposure limit set forth for an uncontrolled environment and are safe for intended operation as described in this manual. The further RF exposure reduction can be achieved if the product can be kept as far as possible from the user body or set the device to lower output power if such function is available.

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Garantie | Garantie | Garantia | Garanzia

16

## US Withings™ Two (2) Years Limited Warranty - Withings Body Cardio

Withings SA, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, («Withings») warrants the Withings-branded hardware product («Withings Product») against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser («Warranty Period»). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

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## Need help?

Besoin d'aide ? | Brauchen Sie Hilfe? | ¿Necesitas ayuda? | Ti serve aiuto?

➔ [withings.com/support/bodycardio](https://withings.com/support/bodycardio)

Mode	Frequency Band (MHz)	Maximum Output Power (dBm)
BT EDR	2402-2480	-0.21
BT LE	2402-2480	1
WLAN	2412-2472	9.98

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

The crossed out wheeled bin symbol on your product, battery and/or accessories means that the item is classified as electrical and electronic equipment. Such items should not be mixed with general household waste, and must be taken to dedicated collection points at the end of their working life for proper treatment, recovery and recycling. This way you help save valuable resources and promote recycling of materials.

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withings

Withings Body Cardio | Heart health & Body Composition Wi-Fi Scale  
© 2018 Withings SA. All rights reserved.