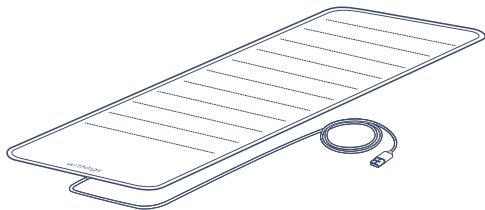
Withings Sleep

QUICK INSTALLATION GUIDE

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EN | 簡 | JP

THANK YOU FOR CHOOSING WITHINGS SLEEP

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Need help?

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
→ withings.com/support/sleep

This guide is available in additional languages at: www.withings.com/guides

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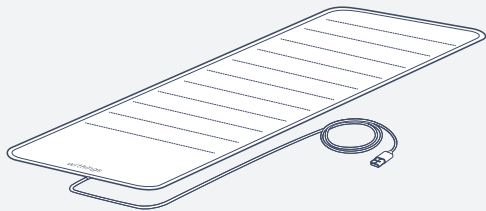
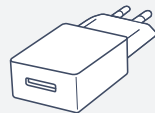
1

Place the entire sleep sensor under your mattress, so that it will be positioned underneath your chest. Only the cable should extend beyond the edge of your mattress.

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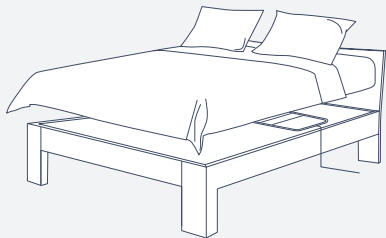
2

Plug the sensor into an outlet using the provided adapter.

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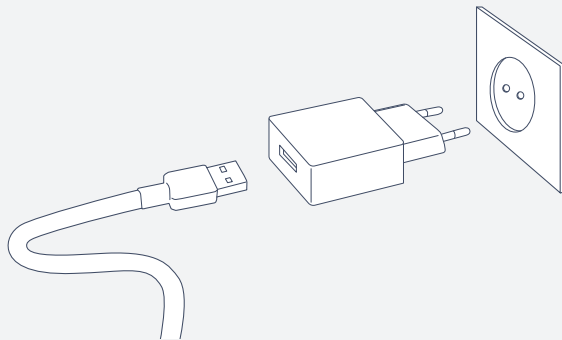4 - 5

3

Visit go.withings.com from your smartphone or tablet, and follow the instructions to install the Health Mate app.

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4

In the Health Mate app, follow the instructions to install Withings Sleep.

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
To clean the textile cover of the sensor, gently remove the cover first. The cover can be machine washed, but do not tumble dry or iron it. Wait until the cover is completely dry before replacing it on the sensor.

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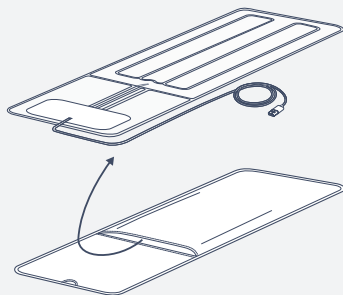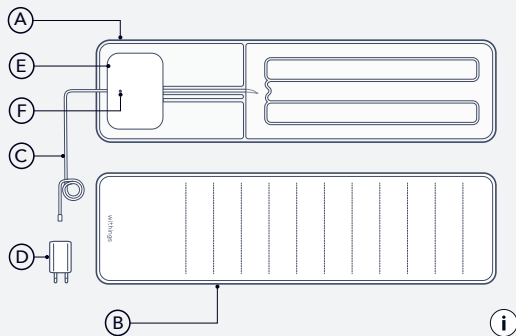
(E) Microphone for snoring detection

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EN You can deactivate the microphone in your settings in the Health Mate app

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EN Withings Two (2) Years Limited Warranty - Withings Sleep

Withings SA, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, France ("Withings") warrants the Withings-branded hardware product ("Withings Product") against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser ("Warranty Period"). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

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Withings SA, hereby declares that the device Withings Sleep is in conformity with the essential requirements and other relevant requirements of Directive 2014/53/EU. The declaration of conformity can be found at: withings.com/compliance

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation.

Mode	Frequency Band (MHz)	Maximum Output Power (dBm)
802.11 b/g/n	2400-2483.5	24.38
BT-LE	2402-2480	7.99
BT EDR	2402-2480	-6.47

CMIIT ID: 2018DJ0512

Notes

Notes

10

Notes

QIG_WSM02_029

withings

Withings Sleep Sensor - Model: WSM02 - V 1.0

© 2018 Withings SA. All rights reserved.