EN-US

EN

FR

DE

ES

IT

WITHINGS

PRODUCT GUIDE

USER GUIDE

EN - US REGIONS

This guide explains how to use your Withings Body Comp. Disclaimer: Information in this guide may change without notice.

Please contact Withings when in need of assistance, setting up, using or maintaining the device or to report unexpected operations or events.

Any serious incident that has occurred in relation to the Withings Body Comp should be reported to Withings and competent authorities in your country of residence.

Intended Use (US Regions)

Withings Body Comp is a connected smart scale providing the following wellness features :

- Measure weight
- Calculate the Body Mass Index (BMI)
- Measure body composition
- Measure pulse rate
- Estimate vascular age
- Compute electrodermal activity score

Additionally, Withings Body Comp is intended to measure body composition for weight management purposes, and to maintain healthy levels of muscle, bone and fat mass, and encourage healthy living.

It is intended for use in adults.

Warnings & safety notice

- Self-diagnosis and self-treatment can be dangerous.
Always consult your doctor in case of questions or doubt.
- Do not use the device if you have a cardiac pacemaker, an implantable cardioverter-defibrillator (ICD), or other implanted electronic devices.
- If you are pregnant, accuracy may be impacted for body composition analysis, electrodermal activity score, vascular age. Consult your doctor during pregnancy.
- Misuse of the device can cause injury.
- Do not step on the scale with wet feet. Potential allergic reactions may occur due to contact with materials used in the device.
- The device is not intended to continuously monitor vital signs in critical conditions or where the nature of the variations is such that it could result in immediate danger to the user.
- The device does not provide alarms.
- Keep away from children and pets. Small parts may be inhaled or swallowed.
- Exposing the device to prolonged lint, dust or light (including sunlight) may reduce its life or damage it.
- Inspect electrodes for warping, surface damage or corrosion and check for any other form of damage.
- Do not use the device if it is damaged. Use of a damaged device could cause injury or equipment failure.
- Do not submerge the scale or any of the components in water.
- Do not try to repair or modify this equipment yourself.
No service or repairs should be performed on the device other than the maintenance listed in this guide.

- Do not use accessories, detachable parts, or materials not described in the product guide.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Do not interconnect this equipment with other equipment not described in the product guide.
- Follow operational and storage conditions as described in the technical specifications section of this guide. Otherwise, measurement results may be impacted.
- Avoid using the device on feet with poor skin integrity, such as cuts, wounds or damaged skin.

How to use Withings Body Comp and take a first measurement

- Follow the information in the Quick Start Guide and in the Withings application to set-up your device.
- To wake up your device for the first time, please pull the tab in the battery cover. If there is no tab, press the power button for 3 seconds if needed. Once your product is installed and configured, it will appear in your Devices list in settings.
- To take the first measurement, step on the scale with bare feet. Your name, or trigram (as configured in the app), will appear on the scale's screen at the end of the weight measurement: it means your scale has correctly identified you and the measurements sequence (as defined in the app) will follow.
- If it is not the case, the measurement will most likely appear in "Unknown Measurements" in the application. Please assign it to your user so that the scale can automatically recognize the user during the next weigh-in.

Troubleshooting – Withings Body Comp

If you cannot fix the problem using the following troubleshooting instructions, please contact Withings or go to: withings.com/support

Problems	Solutions
Scale does not wake up when I step on it.	<ul style="list-style-type: none">- Make sure your Withings Body Comp has functional alkaline batteries. Change the batteries if necessary.
I'm having syncing issues with my scale.	<ul style="list-style-type: none">- Make sure that your Withings Body Comp appears in Devices in the Withings App. If not, please install it.- Make sure your Withings Body Comp has functional alkaline batteries. Change the batteries if necessary..- If your Withings Body Comp is configured with Wi-Fi, make sure your router is plugged in, that your Wi-Fi is operational, and that the scale is in range of your router.- If necessary, reboot your scale by removing a battery.- If necessary, reboot your mobile phone.
I want to change the units of my scale (lb, kg, st lb).	<ul style="list-style-type: none">- Press the button located on the back side of your scale.- Press the button again to make the arrow point towards the right unit.- Press the button once more and maintain it until the charging bar is complete.

Technical specifications

Product Name: Withings Body Comp

Model: WBS12

Battery Operated: 1 year typical use

Power Source: Four (4) alkaline cells

Display: 2.86 inch monochrome screen

Device Weight: 2,2 kg

Operation time: 30 seconds

Parts in contact with the skin: Glass plate of the scale

Units: Kilograms (kg), Pounds (lb), Stone (st lb)

Transport and storage conditions:  -10°C (14°F) – 50°C (122°F)

 10% – 90%

Max altitude: 2000m

Operation conditions:  5°C (41°F) – 35°C (95°F)

 20% – 80%

Max altitude: 2000m

IP Protection level: IP22

Mode of operation: Continuous Operation

Wireless information

EN-US

Wireless Specifications

Mode	Frequency Band (MHz)	Maximum Output Power (dBm)
BT EDR	2402-2480	1 dBm, 2.4G
BT LE	2412-2472	8 dBm, 2.4G
WLAN	2400	20 dBm

The wireless communication of the Withings Body Comp is supported by BLE and Wi-Fi communication. This communication is established between the Withings Body Comp and the Withings application. The communication between the Body Comp and the Withings application is encrypted through an exchange of a paired key.

The communication latency between the Body Comp and the Withings application takes less than 10 seconds when the Body Comp and the smartphone are less than 16 feet (5 meters) apart.

The communication between the Withings Body Comp and the Withings application is not modified with sources of interference located within 5 meters. Wireless coexistence has been tested in accordance with the following standards:

- ANSI C63.27:2017 and,
- AAMI TIR69:2017

Electromagnetic disturbances have been tested in accordance with the standard IEC 60601-1-2:2014.

After Use

Cleaning and maintenance the Withings Body Comp

- It is recommended to clean the glass plate before each measurement.
- Use only a cloth dampened with warm water to clean your scale. The use of any detergents or cleaners may cause irreversible damage to the conductive coating on top of the glass plate.
- Do not use the device while doing cleaning and maintenance operations.

Storage

- Store the device in a clean and safe location between use.
- If storage conditions are different from the operating conditions indicated in the technical specifications, wait 30 minutes before taking a measurement.

Safety and performance

RF Statement

Guidance and manufacturer's declaration-electromagnetic emissions

Withings Body Comp is intended for use in the electromagnetic environment specified below. The customer or the user of the Withings Body Comp should ensure that it is used in such an environment. Portable and mobile RF communications equipment can affect medical electrical equipment.

Emissions Test	Compliance	Electromagnetic environment - guidance
CE emissions CISPR11	Group 1	Withings Body Comp uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RE emissions CISPR11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	Withings Body Comp is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Not applicable	

Safety and performance

EN-US

Declaration - electromagnetic immunity

Withings Body Comp is intended for use in the electromagnetic environment specified below.

The customer or the user of the Withings Body Comp should ensure that it is used in such an environment.

Immunity test	IEC 60601 Test Level	Compliance Level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150kHz to 80MHz	N/A	N/A
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the equipment or system including cables than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Interference may occur in the vicinity of equipment marked with the following symbol:  .
Electrostatic discharge (ESD) IEC 61000-4-2	Contact: ± 8 kV Air: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	Contact: ± 8 kV Air: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material the relative humidity should be at least 30%.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz	The power frequency magnetic field should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Federal Communication Commission interference statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions:
(1) This device may not cause harmful interference, and
(2) this device must accept any interference received, including interference that may cause undesired operation.

FCC Caution: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation.

This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

Disposal

Actuation of European directive 2012/19/EU,for reduction in use of dangerous substances in the electric and electronic device and for garbage disposal. The symbol applied on the device or its packaging means that at the end of its useful life, the product not be disposed of with domestic waste. At the end of the device's useful life, the user must deliver it to a collection center for electric and electronic garbage, or return it to the retailer when purchasing a new device. Disposing of the product separately prevents possible negative environmental and health consequences deriving from inadequate disposal. It also allows the recovery of materials it is composed of to save energy and resources and avoid negative effects to the environment and health. In case of abusive disposal of devices by the user, will be applied administrative endorsements in compliance with current standards. The device and its parts must be disposed of as appropriate, in accordance with national or regional regulations.

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Warranty

Withings™ Two (2) Years Limited Warranty - Withings Body Comp
Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, ("Withings") warrants the Withings-branded hardware product ("Withings Product") against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser ("Warranty Period"). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or the quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free.

Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

Need help?

→ withings.com/support

Equipment Symbol Description

Do not dispose of this product as unsorted municipal waste, take it to electronic recycling.

IP22 Ingress of water or particulate matter

Type BF-Applied Part

Do not use if package is damaged and consult instructions for use

Caution

Regulatory Compliance Mark

Complies with FCC regulations

Serial number

— Direct Current

Temperature range

Upper and lower limits of pressure

Upper and lower limits of relative humidity

USER GUIDE

EN - NON-US REGIONS

This guide explains how to use your Withings Body Comp. Disclaimer: Information in this guide may change without notice.

Please contact Withings when in need of assistance, setting up, using or maintaining the device or to report unexpected operations or events.

Any serious incident that has occurred in relation to the Withings Body Comp should be reported to Withings and competent authorities in your country of residence.

Intended Use (Non-US Regions)

Withings Body Comp is a connected smart scale providing the following features without medical purposes:

- Measure weight
- Calculate the Body Mass Index (BMI)
- Measure body composition
- Measure pulse rate
- Estimate vascular age
- Compute nerve activity score

Warnings & safety notice

- Do not use the Withings Body Comp if you have a cardiac pacemaker, an implantable cardioverter-defibrillator (ICD), or other implanted electronic devices.
- Self-diagnosis and self-treatment can be dangerous. Always consult your doctor in case of questions or doubt.
- Withings Body Comp is not intended to continuously monitor vital signs in critical conditions or where the nature of the variations is such that it could result in immediate danger to the user.
- Withings Body Comp does not provide alarms.
- Exposing the device to prolonged lint, dust or light (including sunlight) may reduce its life or damage it.
- Misuse of the Withings Body Comp can cause injury.
- Keep away from children and pets.
- Inspect electrodes for warping, surface damage or corrosion and check for any other form of damage.
- Avoid using the Withings Body Comp on feet with poor skin integrity, such as cuts, wounds or damaged skin.
- Do not step on the scale with wet feet. Potential allergic reactions may occur due to contact with materials used in the Withings Body Comp.
- Small parts may be inhaled or swallowed.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the Withings Body Comp, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Do not use accessories, detachable parts, or materials not described in the product guide.
- Do not interconnect this equipment with other equipment not described in the product guide.
- Follow operational and storage conditions as described in the technical specifications section of this guide. Otherwise, measurement results may be impacted.
- Do not submerge the scale or any of the components in water.
- Do not use the Withings Body Comp if it is damaged. Use of a damaged device could cause injury or equipment failure.
- Do not try to repair or modify this equipment yourself.
- No modification of this equipment is allowed.
- No service or repairs should be performed on the Withings Body Comp other than the maintenance listed in this guide.
- If you are pregnant, accuracy may be impacted for body composition analysis, nerve activity score, vascular age. Consult your doctor during pregnancy.

How to use Withings Body Comp and take a first measurement

EN

- Follow the information in the Quick Start Guide and in the Withings application to set-up your device.
- To wake up your device for the first time, please pull the tab in the battery cover. If there is no tab, press the power button for 3 seconds. Once your product is installed and configured, it will appear in your Devices list in settings.
- To take the first measurement, step on the scale with bare feet. Your name, or trigram (as configured in the app), will appear on the scale's screen at the end of the weight measurement: it means your scale has correctly identified you and the measurements sequence (as defined in the app) will follow.
- If it is not the case, the measurement will most likely appear in "Unknown Measurements" in the application. Please assign it to your user so that the scale can automatically recognize the user during the next weigh-in.

Troubleshooting – Withings Body Comp

If you cannot fix the problem using the following troubleshooting instructions, please contact Withings or go to: withings.com/support

Problems	Solutions
Scale does not wake up when I step on it.	<ul style="list-style-type: none">- Make sure your Withings Body Comp has functional alkaline batteries. Change the batteries if necessary.
I'm having syncing issues with my scale.	<ul style="list-style-type: none">- Make sure that your Withings Body Comp appears in Devices in the Withings App. If not, please install it.- Make sure your Withings Body Comp has functional alkaline batteries. Change the batteries if necessary.- If your Withings Body Comp is configured with Wi-Fi, make sure your router is plugged in, that your Wi-Fi is operational, and that the scale is in range of your router.- If necessary, reboot your scale by removing a battery.- If necessary, reboot your mobile phone.
I want to change the units of my scale (lb, kg, st lb).	<ul style="list-style-type: none">- Press the button located on the back side of your scale.- Press the button again to make the arrow point towards the right unit.- Press the button once more and maintain it until the charging bar is complete.

Technical specifications

Product Name: Withings Body Comp

Model: WBS12

Battery Operated: 1 year typical use

Power Source: Four (4) alkaline cells

Display: 2.86 inch monochrome screen

Weight Withings Body Comp: 2,2 kg

Operation time: 30 seconds

Parts in contact with the skin: Glass plate of the scale

Units: Kilograms (kg), Pounds (lb), Stone (st lb)

Transport and storage conditions:  -10°C (14°F) – 50°C (122°F)

 10% – 90%

Max altitude: 2000m

Operation conditions:  5°C (41°F) – 35°C (95°F)

 20% – 80%

Max altitude: 2000m

IP Protection level: IP22

Mode of operation: Continuous Operation

Wireless information

EN

Wireless Specifications

Mode	Frequency Band (MHz)	Maximum Output Power (dBm)
BT EDR	2402-2480	1 dBm, 2.4G
BT LE	2412-2472	8 dBm, 2.4G
WLAN	2400	20 dBm

The wireless communication of the Withings Body Comp is supported by BLE and Wi-Fi communication. This communication is established between the Withings Body Comp and the Withings application. The communication between the Withings Body Comp and the Withings application is encrypted through an exchange of a paired key.

The communication latency between the Body Comp and the Withings application takes less than 10 seconds when the Body Comp and the smartphone are less than 16 feet (5 meters) apart.

The communication between the Withings Body Comp and the Withings application is not modified with sources of interference located within 5 meters. Wireless coexistence has been tested in accordance with the following standards:

- ANSI C63.27:2017 and,
- AAMI TIR69:2017

Electromagnetic disturbances have been tested in accordance with the standard IEC 60601-1-2:2014.

After Use

Cleaning and maintenance the Withings Body Comp

- It is recommended to clean the glass plate before each measurement.
- Use only a cloth dampened with warm water to clean your scale. The use of any detergents or cleaners may cause irreversible damage to the conductive coating on top of the glass plate.
- Do not use the device while doing cleaning and maintenance operations.

Storage

- Store the device in a clean and safe location between use.
- If storage conditions are different from the operating conditions indicated in the technical specifications, wait 30 minutes before taking a measurement.

EN

Europe - EU Declaration of Conformity

Withings hereby declares that the device Withings Body Comp is in conformity with the essential requirements and other relevant requirements of the applicable EU Directives. The full text of the EU declaration of conformity can be found at: withings.com/compliance

CE

Safety and performance

RF Statement

Guidance and manufacturer's declaration-electromagnetic emissions

EN

Withings Body Comp is intended for use in the electromagnetic environment specified below.

The customer or the user of the Withings Body Comp should ensure that it is used in such an environment. Portable and mobile RF communications equipment can affect medical electrical equipment.

Emissions Test	Compliance	Electromagnetic environment - guidance
CE emissions CISPR11	Group 1	Withings Body Comp uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RE emissions CISPR11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	Withings Body Comp is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/Flicker emissions IEC 61000-3-3	Not applicable	

Safety and performance

EN

Declaration: electromagnetic emissions and immunity for equipment and systems that are not life-supporting and are specified for use only in a shielded location.

Declaration - electromagnetic immunity

Withings Body Comp is intended for use in the electromagnetic environment specified below.

The customer or the user of the Withings Body Comp should ensure that it is used in such an environment.

Immunity test	IEC 60601 Test Level	Compliance Level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150kHz to 80MHz	N/A	N/A
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the equipment or system including cables than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Interference may occur in the vicinity of equipment marked with the following symbol:  .
Electrostatic discharge (ESD) IEC 61000-4-2	Contact: ± 8 kV Air: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	Contact: ± 8 kV Air: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material the relative humidity should be at least 30%. Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material the relative humidity should be at least 30%.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz	The power frequency magnetic field should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Disposal

Actuation of European directive 2012/19/EU, for reduction in use of dangerous substances in the electric and electronic device and for garbage disposal. The symbol applied on the device or its packaging means that at the end of its useful life, the product not be disposed of with domestic waste. At the end of the device's useful life, the user must deliver it to a collection center for electric and electronic garbage, or return it to the retailer when purchasing a new device. Disposing of the product separately prevents possible negative environmental and health consequences deriving from inadequate disposal. It also allows the recovery of materials it is composed of to save energy and resources and avoid negative effects to the environment and health.

In case of abusive disposal of devices by the user, will be applied administrative endorsements in compliance with current standards. The device and its parts must be disposed of as appropriate, in accordance with national or regional regulations.

Warranty

Withings™ Two (2) Years Limited Warranty - Withings Body Comp
Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, ("Withings") warrants the Withings-branded hardware product («Withings Product») against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser ("Warranty Period"). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or the quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free.

Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

Need help?

→ withings.com/support

Equipment Symbol Description

 Do not dispose of this product as unsorted municipal waste, take it to electronic recycling.

 Read the Instruction Manual before use

 Direct Current

 Temperature range

 Upper and lower limits of pressure

 Upper and lower limits of relative humidity

 The CE labeling certifies that the product complies with the general safety and performance requirements of Regulation 2017/745

 IP22 Ingress of water or particulate matter

 Type BF-Applied Part

 Do not use if package is damaged and consult instructions for use

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Withings Body Comp | Proactive Health Tracking © 2022 Withings. All rights reserved. Jul. 2022