SCANWATCH

QUICK INSTALLATION GUIDE

PIKA-ASENNUSOPAS | HURTIG INSTALLATIONSVEJLEDNING |
HURTIGINSTALLASJONSVEILEDNING | SNABBINSTALLATIONSGUIDE

EN | FI | DA | NO | SV

THANK YOU FOR CHOOSING WITHINGS SCANWATCH

KIITOS, KUN VALITSIT WITHINGS SCANWATCHIN | TAK ,FORDI DU VALGTE WITHINGS SCANWATCH |
TAKK FOR AT DU VALGTE WITHINGS SCANWATCH | TACK FÖR ATT DU VÄLJER WITHINGS SCANWATCH

This guide is available in additional languages at: www.withings.com/guides

For US regions, read the regulatory leaflet before using the product

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
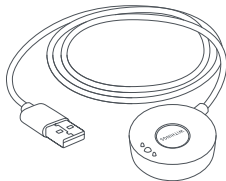
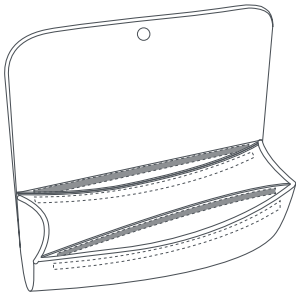
4

Visit go.withings.com to get the app and follow the instructions.

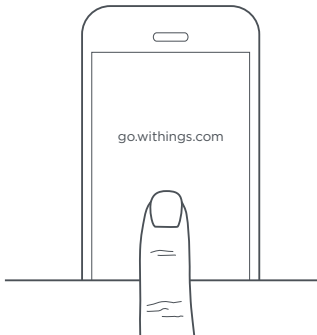- FI** Hae sovellus osoitteesta go.withings.com ja noudata ohjeita.
- DA** Besøg go.withings.com for at hente appen, og følg instruktionerne.
- NO** Besøk go.withings.com for å få appen og følg instruksjonene.
- SV** Besök go.withings.com för att hämta appen och följ instruktionerna.


Product installation

Tuotteen asennus | Produktinstallation | Produktinstallasjon | Produktinstallation

5

Install your watch in the application and follow the guide to discover all the functionalities of your product.

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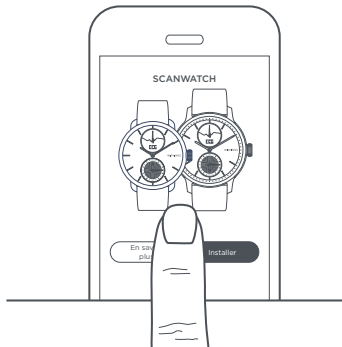
6

The wristband should be tight but comfortable, and worn 1 cm away from the wrist bone

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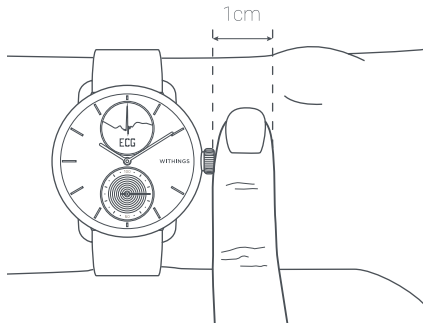
7

Recharge using the cable found inside the pouch. Battery life: up to 30 days + 20 more days in power reserve mode (time and activity tracking only).

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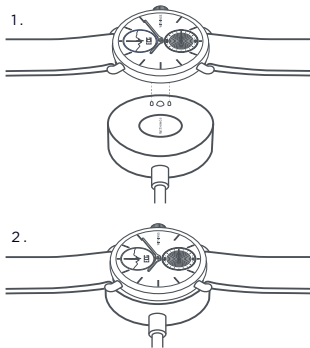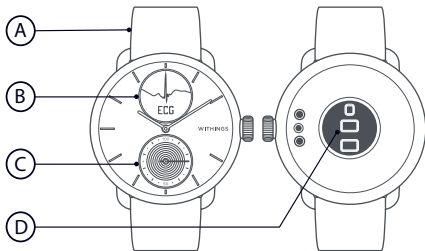
Takuu | Garanti | Garanti | Garanti

9

US Withings Two (2) Years Limited Warranty - Withings ScanWatch

Withings SA, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux ("Withings") warrants the Withings-branded hardware product ("Withings Product") against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser («Warranty Period»). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

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EN This device complies with Part 15 of the FCC Rules and with Industry Canada licence-exempt RSS standard(s). Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) This device must accept any interference received, including interference that may cause undesired operation. Changes or modifications made to this equipment not expressly approved by Withings may void the FCC authorization to operate this equipment. Radiofrequency radiation exposure information: the radiated output power of the device is far below the FCC radio frequency exposure limits. Nevertheless, the device shall be used in such a manner that the potential for human contact during normal operation is minimized.

For cleaning instructions, please visit withings.com/support/scanwatch
Puhdistusohjeet saa osoitteesta withings.com/support/scanwatch.
For rengjøringsinstruksjoner besøg venligst: withings.com/support/scanwatch
For rengjøringsinstruksjoner kan du besøke withings.com/support/scanwatch
Rengjøringsanvisninger finns på withings.com/support/scanwatch

Materials

Materiaalit | Materialer | Materialer | Material

- EN** List of materials potentially in contact with the skin during usage: Fluoroelastomer, silicone, leather, sapphire glass and stainless-steel.
- FI** Ihon kanssa mahdollisesti kosketuksiin joutuvat materiaalit: fluoroelastomeeri, silikoni, nahka, safiirilasi ja ruostumaton teräs.
- DA** Liste over materialer, der potentielt er i kontakt med huden under brug: Fluorelastomer, silikone, læder, safirglas og rustfrit stål.
- NO** Liste over materialer som potensielt kommer i kontakt med huden under bruk: Fluorelastomer, silikon, lær, safirglass og rustfritt stål.
- SV** Förteckning över material som kan komma i kontakt med huden under användning: Fluorelastomer, silikon, skinn, safirglas och rostfritt stål.

EN Withings SA hereby declares that the device Withings ScanWatch is in conformity with the essential requirements and other relevant requirements of Directive 2014/53/EU. The declaration of conformity can be found at: withings.com/compliance

«The crossed out wheeled bin symbol on your product, battery and/or accessories means that the item is classified as electrical and electronic equipment. Such items should not be mixed with general household waste, and must be taken to dedicated collection points at the end of their working life for proper treatment, recovery and recycling. This way you help save valuable resources and promote recycling of materials.»

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© 2020 Withings SA. All rights reserved. © 2020 Withings SA. Kaikki oikeudet pidätetään. © 2020 Withings SA.