# WITHINGS

PRODUCT GUIDE

# USER GUIDE

- ENGLISH -  
CANADA, AUSTRALIA AND NEW ZEALAND

This guide explains how to use your Withings Body Smart.

## Important notice

---

**Before using Withings Body Smart, review the information in this guide. You may also find this guide online at: <https://www.withings.com/guides>**

Installation instructions are available within the Quick Start Guide provided with this user guide. Please contact Withings when in need of assistance, setting up, using or maintaining the device or to report unexpected operations or events.

Any serious incident that has occurred in relation to the Withings Body Smart should be reported to Withings and competent authorities in your country of residence.

**Disclaimer:** Information in this guide may change without notice.

## Intended Use

Withings Body Smart is a connected smart scale providing the following wellness features:

- Measure weight
- Measure body composition
- Calculate Body Mass Index (BMI)
- Measure pulse rate

## Warnings & safety notice

- Self-diagnosis and self-treatment can be dangerous. Always consult your doctor in case of questions or doubts.
- Do not use the device if you have a cardiac pacemaker, an implantable cardioverter-defibrillator (ICD), or other implanted electronic devices.
- Misuse of the device can cause injury.
- Do not step on the scale with wet feet. Potential allergic reactions may occur due to contact with materials used in the device.
- The device is not intended to continuously monitor vital signs in critical conditions or where the nature of the variations is such that it could result in immediate danger to the user.
- The device does not provide alarms.
- Keep away from children and pets. Small parts may be inhaled or swallowed.
- Exposing the device to prolonged lint, dust or light (including sunlight) may reduce its life or damage it.
- Inspect electrodes for warping, surface damage or corrosion and check for any other form of damage.
- Do not use the device if it is damaged. Use of a damaged device could cause injury or equipment failure.
- Do not submerge the scale or any of the components in water.
- Do not try to repair or modify this equipment yourself. No service or repairs should be performed on the device other than the maintenance listed in this guide.
- Do not use accessories, detachable parts, or materials not described in the product guide.
- No modification of this equipment is allowed.
- Use of accessories, transducers and cables other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.

## Warnings & safety notice

- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Do not interconnect this equipment with other equipment not described in the product guide.
- Follow operational and storage conditions as described in the technical specifications section of this guide. Otherwise, measurement results may be impacted.

## How to use Withings Body Smart and take a first measurement

- Follow the information in the Quick Start Guide and in the Withings application to set-up your device.
- To wake up your device for the first time, please pull the tab in the battery cover. If there is no tab, press the power button for 3 seconds if needed. Once your product is installed and configured, it will appear in your Devices list in settings.
- To take the first measurement, step on the scale with bare feet. A user selection screen will appear on your device at the end of the weight measurement. Lean left or right to scroll between users.

## Troubleshooting — Withings Body Smart

---

If you cannot fix the problem using the following troubleshooting instructions, please contact Withings or go to: [withings.com/support](https://withings.com/support)

### Problems

### Solutions

---

**Scale does not wake up when I step on it.**

- Make sure your Withings Body Smart has functional alkaline cells. Change them if necessary.

---

**I'm having syncing issues with my scale.**

- Make sure that your Withings Body Smart appears in Devices in the Withings application. If not, please install it.  
- Make sure your Withings Body Smart has functional alkaline batteries. Change the batteries if necessary.  
- If your Withings Body Smart is configured with Wi-Fi, make sure your router is plugged in, that your Wi-Fi is operational, and that the scale is in range of your router.  
- If necessary, reboot your scale by removing a battery.  
- If necessary, reboot your mobile phone.

---

**I want to change the units of my scale (lb, kg, st lb).**

- Press the button located on the back side of your scale.  
- Press the button again to make the arrow point towards the right unit.  
- Press the button once more and maintain it until the progress bar is complete.

---

## Technical specifications

**Product Name:** Withings Body Smart

**Model:** WBS13

**Battery Life:** 15 months typical use

**Power Source:** Four (4) alkaline cells - AAA

**Display:** Color 2.86 inch LCD screen

**Weight Withings Body Smart:** 2.2kg

**Operation time:** 30 seconds

**Weight Units:** Kilograms (kg), Pounds (lb), Stone (st lb)

**Weight range:** 5kg - 200kg | 11lb - 440lb | 1st - 31st

**Transport and storage conditions:**

🌡️ -10°C (14°F) — 50°C (122°F)

🌀 10% — 90%

🌀 86kPa — 106kPa

Maximum altitude: 2000m

**Operation conditions:**

🌡️ 5°C (41°F) — 35°C (95°F)

🌀 20% — 80%

🌀 86kPa — 106kPa

Maximum altitude: 2000m

**IP Protection level:** IP22

**Mode of operation:** Continuous Operation

## Wireless information

### Wireless Specifications

Mode	Frequency Band (MHz)	Maximum Output Power (dBm)
BT LE	2402-2480	8 dBm
WLAN	2412-2462	20 dBm

The wireless communication of the device is supported by BLE and Wi-Fi communication. This communication is established between the device and the Withings application. The communication between the device and the Withings application is encrypted through an exchange of a paired key.

The communication latency between the device and the Withings application takes less than 10 seconds when the device and the smartphone are less than 16 feet (5 meters) apart.

The communication between the device and the Withings application is not modified with sources of interference located within 5 meters.

## Cleaning, Maintenance and Storage

### Cleaning and maintenance the Withings Body Smart

- It is recommended to clean the glass plate electrodes of the scale before each measurement.
- Use only a cloth dampened with warm water to clean your scale. The use of any detergents or cleaners may cause irreversible damage to the conductive coating on top of the glass plate.
- Do not use the device while doing cleaning and maintenance operations.

### Storage

- Store the device in a clean and safe location between uses.
- If storage conditions are different from the operating conditions indicated in the technical specifications, wait 30 minutes before.

## Safety and performance

### RF Statement

Electrical equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the accompanying documents. Portable and mobile RF communications equipment can affect the device.

### Guidance and manufacturer's declaration-electromagnetic emissions

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should ensure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic environment - guidance
CE emissions CISPR11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RE emissions CISPR11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/Flicker emissions IEC 61000-3-3	Not applicable	

## Safety and performance

**Declaration: electromagnetic emissions and immunity for equipment and systems that are not life-supporting and are specified for use only in a shielded location.**

### Declaration - Electromagnetic immunity

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should ensure that it is used in such an environment.

Immunity test	IEC 60601 Test Level	Compliance Level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150kHz to 80MHz	N/A	N/A
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the equipment or system including cables than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Interference may occur in the vicinity of equipment marked with the following symbol:  .
Electrostatic discharge (ESD) IEC 61000-4-2	Contact: ±8 kV Air: ±2 kV, ±4 kV, ±8 kV, ±15 kV	Contact: ±8 kV Air: ±2 kV, ±4 kV, ±8 kV, ±15 kV	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-8	2 kV for power supply lines 1 kV for input/output lines	N/A	The main power quality should be of the kind used in a typical commercial or hospital environment
Surge IEC 61000-4-5	1 kV differential mode 2 kV common mode	line(s) to line(s): +/- 1kV	The main power quality should be of the kind used in a typical commercial or hospital environment
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	-5% UT (95% dip in UT) for 0.5 cycles, -40% UT (60% dip in UT) for 5 cycles, -70% UT (30% dip in UT) for 25 cycles, -5% UT (95% dip in UT) for 5 sec	N/A	The main power quality should be of the kind used in a typical commercial or hospital environment. If the user of the equipment or system requires continued operation during power main interruptions, it is recommended that the equipment or system be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz	The power frequency magnetic field should be at levels characteristic of a typical location in a typical commercial or hospital environment.

## Information to users in Canada

### IC: 11411A-WBS13

This device complies with Industry Canada's license exempt RSSs. Operation is subject to the following two conditions:

- (1) This device may not cause interference; and
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

### Radiation Exposure Statement

The product complies with the Canada portable RF exposure limit set forth for an uncontrolled environment and is safe for intended operation as described in this manual. Further RF exposure reduction can be achieved if the product can be kept as far as possible from the user body or set the device to lower output power if such function is available.

## Disposal

Actuation of European directive 2012/19/EU, for reduction in use of dangerous substances in the electric and electronic device and for garbage disposal. The symbol applied on the device or its packaging means that at the end of its useful life, the product should not be disposed of with domestic waste. At the end of the device's useful life, the user must deliver it to a collection center for electric and electronic garbage, or return it to the retailer when purchasing a new device. Disposing of the product separately prevents possible negative environmental and health consequences deriving from inadequate disposal. It also allows the recovery of materials it is composed of to save energy and resources and avoid negative effects to the environment and health. In case of abusive disposal of devices by the user, will be applied administrative endorsements in compliance with current standards. The device and its parts must be disposed of as appropriate, in accordance with national or regional regulations.

## Warranty

### WithingsTM Two (2) Years Limited Warranty - Withings Body Smart

Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, France ("Withings") warrants the Withings-branded hardware product ("Withings Product") against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser ("Warranty Period"). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or the quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

**AU** Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

### ? Need help?

→ [withings.com/support](https://withings.com/support)

## Equipment Symbol Description

Do not dispose of this product as unsorted municipal waste, take it to electronic recycling.

Read the Instruction Manual before use

Direct Current

Device manufacturer

Temperature range

Upper and lower limits of pressure

Upper and lower limits of relative humidity

**IP22**

Ingress of water or particulate matter

Do not use if package is damaged and consult instructions for use

Caution

Serial number

Withings hereby declares Withings Body Smart is in conformity with the essential and relevant requirements of applicable EU Directives. The full text of the EU declaration of conformity is available at: [withings.com/compliance](https://withings.com/compliance)

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Withings Body Smart | Proactive Health Tracking © 2023 Withings. All rights reserved. May. 2023