WITHINGS ECG MONITOR

INSTRUCTIONS FOR USE

EN

FR

DE

ES

SV

NL

FI

DA

IT

CS

PL

PT

RO

HU

SK

LT

Table of Contents

ENGLISH - NON UNITED-STATES REGIONS	3
FRANÇAIS - RÉGIONS HORS ÉTATS-UNIS	17
DEUTSCH - REGIONEN AUSSERHALB DER USA	31
ESPAÑOL - REGIONES FUERA DE LOS EE.UU.....	45
SVENSKA - REGIONER UTANFÖR USA.....	60
NEDERLANDS - REGIO'S BUITEN DE VS	74
SUOMI - ALUEET YHDYSVALTOJEN ULKOPUOLELLA	88
DANSK - REGIONER UDEN FOR USA.....	102
ITALIANO - REGIONI FUORI DAGLI STATI UNITI.....	116
ČEŠTINA - REGIONY MIMO USA	130
POLSKI - REGIONY POZA USA	144
PORTUGUÊS - REGIÕES FORA DOS EUA.....	158
ROMÂNĂ - REGIUNI DIN AFARA SUA.....	172
MAGYAR - USA-N KÍVÜLI RÉGIÓK.....	186
SLOVENČINA - REGIÓNY MIMO USA	200
LIETUVIŲ - REGIONAI UŽ JAV RIBŲ	214

INSTRUCTIONS FOR USE

ENGLISH - NON UNITED-STATES REGIONS

This manual explains how to use the Withings ECG Monitor with your Withings Body Scan.

It applies to users in Europe, UK, Switzerland, Australia, New Zealand and Hong Kong.

Important notice

3

EN

Instructions for Use of the Withings Body Scan are described in the Product Guide provided with the product.

Please read the following information carefully before using Withings ECG Monitor.

Please contact Withings when in need of assistance, setting up, using or maintaining the device or to report unexpected operations or events.

Any serious incident that has occurred in relation to the Withings ECG Monitor should be reported to Withings and competent authorities in your country of residence.

Disclaimer

Information in this guide may change without notice.

WITHINGS,
2 rue Maurice Hartmann,
92130 Issy-les-Moulineaux,
FRANCE

1282

Intended Use (Non-US Regions)

EN

The Withings ECG Monitor is a software-only device intended for use with the Withings Body Scan to create, record, store, transfer and display lead II and lead-III of a two-channel electrocardiogram (ECG). In addition, it calculates and displays leads I, aVR, aVF, aVL.

The Withings ECG Monitor determines the presence of atrial fibrillation (AFib), sinus rhythm, and high heart rate (no signs of AFib but a sinus rhythm with heart rate between 100-150 bpm) on a classifiable waveform. The Withings ECG Monitor is not recommended for users with other known arrhythmias.

The Withings ECG Monitor is intended for over-the-counter (OTC) use. The ECG data displayed by the Withings ECG Monitor is intended for informational use only. The user is not intended to interpret or take clinical action based on the device output without consultation of a qualified healthcare professional. The ECG waveform is meant to supplement rhythm classification for the purposes of discriminating AFib from sinus rhythm and is not intended to replace traditional methods of diagnosis or treatment.

The Withings ECG Monitor is intended for use by an adult population.

Contraindications

DO NOT use the Withings ECG Monitor with a cardiac pacemaker, ICD, or other implanted electronic devices.

Cautions

- The Withings ECG Monitor cannot check for signs of heart attacks or ischemic heart conditions. If you experience chest pain, pressure, tightness or what you think is a heart attack, call emergency services. If you believe you are having a medical emergency, call emergency services.
- DO NOT take recordings when Withings Body Scan is in close vicinity to strong electromagnetic fields.
- Interpretations made by Withings ECG Monitor are potential findings, Withings ECG Monitor cannot diagnose cardiac conditions. The user is not intended to interpret or take clinical action based on the device output without consultation of a qualified healthcare professional.
- The waveform generated by Withings ECG Monitor is meant to supplement rhythm classification for the purposes of discriminating Afib from normal sinus rhythm and tachycardia (high heart rate) and not intended to replace traditional methods of diagnosis or treatment.
- The Withings ECG Monitor cannot detect all instances of Atrial Fibrillation. You should contact your physician if you experience any changes to your health.
- DO NOT self-diagnose or self-medicate on the basis of this device. In particular, do not start taking any new medication or change the type and/or dosage of any existing medication without prior approval of your physician.
- Withings ECG Monitor is not intended to identify heart-related conditions other than Afib.
- DO NOT take recordings when Withings Body Scan is outside of the operational temperature range and humidity range indicated in the product guide.
- The Withings ECG Monitor is not intended to continuously monitor vital signs in critical conditions or where the nature of the variations is such that it could result in immediate danger to the patient. Withings ECG Monitor does not provide alarms.

Using Withings ECG Monitor with Withings Body Scan

EN

On-boarding for new Withings' users:

- Before being able to use the Withings ECG Monitor with the Withings Body Scan, you will need to download the Withings app and create an account.
- Only use official app stores to download the Withings app. To ensure that the app is the official Withings space, use the following link: <https://go.withings.com>
 - The Withings app is compatible with smartphones or tablets with iOS 14 or later, or with Android 8 or later;
 - Use a trusted Wi-Fi network with your Withings app. Do not use a public Wi-Fi network you don't know;
 - The mobile application is not intended to be used on a computer. No anti-virus software is needed.

You are now ready to set up the scale Withings Body Scan with the Withings app on your iOS or Android device. Please note that the Withings ECG Monitor is only compatible with Withings Body Scan.

Setup:


- Once you have downloaded the Withings app from an official store (App store and Google Play store), open the app — see previous page for more information.
- In the Devices tab, select "Install a device," then select "Scales."
- Select the product "Body Scan."
- Follow the on-screen instructions. You will be prompted to pair your device via Bluetooth.
- After pairing, a tutorial section will be available. Follow the ECG tutorial in order to activate Withings ECG Monitor.
- You may exit onboarding at any time by tapping Cancel.

Recording an ECG


EN

How to take an ECG recording:


1 - Make sure the Withings Body Scan is on a stable surface, and that you activated and enabled the ECG measurement in the Withings app (see "How to enable ECG Measurement?" section).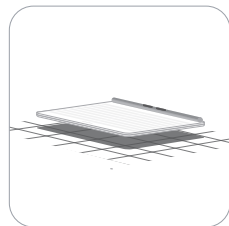


2 - Grab the handle. Hold the handle in both hands, taking care that the part of the handle marked with an "L" is in your left hand and that the part of the handle marked with an "R" is in your right hand. Make sure both hands touch the metallic electrodes on the back of the handle. The handle should be held with the cable positioned vertically, as shown in the figure below.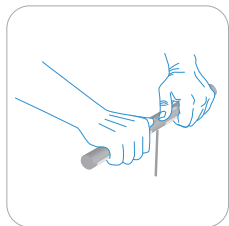

3 - Step on the scale with bare feet. Make sure your feet are centered on the scale's stripes (electrodes).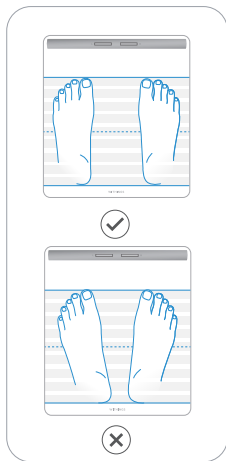


4- Keep your arms along the side of your body and relax. You do not need to grip the handle very tightly.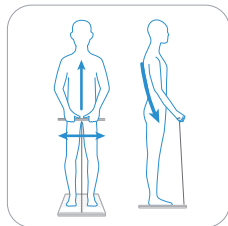


Recording an ECG

EN

5 - A series of measurements will start depending on the configuration you chose in the application (see page 04 "Using Withings ECG Monitor with Withings Body Scan"). You will know that the ECG measurement has begun when the ECG recording screen is displayed on the scale LCD screen and after the first vibration. Do not speak or move during the ECG recording.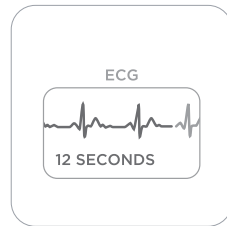

6 - It takes 30 seconds to complete a recording.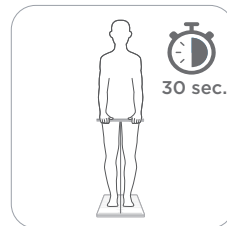

7 - Wait for the measurement to end. A countdown indicates the remaining time.

8 - When the measurement is complete you will feel a double vibration.

What is a 6-Lead ECG?

EN

How Withings ECG Monitor obtains an ECG:

- An ECG, or electrocardiogram, is the graphical representation of the electrical activity of the heart. With each heartbeat, an electrical wave travels through your heart. This wave causes your heart to contract and pump blood.
- In a doctor's office, a standard 12-lead ECG is usually taken by placing electrodes on your arms, legs, and torso. A lead measures the difference of electric potential between a pair of electrodes. Each lead measures this from a different angle.
- The Withings ECG Monitor uses electrodes from the Withings Body Scan to record a 6-lead ECG. Electrodes are in contact with the hands and feet.
- A 6-lead ECG is able to provide information about heart rate and certain heart rhythms: Sinus rhythm and Atrial Fibrillation (AFib). 6-lead ECGs are sometimes prescribed by doctors for people to wear at home or within the hospital so that the doctor can get a better look at the underlying rate and rhythm of the heart.

ECG Outputs

ECG classification:



After an ECG recording, you will see one of the following classifications for the recording:

- Low Heart Rate (heart rate < 50 bpm)
- High Heart Rate (heart rate > 150 bpm)
- Sinus Rhythm (heart rate between 50-99 bpm)
- High Heart Rate (No signs of AFib) (heart rate between 100-150 bpm)
- Atrial Fibrillation (heart rate between 50-99 bpm)
- Atrial Fibrillation — High Heart Rate (heart rate between 100-150 bpm)
- Inconclusive
- Poor Recording

After the ECG recording you will also see your average heart rate and your ECG tracing.

ECG Outputs

EN

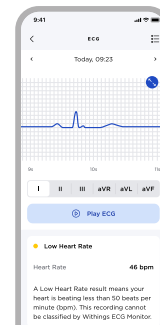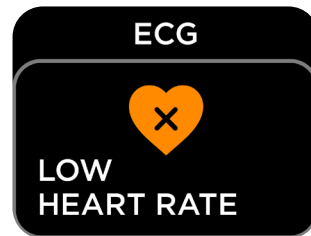Low Heart Rate

A Low Heart Rate result means your heart is beating less than 50 beats per minute (bpm). This recording cannot be classified by Withings ECG Monitor.

A low heart rate can happen if electrical signals are not properly conducted through the heart. Some medicines can also cause a low heart rate. Some elite athletes may also have a low heart rate. Talk to your doctor if you have questions about your ECG recording.



High Heart Rate

A High Heart Rate result means your heart is beating above 150 beats per minute (bpm). This recording cannot be classified by Withings ECG Monitor. Many different things can cause a high heart rate. A heart rate may be high because of exercise, stress, dehydration, infection, AFib, another arrhythmia or another cause.



Talk to your doctor if you have questions about your ECG recording.

ECG Outputs

EN

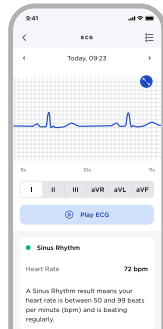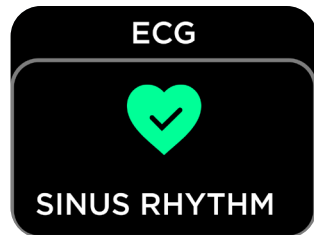**Sinus Rhythm**

A Sinus Rhythm result means your heart rate is between 50 and 99 beats per minute (bpm) and is beating regularly.

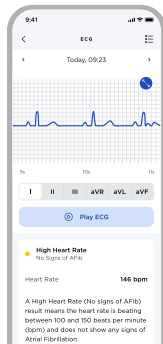
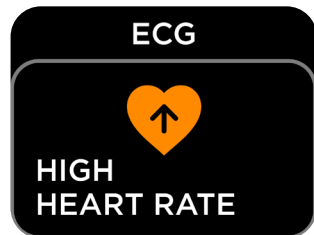
**High Heart Rate (No signs of AFib)**

A High Heart Rate (No signs of AFib) result means the heart rate is beating between 100 and 150 beats per minute (bpm) and does not show any signs of Atrial Fibrillation.



Many different things can cause a high heart rate. A heart rate may be high because of exercise, stress, dehydration, infection, an arrhythmia, or another cause. Talk to your doctor if you have questions about your ECG recording.

ECG Outputs


EN

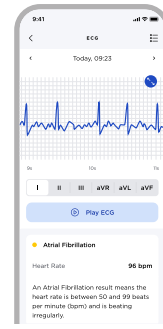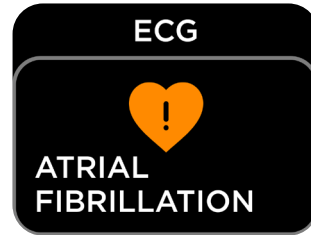**Atrial Fibrillation**

An Atrial Fibrillation result means the heart rate is between 50 and 99 beats per minute (bpm) and is beating irregularly. If you have not been diagnosed with AFib before, you should talk to your doctor.

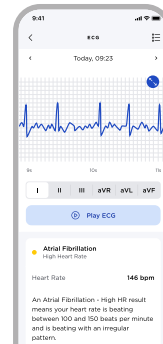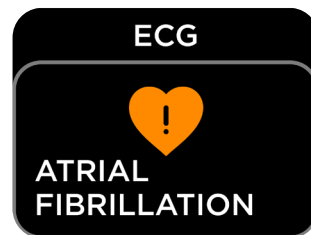**Atrial Fibrillation - High Heart Rate**

An Atrial Fibrillation — High HR result means your heart rate is beating between 100 and 150 beats per minute and is beating with an irregular pattern. If you have not been diagnosed with AFib before, you should talk to your doctor.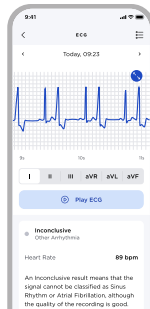


Inconclusive

An Inconclusive result means that the signal cannot be classified as Sinus Rhythm or Atrial Fibrillation, although the quality of the recording is good.

This may be due to various conditions, including but not limited to other arrhythmias or other heart conditions.

Talk to your doctor if you have questions about your ECG recording.

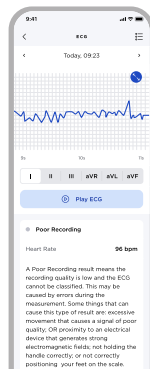Poor Recording

A Poor Recording result means the recording quality is low and the ECG cannot be classified. This may be caused by errors during the measurement. Some things that can cause this type of result are: excessive movement that causes a signal of poor quality; OR proximity to an electrical device that generates strong electromagnetic fields; not holding the handle correctly; or not correctly positioning your feet on the scale. A small percentage of people may have certain physiological conditions that prevent them from creating enough signal to produce a quality recording. You may try to re-record your ECG. You can review how to take an ECG during setup or by tapping Take a Recording in the ECG section of the Health Mate app on your smartphone.

How to share your ECG with your doctor:

Once the ECG measurement feature is activated (see page 04 "Using Withings ECG Monitor with Withings Body Scan"), the results of the ECG can be shared with a physician as a PDF, generated by the Withings app.

The pdf includes the following information:

- The ECG strip for the 6 Leads
- The average heart rate
- The classification of the ECG according to the detection algorithm.

How to enable ECG Measurement?

- First, make sure your Withings Body Scan is configured properly and that you have activated Withings ECG Monitor by following the tutorial in the Withings app.
- In order to activate or deactivate ECG measurement, go in the settings tabs of the application and find your device and select "Customize Screens."
- In the list you should be able to see "ECG 6-Lead." You can select the measurement by tapping on it. A check mark will be shown next to the measurement when activated.
- Please take into consideration that the scale will need to be synchronized once before the change is applied to your device. Performing an ECG measurement will apply the changes to your device. If you don't perform an ECG measurement, the changes will be applied to your device during a routine synchronization overnight.

Withings ECG Monitor's ability to accurately classify an ECG recording into Atrial Fibrillation and Sinus Rhythm categories was tested in a clinical study.

Rhythm classification by Withings ECG Monitor was compared to the review by cardiologists of simultaneously recorded 12-lead ECG. The study included 233 patients. 90.3% of recordings were conclusive (i.e. neither "inconclusive" nor "poor recording"). On conclusive recordings, the sensitivity in classifying Atrial Fibrillation was 98% (lower bound of the 95% confidence interval: 92%) and the specificity in detecting normal sinus rhythm was 100% (lower bound of the 95% confidence interval: 98%).

The accuracy of heart rate measured from the ECG was also assessed in this clinical study. The mean difference between the algorithm calculated heart rate from Withings ECG Monitor's strips and the heart rate measured by cardiologists on the 12-lead ECGs was -1.1 bpm (standard deviation 3.3 bpm).

The performance of the ECG function passed the applicable requirements on ECG waveform and heart rate measurements specified in ANSI/AAMI/IEC 60601-2-47:2012 Requirements for the Basic Safety and Essential Performance of Ambulatory Electrocardiographic Systems and ANSI/AAMI EC57:2012 (R2020) Testing And Reporting Performance Results Of Cardiac Rhythm And ST Segment Measurement Algorithms.

Troubleshooting — Syncing with Withings App

EN

Problems	Solutions
I'm having syncing issues with my scale.	- Make sure that your Withings Body Scan appears in Devices in the Withings app. If not, please install it. - Make sure your Withings Body Scan has enough battery life. Recharge your scale if necessary using the included USB-C power cable. When plugged in, your Withings Body Scan will display the percentage of battery left. - If your Withings Body Scan is configured with Wi-Fi, make sure your router is plugged in, that your Wi-Fi is operational and that the scale is in range of your router. - If necessary, reboot your scale. Press and hold the button of your scale for 15 seconds. - If necessary, reboot your mobile phone.

Troubleshooting — ECG

EN

If you experience difficulties with operating your Withings ECG Monitor, refer to this troubleshooting guide. If you cannot fix the problem using the troubleshooting instructions, please contact Withings.

Problems	Solutions
I cannot get the Withings ECG Monitor to take an ECG reading.	- Ensure that you have completed all of the setup steps in the Withings app on your smartphone. - Ensure that your legs, feet, arms, hands and fingers remain still during recordings and that your feet and fingers are in contact with the electrodes. - Ensure that no rings or other jewelry are touching the electrodes. - Make sure the glass plate and handle of your Withings Body Scan are clean and dry. Make sure that your hands and feet are dry. Water and sweat can be the cause of a poor quality ECG.
I have an inconclusive measurement. It looks like the ECG recording has a lot of artifact, noise, or interference.	- Ensure that your legs, feet, arms, hands and fingers remain still during recordings and that your feet and fingertips are in contact with the electrodes. - Try to relax and not to move. - Make sure the glass plate and handle of your wrist and your Withings Body Scan are clean and dry. Make sure that your hands and feet are dry. Water and sweat can be the cause of a poor quality ECG. - Move away from any electronics that are plugged into an outlet to avoid electromagnetic interference. - Remove any ring you might be wearing.
The ECG waveforms appear upside down.	- The handle may be held with the left electrode in the right hand and the right electrode in the left hand.

Withings recommends that you add a passcode (Face ID or Touch ID (fingerprint) to your phone to add a layer of security. It is important to secure your phone since you will be storing personal health information. Users should follow the one time code received by email and two-factor authentication guidelines when logging into Withings App. For more information on two-factor authentication refer to <https://support.withings.com/hc/en-us/articles/4409345943697> for iOS users and refer to <https://support.withings.com/hc/en-us/articles/21600361983633> for Android users. Users will receive email alerts in case of changes related to password, two-factor authentication and recovery code. Users will also receive additional software update notifications on the device via the Withings App, and updates are delivered wirelessly, encouraging rapid adoption of the latest security fixes. Users are able to see the currently installed firmware in the Withings App, under Devices > Body Scan. This tab also indicates if an update is available.

Do not install the device on a smartphone that you do not own. Do not use a public Wi-Fi network you don't know. Use a trusted Wi-Fi network with your device. Use a secure channel when you are sharing ECG information with your doctor. Withings also recommends upgrading your Withings App when an upgrade is available. The Withings App is not intended to be used on a computer. No anti-virus software is needed. Only use official app stores to download the Withings application. In case of doubt, use the link go.withings.com.

If needed, users can restore device configurations by following the factory reset procedure. For more information refer to <https://support.withings.com/hc/en-us/articles/10282288821521>.

Europe - EU Declaration of Conformity

Withings hereby declares that the device Withings ECG Monitor is in conformity with the general safety and performance requirements of the applicable Medical Devices Regulation (EU) 2017/745. The full text of the EU declaration of conformity can be found at: withings.com/compliance

1282

Equipment Symbols

Manufacturer

Serial number

Consult instructions for use

Medical device

Caution

Version

Instructions for Use

Model

Swiss Authorized Representative

Importer

The CE labeling certifies that the product complies with the general safety and performance requirements of Medical Device Regulation 2017/745

Australian sponsor
Emergo Australia
Level 20, tower II, Darling Park, 201
Sussex Street,
Sydney, NSW 2000,
Australia

UK Responsible Person
Emergo Consulting (UK) Ltd c/o
Cr360 - UL International
Compass House, Vision Park Histon
Cambridge CB24 9BZ
United Kingdom

CH REP

MedEnvoy Switzerland
Gotthardstrasse 28
6302 Zug
SWITZERLAND

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Withings Body Scan | Proactive Health Tracking © 2024 Withings. All rights reserved. Aug. 2024
Emergo Consulting (UK) Limited