

EN

WITHINGS

PRODUCT GUIDE

WITHINGS
BeamO

PRODUCT GUIDE

Applicable only to the
UNITED STATES

R_x

Federal law (USA) restricts this device to sale by
or on the order of a licensed practitioner.

WITHINGS, 2 rue Maurice Hartmann,
92130 Issy-les-Moulineaux, FRANCE

Important notice	3
Disclaimer	3
Product Overview	3
Intended use	4
Warnings	5
General precautions	7
How to take measurements	8
Temperature measurement	9
Stethoscope measurement	10
Electrocardiogram (ECG) measurement	11
Sharing Results	12
Troubleshooting	13
Technical specifications	14
Wireless information	16
Cleaning, Maintenance and Storage	17
Safety and performance	17
Disposal	20
Warranty	21
Security	22
Device configuration	22
Security notifications	23
Decommissioning	23
Equipment Symbol Description	24

Important notice

Before using Withings BeamO, review the information in this guide. You may also find this guide online at: <https://www.withings.com/guides>

Retain this documentation for future reference. Installation instructions are available within the Quick Start Guide provided with this Product Guide. Please contact Withings when needing assistance, setting up, using, or maintaining the device, or reporting unexpected operations or events. Any serious incident concerning Withings BeamO should be reported to Withings and competent authorities in your country of residence.

Disclaimer

Information in this guide may change without notice. To use your Withings BeamO, you need an iOS or Android device to install it. Thereafter, the product can be used without your mobile device on you, thanks to the Wi-Fi, Bluetooth®, and optional cellular (LTE Cat M1) connections. For Bluetooth connection, you will need your phone to synchronize your results to see them in the Withings app.

Product Overview

Withings BeamO is a multifunctional device for at-home health checkups, featuring:

- A 1-lead electrocardiogram with two stainless steel electrodes,
- A contactless thermometer for body temperature measurements,

Product Overview

- A digital stethoscope to auscultate heart and lung sounds.
The measurements can be used on children and adults according to the table below:

Measurement	Infant & Child (>0 years old)	Adult (≥ 22 years old)
Thermometer	✓	✓
Stethoscope	✓	✓
1-Lead Electrocardiogram	✗	✓

This medical device is intended to be used or operated by adults only (referred to as active patients), with or without medical training.

Withings BeamO is suitable for non-professional use at home and can transmit data remotely. It is also designed for use by healthcare professionals.

Intended use

Withings BeamO is a non-sterile, contactless, reusable clinical thermometer intended for the intermittent determination of human body temperature over the temporal artery as the measurement site on people of all ages.

Withings BeamO is also an electronic stethoscope that enables the recording and transmission of auscultation sound data. Withings BeamO is intended to be used by professional users in a clinical environment or by lay users in a non-clinical environment on people of all ages. The electronic stethoscope is for medical diagnostics purposes only. The device is not intended for self-diagnosis.

Withings BeamO is intended to record, display (when prescribed or used under the care of a physician), store, and transfer single-channel electrocardiogram (ECG) rhythms. It is indicated for use with individuals 22 years and older.

Warnings

This device should still be handled with care:

- DO NOT USE the ECG feature with a pacemaker, a defibrillator, or another electric implant.
- DO NOT USE the Thermometer feature on children born before term.
- The device is not intended to continuously monitor vital signs in critical conditions or where the nature of the variations is such that it could result in immediate danger to the user.
- The device is intended to be used on intact skin only. Do not take measurements over irritated skin or scars. Do not perform auscultation if there are any wounds or abrasions within the spot examined.
- The device does not provide alarms.
- During pregnancy, accuracy may be impacted.
- This medical product can be operated by adults

only. Temperature and stethoscope measurements on children must be performed by an adult. Do not perform ECG measurements on children.

- Withings BeamO contains an electronic stethoscope. It should be used solely for recording body auscultation sounds and transmitting them remotely. This medical product does not analyze the sounds recorded.

- The device contains an electrocardiograph. It should be used solely for the recording of an electrocardiogram, and transmitting ECG waveform remotely. This medical product does not analyze the ECG waveform.

- The power cord of the charger may pose a strangulation risk. Keep it away from children and pets.

- DO NOT take recordings while in proximity to or receiving medical treatment from other equipment (e.g., magnetic resonance imaging (MRI), diathermy (deep heating), lithotripsy, cauterization, and external defibrillation).

- DO NOT USE the thermometer feature on hypothermic patients (with a body temperature below 35°C (95°F)).

- DO NOT attempt to secure it to the finger using any external means (e.g., tapes or bands). Too much pressure on a finger can cause skin injury.

- DO NOT USE the ECG feature with a pacemaker, a defibrillator, or another electric implant.

- DO NOT USE the ECG feature if you have motricity issues resulting in significant tremor that prevents you from being able to hold still.

- DO NOT USE the ECG feature if you had the

Warnings

following medical events in the past 90 days:

- Myocardial Infarction (MI)
- Pulmonary embolism or pulmonary infarction
- Stroke or Transient Ischemic Attack (TIA)
- DO NOT USE the ECG feature if you have the following medical conditions (active or history of):
 - life-threatening heart rhythms (e.g. ventricular tachycardia, ventricular fibrillation, 3rd degree heartblock).
 - unstable angina, decompensated heart failure, active myocarditis or pericarditis.
- DO NOT USE the ECG feature if you have known allergy or sensitivity to stainless steel (used in BeamO electrodes)

Self-diagnosis and self-treatment are dangerous. Contact your physician in case of symptoms, doubt, questions, or the following cases:

- If you experience symptoms that could indicate you are experiencing a sudden and/or severe change in health.
- If the temperature increases in neonates and babies under 3 months, patients over 60 years old, immunocompromised patients, bedridden patients, and transplant patients.
- If other symptoms occur such as vomiting, diarrhea, pain, shivering, stiff neck, etc., even if there is no fever
- During pregnancy.

General precautions

- Follow operation and storage conditions as described in the technical specifications section of this guide. Otherwise, measurement results may be impacted.
- Exposing the device to prolonged lint or dust might damage the device or reduce its life.
- Do not submerge the device in water or liquids.
- The USB port should only be used for charging the device or connecting the provided adapter during stethoscope recordings. To recharge the battery, use a power cord that complies with the safety standards of the country in which it is used and that matches the voltage of the power outlet.
- Do not try to repair or modify this device yourself. Do not open or disassemble the device for battery replacement. If the device does not appear to be operating correctly, see the Troubleshooting section.
- The battery inside the device will stop charging when the temperature is less than 0°C (+/-5°C or) or over 45°C (+/-5°C).
- Do not use the device or the provided accessories if they are damaged. Do not shake the unit violently. Damaged sensors may lead to incorrect measurements. Inspect for warping, surface damage, or corrosion. The temperature sensor lens is fragile: do not touch it with your fingers.
- Do not interconnect this equipment with other equipment or use accessories, detachable parts, or materials not described in the instructions for use. Use of parts and components other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Use of the device adjacent to other equipment should be avoided because it could result in improper operation such as poor recording with the ECG measurement. If such use is necessary, the device and the other equipment should be observed to verify that they are operating normally.

How to take measurements

Withings BeamO allows you to take 3 types of measurements:

- ① Temperature measurements using the contactless thermometer.
- ② Heart and lung sound recordings using the digital stethoscope.
- ③ ECG using two electrodes.

To set up your device, follow the information in the Quick Start Guide and in the Withings application.

To start a measurement:

1. Press the button to TURN ON the device.
2. Select the correct user.
3. Select the measure to start.

To TURN OFF the device use the multidirectional button to navigate to the  icon and press the center button. There is also an automatic power-off after 30 seconds.



Temperature measurement



⚠ Before you start:

- The device and the patient should stay in the same ambient temperature room for 10 minutes before taking a measurement.
- Infant body temperature may vary more than adult body temperature. Avoid taking measurements on babies after nursing or while they are crying. It is recommended to take measurements on children when they are calm.

- For children under 3 months, perform 3 measurements in a row. If the 3 measurements are different, always take the highest one.
- If the patient has taken a bath or exercised, please wait for 15 minutes before taking a measurement.
- Move hair out of the way and dry any sweat before taking a measurement.

This thermometer returns an oral equivalent temperature value.

Taking your temperature:

1. Slowly scan once from the middle of the forehead to the top of the ear, as close as possible to the skin.
2. When the measurement is complete, the temperature will be displayed on the device and the colored icon indicates the temperature range according to the user's age.


Clinical research shows that the definition of elevated and high temperature changes with the person's age.¹

LED Colors meaning:

-  Normal
-  Elevated temperature
-  High temperature

The unit is either in °F or °C. You can change the unit in the Withings app settings.

Age adjusted temperature range:

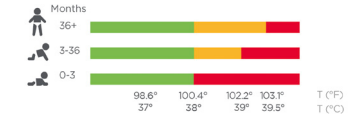

¹ Herzog L, Phillips SG. Addressing concerns about fever. Clin Pediatr (Phila) 2011 May;

Stethoscope measurement

There are 3 available recording modes:

Heart: Recordings performed on 4 positions on the chest to listen to cardiac sounds.

Lungs: Recordings performed on 8 positions on the chest and back to listen to respiratory sounds.


Wide: Wide band recordings to listen to respiratory and cardiac sounds.



Heart



Lung

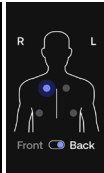

Before you start:

- Choose a calm, quiet room. Noises or murmurs may hurt the quality of recordings.
- Ensure you are in a comfortable position (preferably sitting down) with support for your hands (thighs or table).
- Do not speak or move during the measurement.
- Place the stethoscope directly on bare skin or wear at most one thin layer of clothing.

Taking a stethoscope recording:

1. Select either Heart, Lungs, or Wide (free position) mode on the device screen.
2. Place the stethoscope at the precise point indicated on the device screen.
3. When you are ready, press the button to start recording at the selected position. Make sure that the stethoscope stays in contact with your body during the whole duration of the measurement.

You can listen to the auscultation sounds during the recording by connecting your headphones to the provided adapter and the USB-C port on the device.

You may end the exam before completing recordings in all indicated positions.

Electrocardiogram (ECG)

Before you start:

- Choose a calm, quiet room.
- Ensure you are in a comfortable position (preferably sitting down) with a support for your hands (thighs or table).
- Do not speak or move during the measurement.

Recording an ECG measurement:

Place your index fingers on the electrodes. Your finger should cover the full length of the electrodes. Your finger contact should be light.

1. Select ECG in the menu screen. Launch the measurement by pressing the button.
- **Note:** if the ECG feature has already been activated, the measurement will start; otherwise, the screen will display an invitation to activate the feature.
2. The recording will last for 30 seconds.
3. The end of the measurement is confirmed by a vibration.

- For users located in the U.S., the first ECG recording will be reviewed by a healthcare professional. It should take less than 24 hours. Once completed, the feature will be unlocked and the ECG reading will be displayed in the Withings app.

What is an ECG?

- ECG, or electrocardiogram, is the graphical representation of the electrical activity of the heart.
- With each heartbeat, an electrical wave travels through your heart. This wave causes your heart to contract and pump blood.
- In a doctor's office, a standard 12-lead ECG is usually taken. This 12-lead ECG records electrical signals from different angles in the heart to produce twelve different waveforms. The device measures a waveform similar to the first lead of those twelve waveforms. This configuration is known as a single-lead ECG.



Sharing Results

Sharing a PDF: You can share your results with a physician via a PDF file, generated by the Withings app. This PDF can be used for immediate sharing or in preparation for a later televisit.

Sharing a HealthLink: Share your health record and measurements history during a televisit or at any time.

- Tap on the share tab
- Tap on Share a HealthLink and choose how you will share the measurements. The link is valid for 7 days and can be revoked at any time in the app.

The HealthLink and PDF can include the following information:

For ECG results (once ECG feature is unlocked):

- The ECG strip

For stethoscope recordings:

- The stethoscope recording
- The type (heart, lung, and wide) and positions associated with the recording.

For temperature results:

- The value of temperature and its indication temperature output.

Troubleshooting

If you cannot fix the problem using the following troubleshooting instructions, please contact Withings or go to: withings.com/support

Problems	Solution
The low battery icon is displayed on the screen	Charge the device with the cable provided
The measured temperature is too low or too high	The temperature is outside the range. i.e lower than 34.0°C (93.2°F) or higher than 42.2°C (108.0°F). Take a new temperature measurement, referring to the user manual.
Device is outside of operating temperature range	The unit has been stored in a room outside of the operating range. Place the device in an ambient temperature room for 10 minutes and try again.
The temperature seems too low	The skin has sweat or hair covering the area. Ensure that there is no hair in front of the sensor. Clean the skin with a dry cloth and wait for 5 minutes before taking a measurement. The patient has been in a cold room. Wait until the patient is warmer before taking a measurement. Measurement was not taken on the temple. Please refer to the user manual to make the correct gesture with the sensor.
Temperature accuracy	The thermometer is calibrated during manufacture. If this device is used according to the use instructions, periodic re-calibration is not required. You should not perform calibration. Contact Withings in case of doubt.
Bluetooth does not seem to work	Smartphone is out of range. Please bring your smartphone closer to your device. Smartphone's Bluetooth is OFF. Please switch your smartphone's Bluetooth ON.

Troubleshooting

Problems	Solution
WiFi does not seem to work	Device is out of range from the Wi-Fi source. Please get your smartphone and device closer to your Wi-Fi source.
The stethoscope recording seems too noisy	This issue can be due to excessive movement, speaking during a recording, or ambient noise in the room, which causes a signal of poor quality. You can review the best practices on how to take a stethoscope by tapping Take a Stethoscope Recording in the Stethoscope section of the Withings app on your smartphone.
ECG result screen displays “Measurement failed” or “Poor recording”	The recording quality is too low to be reviewed by a physician. Some things that can cause this type of result are excessive movement that causes a signal of poor quality; proximity to an electrical device that generates strong electromagnetic fields; OR not following best practices for the gestures to be adopted; A small percentage of people may have certain physiological conditions that prevent them from creating enough signal to produce a quality recording. You may try to re-record your ECG. You can review how to take an ECG during setup or by tapping Take a Recording in the ECG section of the Withings app on your smartphone. If you think you may be having a heart attack (myocardial infarction) or are facing a medical emergency, call the emergency services.

Technical specifications

Product Name: Withings BeamO Model: SCT02 Default Setiings: The device is provided in the default settings (brightness, navigation, BLE= ON, WiFi=ON) and may not be changed by the user. ECG Sensors: 2 Stainless steel electrodes Stethoscope sensor: Piezoelectric Sensor Auscultation Frequency Ranges: Heart: 20 Hz - 400 Hz, Lung: 60 Hz - 600 Hz, Wide: 20 Hz - 1000 Hz Thermometer sensor: Thermopile Temperature Display: 3 digits (°C and °F - can be changed in the Withings App) Temperature Measuring Range: 34.0°C - 42.2°C (93.2°F - 108.0°F) Temperature Clinical Accuracy: ± 0.2°C on 35.5°C - 42°C range (± 0.4°F on 95.9°F - 107.6°F range) ± 0.3°C (0.5°F) outside this range. ASTM laboratory accuracy requirements in the display range of 37 to 39 °C (98 to 102 °F) for IR thermometers is ± 0.2 °C (± 0.4 °F), whereas for mercury in-glass and electronic thermometers, the requirement per ASTM Standards E 667-86 and E 1112-86 is ± 0.1 °C (± 0.2 °F). Unadjusted Temperature Test Mode Accuracy (for laboratory tests only): ± 0.3°C (± 0.5°F) on 34.0°C - 42.2°C (93.2°F - 108.0°F) range Battery Operated: 8 months use on a single charge Power Source: 3.7 VDC Lithium-ion Battery (use the USB-C charging cable provided) and a DC 5V power adapter	Operation time: 3 minutes Parts in contact with the skin: Entire product surface Transport and storage conditions: -25 to 70°C, 20-90% RH, atmospheric 86kPa~106kPa, Operation conditions: 15 to 40°C, 20 to 90% RH, atmospheric 86kPa~106kPa, altitude: 2000m IP Protection level: IP22 Mode of operation: Intermittent operation Expected minimum product life: 3 years Wireless Transmission: Wi-Fi, BLE, and optional LTE Cat M1 (Cellular) Weight: Approx. 80g Dimensions: 3.7 x 1.9 x 13.6 cm (1.465x0.765x5.355 in) Packaging content: Main unit, USB-C charging cable, USB-C to jack adapter, Travel pouch, Quick Start Guide, Product Guide.
--	---

Wireless information

Mode	Frequency Band (MHz)	Maximum Output Power (dBm)	Wireless Specifications:	
BT LE	2402-2480	6	Wireless Technology	Bluetooth BLE
WLAN	2412-2484	15	Version	Supported BT5.1
LTE Cat M1 (Cellular)	1920-1980 (B1)	23	Operation Frequency	2402MHz - 2480MHz
	1850-1910 (B2)	23	Transmission power	6 dBm (max)
	1710-1785 (B3)	23	Modulation	GFSK
	1710-1755 (B4)	23	Receiver sensitivity	-96dBm
	880-915 (B8)	23	The wireless communication of the device is supported by BLE, Wi-Fi, and optional cellular communication. The communication is encrypted through an exchange of a paired key and established between the device and the Withings app for BLE and Wi-Fi. The communication latency between the device and the Withings app takes less than 10 seconds when the device and the smartphone are less than 16 feet (5 meters) apart.	
	699-716 (B12)	23		
	777-787 (B13)	23	The communication between the device and the Withings app is not modified with sources of interference located within 5 meters. Wireless coexistence has been tested by the following standards: - ANSI C63.27:2017 and, AAMI TIR69:2017	
	832-862 (B20)	23		
	703-748 (B28)	23		

Cleaning, Maintenance and Storage

- Clean the device with a soft and dry cloth before use. Do not use an alcohol-based or solvent agent.
 - [FOR HOSPITAL USE ONLY] For disinfection you may use Liquinox (Alconox).
 - Do not submerge the device in water.
 - Do not use the device while cleaning or charging.
 - While the device is charging, it cannot be used.
- The USB-C port does not supply power during use.
- Store the device and the components in a clean and safe location.
 - If you see changes to the exterior of the device (corrosion, surface damage, warping) , it may be at the end of its reuse life. Performance may start to become affected. If this occurs, you should stop using the device.

Safety and performance

Temperature Accuracy Validation Study Summary

Withings BeamO’s ability to accurately measure temperature was validated in line with the standard EN ISO 60601-2-56:2017. The study included a total of 106 participants across the distribution of age groups indicated in the standard. The clinical evaluation provided the following metrics on the device’s clinical accuracy:

- Clinical Bias: -0.074°F, indicating that the BeamO measured slightly lower than the predicate device thermometer.
- Limits of Agreement: ±1.023°F.
- Repeatability: 0.29°F.

RF Statement

Medical Electrical Equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the following. | Portable and mobile RF communication equipment can affect Medical Electrical Equipment.

Safety and performance

Guidance and manufacturer’s declaration - electromagnetic emissions

The device BeamO is intended for use in the electromagnetic environment specified below. The customer or the user of the device BeamO should ensure that it is used in such an environment.

Emissions test	Compliance	Electromagnetic environment — guidance
CE Emissions CISPR11	Group 1	The BeamO uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference with nearby electronic equipment.
RE emissions CISPR11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	This BeamO is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Not applicable	

Safety and performance

- Declaration: electromagnetic emissions and immunity for equipment and systems that are not life-supporting and are specified for use only in a shielded location.
Federal Communications Commission (FCC) Statement: Model SCT02 FCC ID: XNASCT02 contains FCC ID: 2AAGMGM02SA
This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) this device may not cause harmful interference and (2) this device must accept any interference received, including interference that may cause undesired operation of the device. FCC Statement §15.21: You are cautioned that changes or modifications not expressly approved by the part responsible for compliance could void the user’s authority to operate the equipment. §15.105(b): This equipment has been tested and found to comply with the limits for a Class B digital device, under part 15 of the FCC rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used by the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:- Reorient or relocate the receiving antenna.- Increase the separation between the equipment and receiver. -Connect the equipment to an outlet on a circuit different from that to which the receiver is connected. -Consult the dealer or an experienced radio/TV technician for help.

IMPORTANT NOTE:

FCC RF Radiation Exposure Statement: This Transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Disposal

The device and its parts must be disposed of as appropriate, by national or regional regulations.



Warranty

Withings Two (2) Years Limited Warranty - Withings BeamO

Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, (“Withings”) warrants the Withings-branded hardware product (“Withings Product”) against defects in materials and workmanship when used normally by Withings’ published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser (“Warranty Period”). Withings’ published guidelines include but are not limited to information contained in technical specifications, safety instructions, or Quick Start Guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product’s use.

You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

Withings recommends that you add a passcode (personal identification number [PIN]), Face ID, or Touch ID (fingerprint) to your phone to add a layer of security. It is important to secure your phone since you will be storing personal health information. Users should follow the authentication guidelines when logging into the Withings app.

For more information on password requirements and two-factor authentication, please refer to support.withings.com. Users will receive email alerts in case of changes related to passwords, two-factor authentication, and recovery code. Users will also receive additional software update notifications via the Withings app, and updates are delivered wirelessly, encouraging rapid adoption of the latest security fixes. Users can see the currently installed firmware in the Withings app, under Devices > Withings BeamO. This tab also indicates if an update is available. Do not install the device on a smartphone that you do not own. Do not use a public Wi-Fi network you don't know. Use a trusted Wi-Fi network with your device. Use a secure channel when you are sharing personal information with your doctor. Withings also recommends upgrading your Withings App when an upgrade is available. The Withings App is not intended to be used on a computer. No anti-virus software is needed. Only use official app stores to download the Withings app. In case of doubt, use the link go.withings.com.

Device configuration


If needed, users can restore device configurations by following the factory reset procedure which is available through the Withings' Help Center. The device installation will set secured parameters by design. If you notice any cybersecurity vulnerability or incident, do not hesitate to reach the customer support.

Security notifications

You will receive emails when sensitive actions are performed by yourself such as password update. Moreover, you can subscribe to our status webpage (status.withings.io) in order to be notified if there is an incident or maintenance ongoing. More information about how Withings protects your data can be found through the Withings' security web page (withings.com/us/en/data-security).

Decommissioning:

If you no longer use the device, we recommend that you perform a factory reset of your device in order to remove the personal data stored within the device.

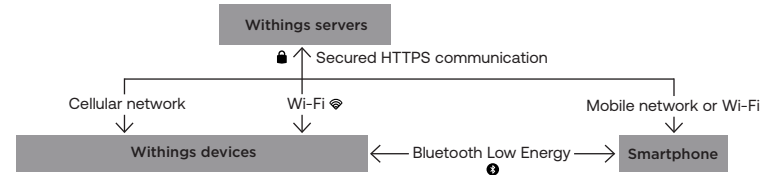

Equipment Symbol Description

- Direct current
- Model Number
- Catalog number
- Do not use if package is damaged and consult instructions for use
- Caution
- Type BF applied part
- Regulatory Compliance Mark
- Consult instructions for use or consult electronic instructions for use
- Medical device

- Follow instructions for use
- Manufacturer
- Date of Manufacture
- Do not dispose of this product as unsorted municipal waste, take it to electronic recycling.
- The CE labeling certifies that the product complies with the general safety and performance requirements of Regulation 2017/745 Medical device
- Cardboard
- Swiss Authorized Representative
- Importer

- Serial number
- Unique device identifier
- Temperature range
- Upper and lower limits of relative humidity
- Upper and lower limits of atmospheric pressure
- Ingress of water or particulate matter
- Keep dry
- WARNING: Cancer and Reproductive Harm
www.P65Warnings.ca.gov
- Prescription Use

AW_PRODUCT_GUIDE_SCTo2_ALL_US_G

WITHINGS

Withings BeamO | Proactive Health Tracking | © 2025 Withings. All rights reserved. August 2025

 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, France, support.withings.com - WITHINGS