EN
FR
DE

WITHINGS

PRODUCT GUIDE

INSTRUCTIONS FOR USE

EN – US & NON-US REGIONS

Content

3

INTRODUCTION	4	GENERAL INFORMATION	20
Important notice	4	Technical specifications	20
DISCLAIMER	4	European Union – Declarations of Conformity	20
About the Withings U-Scan Nutrio Cartridge	5	Warranty	21
About the Withings U-Scan System	5	Security	21
Warnings & safety notice	5	Equipment symbol description	22
THE WITHINGS U-SCAN NUTRIO CARTRIDGE	6		
Overview	6		
Content of the Withings U-Scan Nutrio Cartridge packaging	6		
Chemistry performance	7		
Interfering molecules	7		
Set-Up Process and Maintenance	8		
1. Charge your Withings U-Scan Reader	8		
2. Cartridge installation	9		
3. Toilet installation	10		
4. Changing the Cartridge	12		
5. Cleaning the Withings U-Scan System	14		
6. Storing the Withings U-Scan System	18		
7. Cartridge Lifespan	18		
8. Troubleshooting	19		

EN

Introduction

EN

Important notice

Thank you for choosing Withings U-Scan. Before using Withings U-Scan Nutrio Cartridge, review the information in this guide. You may also find this guide online at <https://www.withings.com/guides>. Retain this documentation for future reference. Installation instructions are available within the Quick Start Guide provided alongside this Product Guide. Please contact Withings when in need of assistance for setting up, using or maintaining the device or to report unexpected operations or events.

For more information on the Withings U-Scan System, please refer to the Withings U-Scan Reader Product Guide.

Please note that Withings U-Scan Nutrio Cartridge is a wellness product. Withings U-Scan Nutrio Cartridge is not a medical device, and is not intended for use in the diagnosis of disease or conditions or in the cure, mitigation, treatment or prevention of disease.

Disclaimer

Information in this guide may change without notice. You will need an iOS (16.0 or higher) or Android (9.0 and higher) tablet or smartphone to install your Withings U-Scan Nutrio Cartridge. Thereafter, the device can be used without your mobile device on you, thanks to the Wi-Fi and Bluetooth® connections. While, it should be noted that it is possible to launch a measurement without a mobile device, the use of a mobile device is required for the installation of the Withings U-Scan System, and to synchronise and consult your results.

About the Withings U-Scan Nutrio Cartridge

The Withings U-Scan Nutrio Cartridge is a component of the Withings U-Scan System consisting of an application (Withings App), a reader and a consumable (Cartridge) that will track the evolution of certain urinary parameters (pH, Ketones, Vitamin C and Specific Gravity) to provide you with personalised recommendations in terms of nutrition and sports activities.

About the Withings U-Scan System

In order to use the Withings U-Scan System, you need to have the Withings U-Scan Reader and a Withings U-Scan Nutrio Cartridge.

To use the Withings U-Scan System, you must first create an account on the Withings App. Once you have installed the Withings App and created your account, you will need to connect/pair your device via Bluetooth. Once the device is connected, please follow the installation instructions.

The Withings U-Scan Reader is equipped with Bluetooth Low Energy (BLE) and Wi-Fi connectivity. The test results are transmitted by Wi-Fi, as a result the Withings U-Scan Reader cannot be used without a functional Wi-Fi connection.

Warnings & safety notice

Before using this equipment, be sure to read this manual carefully.

Withings U-Scan Nutrio Cartridge is not a medical device, and is not intended for use in the diagnosis of disease or conditions or in the cure, mitigation, treatment or prevention of disease. Self-diagnosis and self-treatment can be dangerous. Always consult your doctor in case of questions or doubts.

- Withings U-Scan Nutrio Cartridges are only compatible with Withings U-Scan Reader.
- The data provided is for informational purposes only, and is not intended to replace the results of laboratory medical analysis tests.

EN

- This cartridge is intended for adults only.
- Samples to be used with the Withings U-Scan Nutrio Cartridges are exclusively human urine.
- Do not use if you observe that the packaging is damaged / cracked. Use of a damaged product could cause injury or equipment failure.
- Follow the operational and storage conditions indicated in the technical specifications section of this manual. Otherwise, measurement accuracy may be impacted.
- Do not use accessories, detachable parts, or materials that are not described in this guide.
- Certain substances or intakes can affect the accuracy of the results. Check the Interfering molecules part of this manual for more information.
- Discard the Withings U-Scan Nutrio Cartridges 90 days after opening
- Do not install or use the Withings U-Scan Nutrio Cartridges after expiry date. Check the expiry date on the aluminium pouch of the cartridge or on the packaging box of the cartridge.
- Do not expose the Withings U-Scan Nutrio Cartridge to humidity
- A cleaning detergent is provided within the cartridge packaging. Please check the security guidelines before using.
- The cleaning detergent is only intended to be used with the U-Scan station.
- Do not use this product when on periods or when you have blood in your urine.


The Withings U-Scan Nutrio Cartridge | Overview

EN

Content of the Withings U-Scan Nutrio Cartridge packaging.

The Withings U-Scan Nutrio Cartridge packaging includes the following components. If something is missing, please contact us at support.withings.com

U-SCAN CARTRIDGE Box content

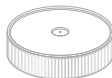Withings U-Scan Cartridge

Protective gloves

Cleanser powder

Product guide

Chemistry performance

Please check the performance on our website with this link

<https://support.withings.com/hc/en-us/articles/27579993272849-U-Scan-Nutrio-Chemistry-Performance>

Interfering molecules

Certain substances or intakes can affect the accuracy of the results. Measures should not be taken when taking them. The substances are the followings:

- Captopril
- Nitrofurantoin
- Phenazopyridine
- Riboflavin

In case of behaviours known to increase creatinine concentration in urine, or of pathology triggering bilirubin and blood in urine, U-Scan should not be used.

Urine specimens with a pH of < 4 may increase the specific gravity value; specimens with a pH > 7,5 may decrease the specific gravity value reported by the U-Scan test.

EN


Setup Process and Maintenance

EN

1 – Charge your Withings U-Scan Reader

To charge the Reader, first plug in the U-Scan station with the supplied USB-C cable and follow these steps:

1. Place the U-Scan inside of the U-Scan station,
 - a. The sample collection point should face downward with the shell side aligned with the charging wall of the U-Scan station
2. Close the U-Scan station.
3. The status light will turn orange as it charges and you will receive a notification once your device is fully charged.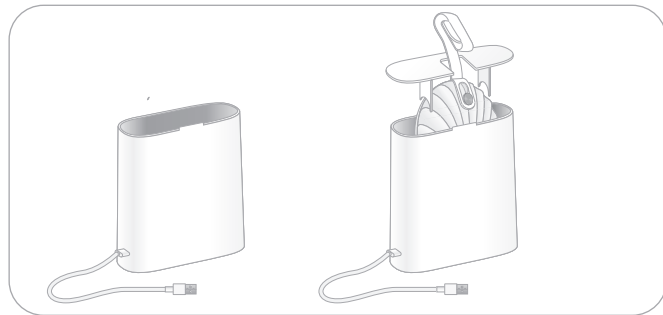


EN

2. Cartridge installation

To install the cartridge, you can launch the cartridge installation process from your Withings app and follow these steps:

1. Make sure your Withings U-Scan Reader is opened.
2. Remove your Withings U-Scan Nutrio Cartridge from its packaging and insert the Cartridge into the Reader.
3. While gently pushing down, rotate the Cartridge in either direction until you feel it lock in place with the Reader.
4. The indicator light will change from blue to green to confirm that the Cartridge has been activated.
5. To close the device, hold it in your palm with the smooth side facing up and turn the Reader cover clockwise. The Withings U-Scan Nutrio Cartridge is now properly installed in the Reader.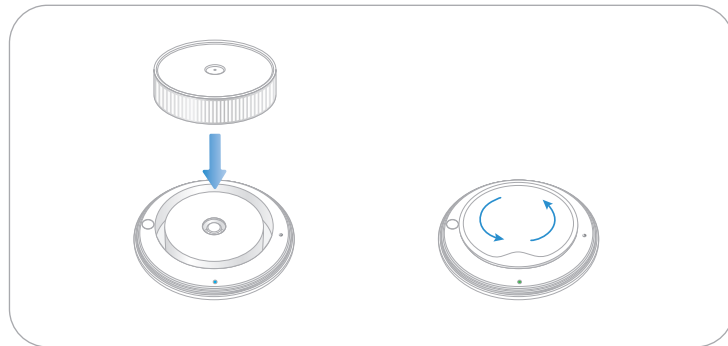


Setup Process and Maintenance

EN


3 – Toilet installation

To install the U-Scan with your toilet, follow these steps:


1. Hold the Withings U-Scan Reader in your palm, with the smooth side facing up and the metal anchor point facing inward. Close the device with your upper palm by turning the Reader cover clockwise.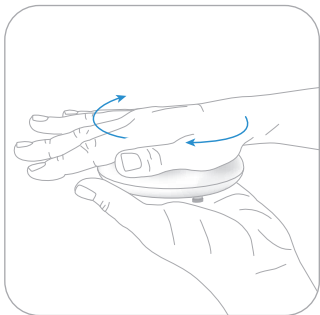


2. The Withings App will ask you what type of toilet you have. This information will allow you to define which size toilet clip you will have to use.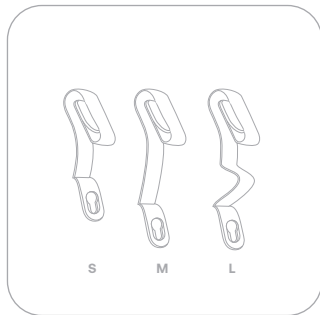

3. Attach the toilet clip to the metal anchor point by sliding the anchor point into the smaller slot of the clip to ensure a secure fit. If the device doesn't slide then the clip is facing the wrong direction.

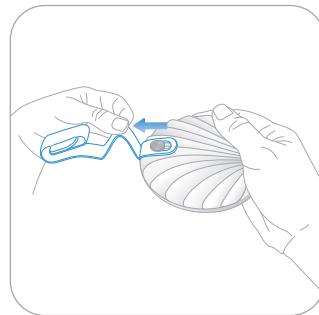4. Once the toilet clip has been correctly attached, lift up the toilet seat and attach the clip to the front/center of the toilet rim.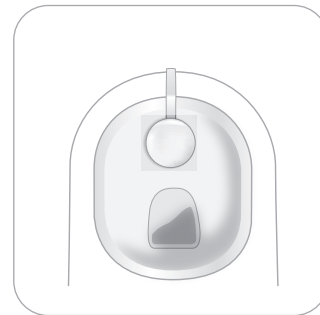

EN

Setup Process and Maintenance | 4. Changing the Cartridge

EN

We advise that you use the pair of protective gloves provided.

To change your Withings U-Scan Nutrio Cartridge, there are two possible scenarios:

- If you have not deep cleaned your device during the previous month using the station and the app tutorial, you will have to proceed with a cleaning before changing the cartridge following the cleaning process from your app.
- If you have deep cleaned your device in the last 30 days using the station and the app tutorial, you can change the cartridge following the changing cartridge tutorial from your app.

If you need to proceed with a deep clean and change the cartridge, here are the steps to follow:

1. Use the toilet clip to remove the device from the toilets.
2. Follow the deep cleaning instructions described in the following part of this manual guide.
3. After following deep cleaning and rinsing instructions, dry the U-Scan with a kitchen or paper towel.
4. Open the U-Scan Reader, remove the old Cartridge and dispose of it. Please note that it is not recyclable.
5. To install the new Cartridge, remove it from the aluminium pouch.
6. Rotate the Cartridge in either direction while gently pushing down until you feel it locked in place with the Reader.
7. The status light will change from blue to green to confirm that the Cartridge has been activated and ready to use.
8. Close the device by turning the Reader cover clockwise.
9. Place the new cartridge in the Reader and close the reader.
10. Place the device back in the toilet.

EN

If you only need to change the cartridge, here are the steps to follow:

1. Use the toilet clip to remove the device from the toilets.
2. If needed, rinse the Reader under cold water and clean it with a mild soap. Make sure that you dry the device completely before opening it.
3. To open the Reader, hold it in your palm with the smooth side facing up and rotate the cover in a counterclockwise direction.
4. Remove the old Cartridge and dispose of it. Please note that it is not recyclable.
5. To install the new Cartridge, rotate the Cartridge in either direction while gently pushing down until you feel it lock in place with the Reader.
6. The status light will change from blue to green to confirm that the Cartridge has been activated and ready to use.
7. Close the device by turning the Reader cover clockwise.
8. Place the device back in the toilets.

Setup Process and Maintenance | 5. Cleaning the Withings U-Scan System

EN

Warning:

Please only use mild soap, water, and the cleaning solution provided with the Nutrio Cartridge to clean the U-Scan. Do not apply any other cleaning products, as they may affect the U-Scan's lifespan and accuracy. Be cautious when cleaning the toilet bowl to avoid accidentally applying any unwanted cleaning products to the U-Scan.

Daily cleaning

- The device surface is designed to be rinsed with each toilet flush.
- If necessary, you can clean the device with a mild soap and water. Please avoid any products containing bleach, as they could deteriorate your device.

Deep cleaning

Deep cleaning of the device is highly recommended and will be suggested by the app after 3 months of use and before each cartridge change. To deep clean your Withings U-Scan, follow the cleaning device process from your app:

1. Use the U-Scan toilet clip to remove the device from the toilets.
2. Put the provided cleaning detergent* from the pouch and 2/3 cup (150mL) of water inside the U-Scan station, according to the level line drawn in the station.
3. Plug in the U-Scan station.

4. Place the U-Scan Reader with the right position in the U-Scan station for 5 hours for it to be cleaned (see picture).


5. Take out the U-Scan reader, rinse it under running water for 10 seconds. You should hear the device pumping.

6. Dry it with a kitchen or paper towel. Be careful not to use paper toilets, which may block the pumping inlet.

7. Follow the instructions to change the cartridge if necessary.

8. Place the device back in the toilets.

EN

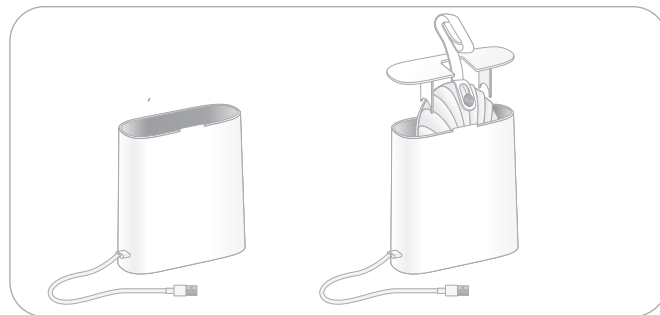Setup Process and Maintenance | 6. Storing the Withings U-Scan System

EN

Please follow the deep cleaning instructions provided in your app before storing the device. After completing the deep cleaning process with the U-Scan station, rinse your product under running water for 10 seconds. Ten minutes later, the device will purge the rinsing water, and you can store the U-Scan according to the storage conditions - in a dry place and out of direct sunlight.

7. Cartridge Lifespan

The cartridge can be used for 15 months following the production date, after which it will be expired and therefore no longer usable. You can find the expiration date on your cartridge packaging, on the cartridge sleeve, on the cartridge itself, and on the application after installing the cartridge.

After installing a cartridge, you have a maximum of 3 months to use it. After this date, you will no longer be able to take measurements. You can find this date on the application after installing your cartridge.

If you receive a cartridge that has expired or will expire in less than 3 months, please contact our customer service department to receive a new cartridge. If you store your cartridge for too long, you will not be able to install it after the expiration date, or even run measurements with it.

8. Troubleshooting

If you cannot fix the problem using the following troubleshooting instructions, please contact Withings or visit: withings.com/support.

EN

Problem	Solution
WiFi does not seem to work	Device is out of range from the Wi-Fi source. Please get your smartphone and device closer to your Wi-Fi source.
Toilet clip does not fit my toilet	3 different sizes of toilet clips are provided. Try a different size if the one selected does not seem correct.
Bluetooth does not seem to work	Smartphone is out of range. Please bring your smartphone closer to your device. Smartphone's Bluetooth is OFF. Please switch your smartphone's Bluetooth ON.
My results seem incorrect.	Your results may vary depending on the medication, food or activities you take. See the list of interferences that can affect the results.
The device does not detect my urine.	Make sure to urinate on the device. If it is not responding, try to charge your product thanks to the U-Scan Station.
The device does not recognize me.	If Automatic recognition always fail, you might prefer using the manual measurement.
I have missed my testing window.	You can launch a manual measurement or wait until your testing window is open. Your test will be automatically rescheduled.

General information

EN

Technical specifications

Product name: Withings U-Scan Nutrio Cartridge

Model: CWPA01

Dimension: Ø 6,4cm

Weight: 13 grams

Shelf Life in pouch (in specified conditions of storage):
12 months

In-Use-Stability (in specified conditions of operation):
3 months

Number of test strips: 45

Operating Conditions:

- Temperature range: 2°C to 30°C (35,6 to 86°F)
- Humidity range: 10% to 90%
- Keep out from the humidity and sunlight

Water and splash resistant: Please avoid excess water exposure with the cartridge, as this could damage the cartridge.

Storage Conditions:

- Temperature range: 2°C to 30°C (35,6 to 86°F)
- Humidity range: 10% to 90%
- Keep out from the humidity and sunlight

European Union - Declaration of Conformity

Withings hereby declares that the device Withings U-Scan Nutrio Cartridge is in conformity with the essential requirements of applicable EU Directives and Regulations. The full text of the declaration of conformity can be found at: withings.com/compliance.

? Need help?

➔ withings.com/support

EN

Warranty

Withings Fifteen (15) Months Limited Warranty - Withings U-Scan

Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, ("Withings") warrants the Withings-branded product ("Withings Product") against defects in materials and work-manship when used normally in accordance with Withings' published guidelines for a period of FIFTEEN (15) MONTHS from the date of original retail purchase by the end-user purchaser ("Warranty Period"). Withings' published guidelines include, but are not limited to, information contained in technical specifications, safety instructions or Quick Start Guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

Australia: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

Security

Withings recommends that you add a passcode (personal identification number [PIN]), Face ID or Touch ID (fingerprint) to your mobile device as an added layer of security. It is important to secure your phone since it will be storing personal health information. Concerning the Withings U-Scan Reader, users will also receive additional update notifications on the Withings app, and updates are delivered wirelessly, encouraging rapid adoption of the latest security fixes. Users are able to see the currently installed software version in the Withings app, under Devices > U-Scan. This tab also indicates if an update is available.

Do not install the device on a smartphone that you do not own.

Do not use a public Wi-Fi network you are unfamiliar with. Use a trusted Wi-Fi network with your device.

Withings also recommends upgrading your Withings app when an upgrade is available.

The Withings application is not intended to be used on a computer. No indication of anti-virus software is needed.

Only use official app stores to download the Withings companion app. If in doubt, use the link go.withings.com.

Ecological information

After use, the cartridge should be disposed of in general waste following local regulations to avoid any contamination.

EN

EN

Equipment symbol description

	Catalog number
	Serial Number
	Consult instructions for use or consult electronic instructions for use
	This product complies with the essential requirements of applicable EU Directives and Regulations
	Humidity limitation
	Temperature limit
	Upper and lower limits of atmospheric pressure
	Model Number
	Keep away from sunlight
	Keep dry
	Do not use if the package is damaged and consult instructions for use
	Non recyclage content
	Use-by date
	Health hazard

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2024 Withings. All rights reserved. Aug. 2024