EN
FR
DE

WITHINGS

PRODUCT GUIDE

INSTRUCTIONS FOR USE

EN – US & NON-US REGIONS

Content

3

INTRODUCTION

Important notice

DISCLAIMER

About the Withings U-Scan Nutrio Cartridge

Warnings & safety notice

Chemistry performance

Interferences

4

4

4

4

5

6

6

THE WITHINGS U-SCAN NUTRIO CARTRIDGE

7

Overview

7

Content of the Withings U-Scan Nutrio Cartridge
packaging

7

Set-Up Process and Maintenance

8

1. Cartridge installation

8

2. Changing the Cartridge

9

3. Cartridge Lifespan

9

GENERAL INFORMATION

12

Technical specifications

12

European Union – Declarations

of Conformity

12

Warranty

13

Security

13

Equipment symbol description

14

EN

Introduction

Important notice

Thank you for choosing Withings U-Scan. Before using Withings U-Scan Nutrio Cartridge, review the information in this guide. You may also find this guide online at <https://www.withings.com/guides>. Retain this documentation for future reference. Installation instructions are available within the Quick Start Guide provided alongside this Product Guide. Please contact Withings when in need of assistance for setting up, using or maintaining the device or to report unexpected operations or events.

For more information on the Withings U-Scan System, please refer to the Withings U-Scan Reader Product Guide.

Please note that Withings U-Scan Nutrio Cartridge is a wellness product. Withings U-Scan Nutrio Cartridge is not a medical device, and is not intended for use in the diagnosis of disease or conditions or in the cure, mitigation, treatment or prevention of disease.

Disclaimer

Information in this guide may change without notice. You will need an iOS (16.0 or higher) or Android (9.0 and higher) tablet or smartphone to install your Withings U-Scan Nutrio Cartridge. It should be noted that it is possible to launch a measurement without a mobile device, the use of a mobile device is required for the installation of

the Withings U-Scan System, and to synchronise and consult your results.

About the Withings U-Scan Nutrio Cartridge

The Withings U-Scan Nutrio Cartridge is a consumable component (cartridge) within the U-Scan System and works in tandem with the application (Withings App) and the U-Scan Reader. The Nutrio Cartridge will use the evolution of certain urinary parameters (pH, Ketones, Vitamin C and Specific Gravity) to provide you with personalized recommendations in terms of nutrition and sports activities. Please note that the Withings U-Scan System, needs all components (the Withings U-Scan Reader and a Withings U-Scan Nutrio Cartridge) to complete the urine analysis. If you have any questions related to the set-up, maintenance and use of the Withings U-Scan Reader, please check in the manual guide of the Withings U-Scan Reader.

Disclaimer

Warnings & safety notice

Before using this equipment, be sure to read this manual carefully.

Withings U-Scan Nutrio Cartridge is not a medical device, and is not intended for use in the diagnosis of disease or conditions or in the cure, mitigation, treatment or prevention of disease. Self-diagnosis and self-treatment can be dangerous. Always consult your doctor in case of questions or doubts.

- U-Scan Nutrio pH measurements can only be used for self-tracking purposes, it can't be used for diagnosis of any disease or condition including clinical dehydration, kidney disease, diabetes, any deficiency or any other disease and conditions. U-Scan Nutrio pH measurements are not intended for medical use, including self-diagnosis or consultation with a doctor, and are only designed for general fitness and wellness purposes.
- U-Scan Nutrio Specific Gravity measurements can only be used for self-tracking purposes, it can't be used for diagnosis of any disease or condition including clinical dehydration, kidney disease, diabetes, any deficiency or any other disease and conditions. U-Scan Nutrio Specific Gravity measurements are not intended for medical use, including self-diagnosis or consultation with a doctor, and are only designed for general fitness and wellness purposes. If you suspect clinical dehydration, please consult your healthcare provider as U-Scan Nutrio is not a diagnostic tool for clinical hydration status.
- U-Scan Nutrio Vitamin C measurements can only

be used for self-tracking purposes it can't be used for diagnosis of any disease or condition including clinical dehydration, kidney disease, diabetes, any deficiency or any other disease and conditions. U-Scan Nutrio Vitamin C measurements are not intended for medical use, including self-diagnosis or consultation with a doctor, and are only designed for general fitness and wellness purposes.

- U-Scan Nutrio Ketone levels can only be used for self-tracking purposes, it can't be used for diagnosis of any disease or condition including clinical dehydration, kidney disease, diabetes, any deficiency or any other disease and conditions. U-Scan Nutrio measurements are not intended for medical use, including self-diagnosis or consultation with a doctor, and are only designed for general fitness and wellness purposes.
- Withings U-Scan Nutrio Cartridges are only compatible with Withings U-Scan Reader.
- The data provided is for informational purposes only, and is not intended to replace the results of laboratory medical analysis tests.
- This cartridge is intended for adults only.
- Samples to be used with the Withings U-Scan Nutrio Cartridges are exclusively human urine.
- Do not use if you observe that the packaging is damaged / cracked. Use of a damaged product could cause injury or equipment failure.
- Follow the operational and storage conditions indicated in the technical specifications section of this manual. Otherwise, measurement accuracy may be impacted.

- Do not use accessories, detachable parts, or materials that are not described in this guide.
- Certain substances or intakes can affect the accuracy of the results. Check the Interfering molecules part of this manual for more information.
- Discard the Withings U-Scan Nutrio Cartridges 90 days after opening
- Do not install or use the Withings U-Scan Nutrio Cartridges after expiry date. Check the expiry date on the aluminium pouch of the cartridge or on the packaging box of the cartridge.
- Do not expose the Withings U-Scan Nutrio Cartridge to humidity
- A cleaning detergent is provided within the cartridge packaging. Please check the security guidelines before using.
- The cleaning detergent is only intended to be used with the U-Scan station.
- Do not use this product during menstruation or if there is blood in your urine.

Chemistry performance

Please check the performance on our website with this link

<https://support.withings.com/hc/articles/27579993272849>

Interferences

The consumption of the following exogenous substances can lead to false positive results for either vitamin C and/or ketones due to their high pigmentation. Phenazopyridine, Captopril and Riflovin will tend to overestimate vitamin C levels, whereas Captopril and Nitrofurantoin may lead to an overestimation of ketone levels.

In case of behaviors known to increase creatinine, bilirubin or blood concentration in urine, U-scan should not be used because the results won't be accurate. This includes menstruation and intensive workout leading to extreme muscle weight gain.

pH values above 7.5 can lead to an underestimation of the fluid intake level.


Acetic acid, citric acid and sodium hypochlorite should never be used to clean U-Scan reader and if used to clean the toilet bowl, the latter should be thoroughly rinsed with water before putting back the U-scan reader.

The Withings U-Scan Nutrio Cartridge | Overview

Content of the Withings U-Scan Nutrio Cartridge packaging.

The Withings U-Scan Nutrio Cartridge packaging includes the following components. If something is missing, please contact us at support.withings.com

U-SCAN CARTRIDGE Box content

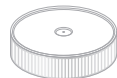Withings U-Scan Cartridge

Protective gloves

Cleaning powder

Product guide


Setup Process and Maintenance

EN

1. Cartridge installation

To install the cartridge, you can launch the cartridge installation process from your Withings app and follow these steps:

1. Make sure your Withings U-Scan Reader is opened.
2. Remove your Withings U-Scan Nutrio Cartridge from its packaging and insert the Cartridge into the Reader.
3. While gently pushing down, rotate the Cartridge in either direction until you feel it lock in place with the Reader.
4. The indicator light will change from blue to green to confirm that the Cartridge has been activated.
5. To close the device, hold it in your palm with the smooth side facing up and place the Reader cover on top. Then turn the Reader cover clockwise. The Withings U-Scan Nutrio Cartridge is now properly installed in the Reader.

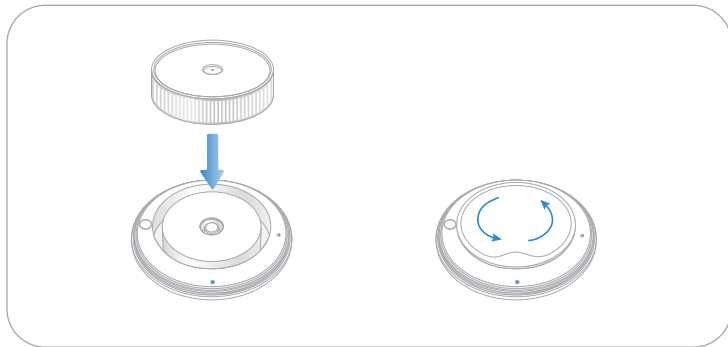Setup Process and Maintenance | 2. Changing the Cartridge

We advise that you use the pair of protective gloves provided.

We strongly advise you to clean and recharge the U-Scan Reader before changing the cartridge, using the cleaning powder* enclosed with your cartridge, the U-Scan Station and following the in-app tutorial.

EN

3. Cartridge Lifespan

The cartridge can be used for 15 months following the production date, after which it will be expired and therefore no longer usable. You can find the expiration date on your cartridge packaging, on the cartridge sleeve, on the cartridge itself, and on the application after installing the cartridge.

After installing a cartridge, you have a maximum of 3 months to use it. After this date, you will no longer be able to take measurements. You can find this date on the application after installing your cartridge.

If you receive a cartridge that has expired or will expire in less than 3 months, please contact our customer service department to receive a new cartridge. If you store your cartridge for too long, you will not be able to install it after the expiration date, or even run measurements with it.

*Information on the cleaning detergent provided: Tergazyme® Enzyme-Active Powdered Detergent is a concentrated, enzymatic detergent for manual and soak commercial kitchen cleaning.

Unique Formula Identifier (UFI)
OS90-10CR-S00Y-G8GH

Composition/information on ingredients

Chemical name	Weight-%	CAS n°	CE n°
Sodium bicarbonate	45-50	144-55-8	205-633-8
Benzenesulfonic acid, mono-C10-16-alkyl derivs., sodium salts	10-15	68081-81-2	207-838-8 (011-005-00-2)
Sodium carbonate	7-13	497-19-8	268-356-1
Tetrasodium EDTA	0.1-1	64-02-8	200-573-9 (607-428-00-2)

Warning

Hazard statements

Harmful if swallowed.

Causes skin irritation.

Causes serious eye irritation.

Precautionary Statements - Prevention

Wash face, hands and any exposed skin thoroughly after handling. Do not eat, drink or smoke when using this product.

Wear protective gloves.

Precautionary Statements - Response

Specific treatment (see information on this label).

Eyes

IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice and attention.

Skin

IF ON SKIN: Wash with plenty of water and soap. If skin irritation occurs: Get medical advice and attention. Take off contaminated clothing and wash it before reuse.

Ingestion

IF SWALLOWED: Call a POISON CENTER or doctor if you feel unwell. Rinse mouth.

Precautionary Statements - Disposal

Dispose of contents and container according to the local relementation.

Unknown acute toxicity

Keep containers tightly closed in a dry, cool and well-ventilated place. Keep out of the reach of children.

Supplier Address

Alconox Inc.
30 Glenn St.,
Suite 309 White Plains,
NY 10603 USA 914-948-4040

Emergency telephone

ChemTel Inc.: North America: 1-888-255-3924
International: +1-813-248-0573

For more information, please check the Safety Data Sheet

<https://alconox.com/wp-content/uploads/2024/06/Tergazyme-SDS-USCA-English.pdf>

General information

EN

Technical specifications

Product name: Withings U-Scan Nutrio Cartridge

Model: CWPA01

Dimension: Ø 6,4cm

Weight: 13 grams

Shelf Life in pouch (in specified conditions of storage):
12 months

In-Use-Stability (in specified conditions of operation):
3 months

Number of test strips: 45

Operating Conditions:

- Temperature range: 2°C to 30°C (35,6 to 86°F)
- Humidity range: 10% to 90%
- Keep out from the humidity and sunlight

Water and splash resistant: Please avoid excess water exposure with the cartridge, as this could damage the cartridge.

Storage Conditions:

- Temperature range: 2°C to 30°C (35,6 to 86°F)
- Humidity range: 10% to 90%
- Keep out from the humidity and sunlight

European Union - Declaration of Conformity

Withings hereby declares that the device Withings U-Scan Nutrio Cartridge is in conformity with the essential requirements of applicable EU Directives and Regulations. The full text of the declaration of conformity can be found at: withings.com/compliance.

? Need help?

→ withings.com/support

Warranty

Withings Fifteen (15) Months Limited Warranty - Withings U-Scan

Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, ("Withings") warrants the Withings-branded product ("Withings Product") against defects in materials and work-manship when used normally in accordance with Withings' published guidelines for a period of FIFTEEN (15) MONTHS from the date of original retail purchase by the end-user purchaser ("Warranty Period"). Withings' published guidelines include, but are not limited to, information contained in technical specifications, safety instructions or Quick Start Guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

Australia: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

Security

Withings recommends that you add a passcode (personal identification number [PIN]), Face ID or Touch ID (fingerprint) to your mobile device as an added layer of security. It is important to secure your phone since it will be storing personal health information. Concerning the Withings U-Scan Reader, users will also receive additional update notifications on the Withings app, and updates are delivered wirelessly, encouraging rapid adoption of the latest security fixes. Users are able to see the currently installed software version in the Withings app, under Devices > U-Scan. This tab also indicates if an update is available.

Do not install the device on a smartphone that you do not own.

Do not use a public Wi-Fi network you are unfamiliar with. Use a trusted Wi-Fi network with your device.

Withings also recommends upgrading your Withings app when an upgrade is available.

The Withings application is not intended to be used on a computer. No indication of anti-virus software is needed.

Only use official app stores to download the Withings companion app. If in doubt, use the link go.withings.com.

EN

Ecological information

After use, the cartridge should be disposed of in general waste following local regulations to avoid any contamination.

EN

Equipment symbol description

Catalog number

Serial Number

Consult instructions for use or consult electronic instructions for use

This product complies with the essential requirements of applicable EU Directives and Regulations

Humidity limitation

Temperature limit

Upper and lower limits of atmospheric pressure

Model Number

Keep away from sunlight

Keep dry

Do not use if the package is damaged and consult instructions for use

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2025 Withings. All rights reserved. Aug. 2025