EN
FR
DE

WITHINGS

PRODUCT GUIDE

INSTRUCTIONS FOR USE

EN – US & NON-US REGIONS

Content

2-3

INTRODUCTION

Important notice

DISCLAIMER

About the Withings U-Scan Reader

About the Withings U-Scan System

Warnings & safety notice

THE WITHINGS U-SCAN SYSTEM

Overview

U-Scan Reader Box content

U-Scan Cartridge Box content

Component names and functions

Withings U-Scan Reader

Status Light

Button

SETUP PROCESS

1 – Download the Withings App
and create a Withings account

2 – Open your Withings U-Scan Reader
with the suction cup

3 – Charge your Withings U-Scan
Reader

4 – Pair and link your U-Scan Reader

5 – Cartridge installation

6 – Toilet installation

4 HOW TO PERFORM A URINALYSIS

4

Measurement Plan

4

Using the automatic measurement

4

Launching a manual measurement

4

via the application

5

MAINTENANCE

6

Cleaning and maintenance

6

Troubleshooting

6

Wireless information

7

GENERAL INFORMATION

8

Technical specifications

8

European Union – Declarations

8

of Conformity

10

Disposal

11

Federal Communication Commission

12

interference statement

12

Warranty

12

Security

12

Equipment symbol description

12

12

13

13

14

15

16

18

18

19

19

19

20

20

22

22

23

24

24

24

24

25

25

25

26

26

26

27

Introduction

Disclaimer

Important notice

EN

Thank you for choosing Withings U-Scan Reader. Before using Withings U-Scan Reader, review the information in this guide. You may also find this guide online at <https://www.withings.com/guides>

Retain this documentation for future reference. Installation instructions are available within the Quick Start Guide. Provided alongside this Product Guide.

Please contact Withings for assistance with setting up, using, or maintaining the device or to report unexpected operations or events.

Information in this guide may change without notice. You will need an iOS (16.0 or higher) or Android (8.0 or higher) tablet or smartphone to install your Withings U-Scan Reader. Thereafter, the device can be used without your mobile device thanks to Wi-Fi and Bluetooth® connections. While a mobile device is not required to launch a measurement, it is necessary for installation, synchronization, and viewing results.

About the Withings U-Scan Reader

The Withings U-Scan Reader is an automatic at-home urine analyzer. Used with Withings U-Scan Cartridges, it tracks various urinary parameters.

Measurement results are displayed in the Withings App.

About the Withings U-Scan System

To use the Withings U-Scan System, you need a Withings U-Scan Reader and a Withings U-Scan Cartridge.

You must create an account on the Withings App, install the app, and pair your device via Bluetooth.

Follow the installation instructions in the “Setup Process” section. The Withings U-Scan Reader features Bluetooth Low Energy (BLE) and Wi-Fi connectivity.

Test results are transmitted via Wi-Fi, meaning the device requires a functional Wi-Fi connection.

Warnings & safety notice

Before using this equipment, read this manual carefully. The Withings U-Scan Reader is not a medical device and is not intended for diagnosis, treatment, or disease monitoring. Self-diagnosis and self-treatment can be dangerous—always consult your doctor if you have questions.

- Compatible only with Withings U-Scan Cartridges.
- Data provided is for informational purposes only and does not replace laboratory medical tests.
- Intended for adults only.
- Only human urine samples should be used.
- Do not use if the device is damaged, as this may cause injury or equipment failure.
- Do not attempt to repair or modify the device. Do not open or disassemble it for battery replacement.
- Follow the cleaning and maintenance instructions in this guide. Using unapproved cleaning agents may damage the device.
- The charger's power cable may cause strangulation; keep it away from children and pets.
- Follow the operational and storage conditions in the technical specifications section to ensure measurement accuracy.

- Only use accessories and parts specified by Withings. Using unauthorized components may lead to increased electromagnetic emissions or decreased immunity, causing improper operation.

- Use the provided power cord to plug in the station.

- Do not conduct a test while the device is charging.

- Ensure the device is properly closed before conducting a test.

- Do not perform tests during menstruation, as blood may contaminate the fluidic circuit.

- Certain substances or intakes may affect result accuracy; refer to the Withings U-Scan Cartridge instruction manual.

- Avoid exposing the device to strong impacts or vibrations (e.g., dropping or shaking).

EN

EN

U-SCAN READER Box content

Withings U-Scan Reader

Withings U-Scan Station

Suction Cup

Toilet Clip (sizes: S, M, L)

Quick start guide
and product guide

USB-C Power cable

U-SCAN CARTRIDGE Box content

EN

Withings U-Scan Cartridge

Protective gloves

Cleanser powder


Product guide

If something is missing,
please contact support.withings.com

EN

WITHINGS U-SCAN READER


Inside view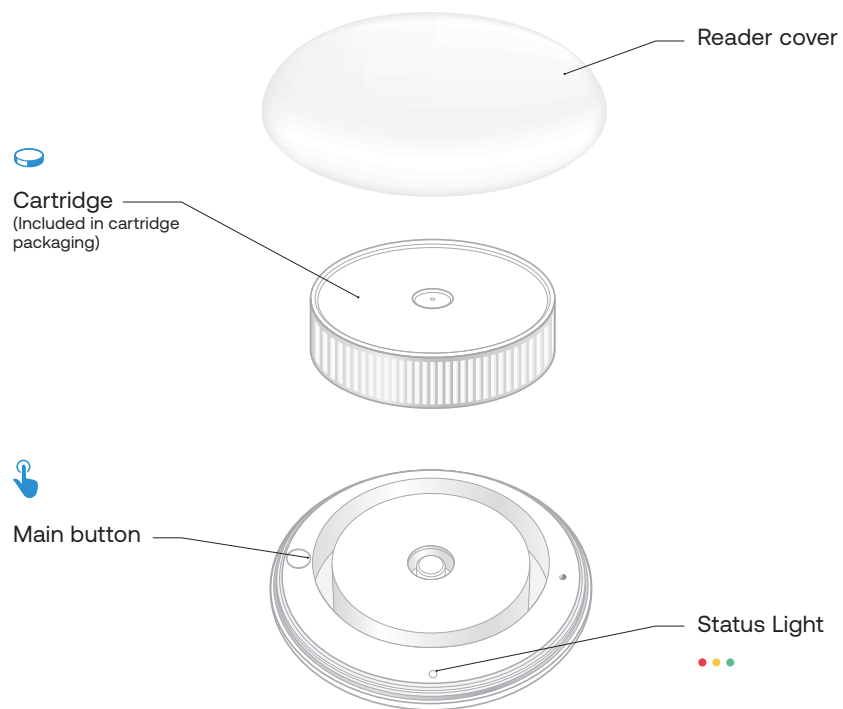

WITHINGS U-SCAN READER

Back view

EN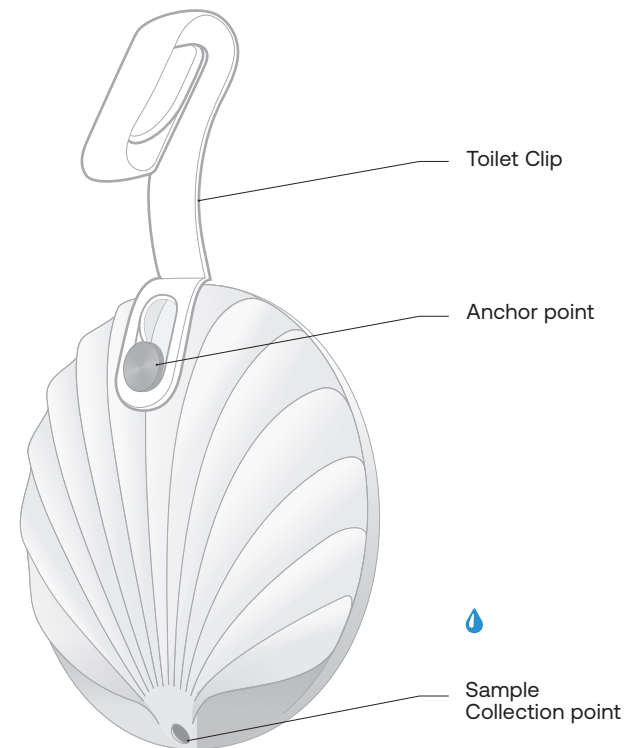

The U-Scan System | Component names and functions

EN

STATUS LIGHT

General

Blue breathing		Waiting for App installation
Blue continuous		Installation ongoing
Blue flashing		Bluetooth update in progress
Yellow flashing light		WiFi synchronization or WiFi update in progress
Red continuous		Error
Green blinking		Synchronization
Green continuous		Success
Orange blinking		Low battery

Manual Launch

Green slow blinking		Ready to collect
Green continuous for 20s		Enough urine sample collected
Red continuous for 20s		Fail collecting enough urine sample

Charge (once in plugged station)

Orange continuous		Battery charging
Green continuous		Battery charged

BUTTON

1 second = 1 flash on the device.

EN

Wake Up	1-second press, the status light flashes blue if the device is not installed yet, green otherwise
Factory Reset	Press and hold for 10 seconds, then once the red LED lights up, press 5 times for 1 second each time.
Hard Reset	20-seconds press, the status light will turn red after 10 seconds

For further information, please consult our help center.

Setup Process

12 - 13

1 – Download the Withings App and create a Withings account

EN

Download and install the Withings App on your phone before first use. Follow the on-screen instructions to sign in with your existing Withings account or create a new one.

2 – Open your Withings U-Scan Reader with the suction cup

Place the suction cup at the center of the Reader's smooth face. Hold the suction cup with one hand, and the other side of the Reader with the palm of your second hand (picture below). Rotate the suction cup anti clockwise to unscrew the upper side of the Reader.

3 – Pair and link your Withings U-Scan Reader

Once connected to your Withings account, you must install your device in the Withings App.

To install your device, follow these steps:

1. Open the Withings App.
2. Go to the Devices tab and tap the “+” icon in the top right corner.
3. Select U-Scan and tap “Install.”
4. If the device is asleep, press and hold the power button for one second.



5. The status light will glow blue when the device is ready to connect to your phone.

6. The Withings App will search for your device via Bluetooth. Once found, you will be prompted to pair your Withings U-Scan Reader with your mobile device.

7. Tap “Accept” to pair the device. The status light will turn solid blue once connected.

8. After pairing, the Withings U-Scan Reader will update its internal firmware. Wait for the update to complete, then follow the instructions to connect your device to your Wi-Fi network.

EN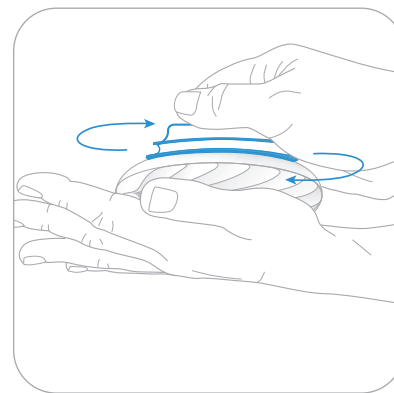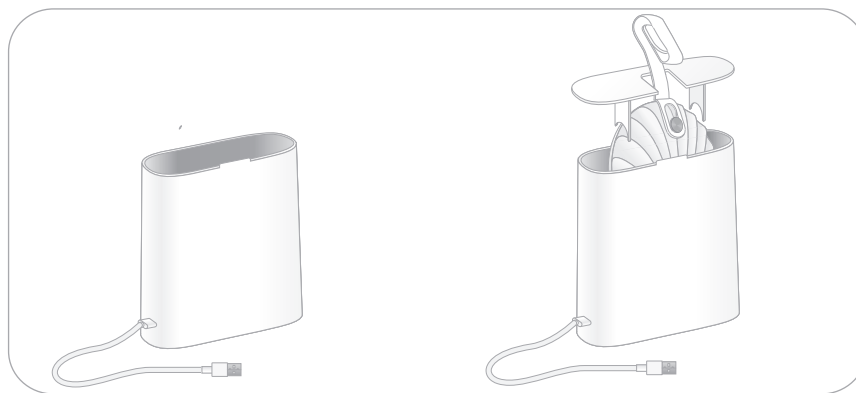

4 – Charge your Withings U-Scan Reader

EN

1. Plug in the Withings U-Scan Station with the dedicated USB-C cable.
2. If the U-Scan Reader is open, close it by reversing instructions provided in *SETUP PROCESS* section 2. There are closure marks on each side of the Reader. Make sure to align the arrow on the lid with one of the dots on the backside of the U-Scan. Place the closed Withings U-Scan Reader in the Withings U-Scan Station.



3. Close the Withings U-Scan Station. The status light of the Station will turn white.
4. The status light on the Withings U-Scan Reader, visible through the Station lid, will turn orange while charging and green when fully charged. You will receive a notification once charging is complete.

5 – Cartridge installation

1. Open the Withings U-Scan Reader with the suction cup following instructions provided in *SETUP PROCESS* section 2.
2. Remove the Withings U-Scan Cartridge from its packaging and insert it into the Reader.
3. While gently pushing down, rotate the Cartridge in either direction until it locks in place.
4. The status light will change from blue to green, confirming that the Cartridge is activated and ready to use.

5. Close the U-Scan Reader with the suction cup reversing instructions provided in *SETUP PROCESS* section 2. There are closure marks on each side of the Reader. Make sure to align the arrow on the lid with one of the dots on the backside of the Reader.

EN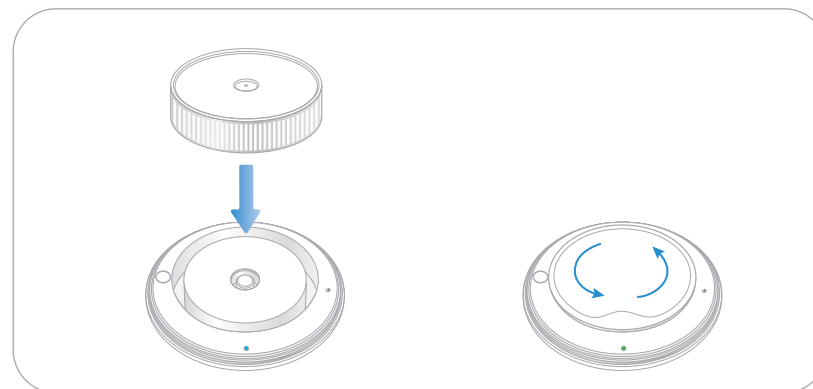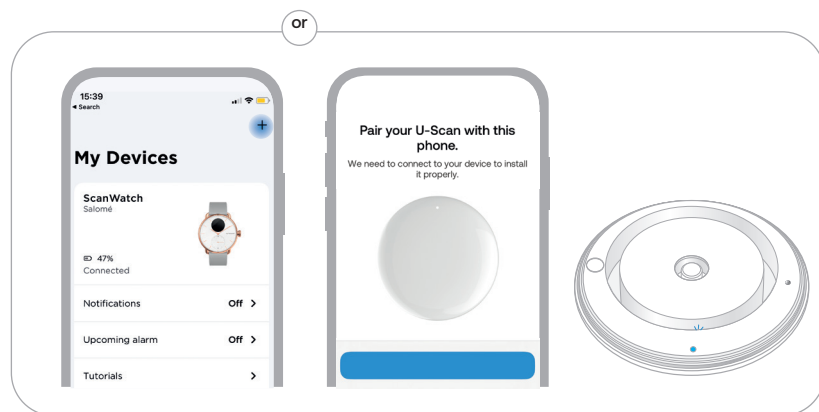

6 – Toilet installation

EN

To install the Withings U-Scan Reader in your toilet, follow these steps:





1. If the U-Scan Reader is not already closed, close it with the suction cup reversing instructions provided in *SETUP PROCESS* section 2.
2. There are closure marks on each side of the Reader. Make sure to align the arrow on the lid with one of the dots on the backside of the U-Scan.

3. The Withings App will ask you what type of toilet you have. This information will allow you to define which size toilet clip you will have to use.

4. Attach the toilet clip to the metal anchor point by sliding the anchor point into the smaller slot of the clip to ensure a secure fit. If the device doesn't slide, it means that the clip is facing the wrong direction.

5. Once the toilet clip has been correctly attached, lift up the toilet seat and attach the clip to the front/ center of the toilet rim.

EN

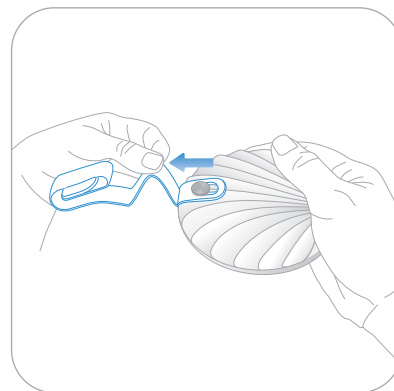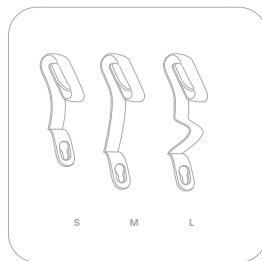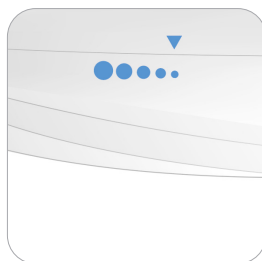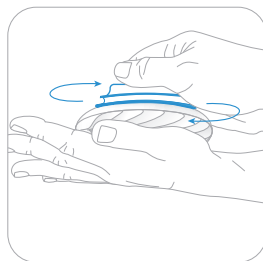EN

Measurement Plan

Upon purchasing the Withings U-Scan Reader, you will gain access to various subscription models that determine the frequency of measurements you can perform each week.

Using your habit information and based on the Cartridge you have, we will suggest specific days and times for measurements to create a personalized measurement plan. Some tests must be performed immediately upon waking or at the end of the day after your last meal.

The frequency of Withings U-Scan Cartridge deliveries will match the measurement frequency selected at the time of purchase. We recommend following the measurement plan provided in the Withings App. However, you will still have the flexibility to launch manual measurements outside of the prescribed plan.

There are two ways to launch a measurement with the Withings U-Scan System: automatic and manual, via the application.

You will receive results within a maximum of 20 minutes after sample collection.

Using the automatic measurement - Available soon

Automatic Measurement is a feature making measurements super easy to take. Based on specific parameters you will be taught on the app, the reader will automatically detect your urine when you are the one urinating on the product.

When you choose the automatic measurement option, the Withings U-Scan Reader will activate to receive a sample based on the measurement windows selected in the App. This mode can be activated either at the end of the Cartridge installation process or in the device settings in the Withings App.

Please note that U-Scan samples will only be collected during the selected measurement windows. Be aware of these windows before using the toilet to ensure successful sample collection. If you are unsure, check the Withings App for measurement window details.

Launching a manual measurement via the application

EN

We recommend performing Withings U-Scan Reader measurements within the designated measurement windows of your plan, but you can also launch a manual measurement at any time. Please note the scheduling rules outlined in your Measurement Plan in the Withings App.

To manually launch a measurement, follow these steps:

1. Open the Withings App.
2. Tap your U-Scan Mission card on the Home tab.
3. Select "Launch Measurement." Keep your mobile device close to the Withings U-Scan Reader during the process.
4. Once the Withings U-Scan Reader is detected via Bluetooth, proceed to urinate.
5. The device will collect a test sample within a maximum window of 2 minutes. The status light will blink green while waiting for the sample and turn solid green once collection is successful.
6. You can flush your toilets directly after
7. You can leave the device in your toilets even when not using it

Warning:

EN

Use only mild soap, water, and the cleaning solution provided with the Cartridge to clean the Withings U-Scan Reader. Do not use any other cleaning products, as they may affect the device's lifespan and accuracy. Be cautious when cleaning the toilet bowl to avoid accidentally applying unwanted cleaning products to the Withings U-Scan Reader. Do not take a measurement with the Withings U-Scan Reader if you have blood in your urine.

How to clean Withings U-Scan Reader

Daily cleaning

- The device surface is designed to be rinsed with each toilet flush.
- If needed, clean the device with mild soap. Avoid products containing bleach, as they may deteriorate the device.
- In addition, every day at around 11 a.m., the U-Scan will automatically flush its internal fluid system to prevent urine from stagnating inside.

Deep cleaning


Deep cleaning is highly recommended and will be suggested by the app after 3 months of use or before each cartridge change. To deep clean your Withings U-Scan Reader, follow these steps:

1. Launch the Cleaning Tutorial from your device settings
2. Use the U-Scan toilet clip to remove the device from the toilet.
3. Pour the provided cleaning detergent from the pouch and room temperature water into the Withings U-Scan Station, following the level line indicated.
4. Plug in the Withings U-Scan Station.
5. Place the Withings U-Scan Reader in the correct position inside the station and leave it for 3 hours and 30 minutes to complete the cleaning process.
6. Once the cleaning completed, remove the Withings U-Scan Reader from the station and rinse it under running water at room temperature. Launch the rinsing process from the app. The device will pump water, and the process is finished once the pump stops. You will receive a confirmation on your app. Withings U-Scan Reader will purge its internal water 20 minutes after.
7. Dry the device using a kitchen towel or paper towel. Avoid toilet paper, as it may block the collection point.
8. If needed, follow the instructions to change the cartridge.
9. Place the device back in the toilet.

Storing the Withings U-Scan System

EN

Before storing the device, follow the deep cleaning and rinsing instructions written above, following the in-app process. You can then store the Withings U-Scan Reader in a dry place, away from direct sunlight.

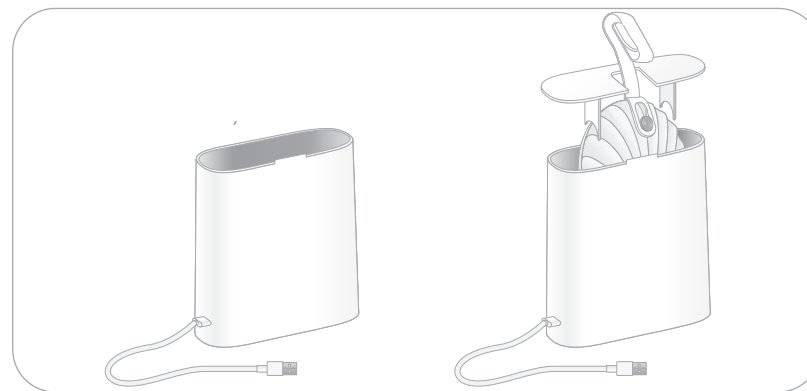Cleaning and maintenance

Troubleshooting

EN

If you cannot fix the problem using the following troubleshooting instructions, please contact Withings or visit: withings.com/support.

Problem	Solution
WiFi does not seem to work	The device is out of range from the Wi-Fi source. Please get your smartphone and device closer to your Wi-Fi source. The Wi-Fi source. You can also restart the Wi-Fi setup in the app if the problem persists.
Toilet clip does not fit my toilet	You also can follow the tutorial on changing toilets from the app settings to review the tutorial on selecting the arm.
Bluetooth does not seem to work	Smartphone is out of range. Please bring your smartphone closer to your device. Smartphone's Bluetooth is OFF. Please switch your smartphone's Bluetooth ON. If the problem persists, you can forget the device from your app and perform a 20-second reset, but please note that this should only be used as a last resort, as you will need to reinstall your product.
My results seem incorrect.	Your results may vary depending on the medication, food, or activities you take. See the list of interferences that can affect the results in the manual guide of your cartridge.
The device does not detect my urine.	Make sure to urinate on the device. You can find more advices on the Help Center article.
The device does not recognize me.	If Automatic recognition always fails, you might prefer using manual measurement.
I have missed my testing window.	You can launch a manual measurement or wait until your testing window is open. Your test will be automatically rescheduled.

General information

22 - 23

Wireless information

Mode	Frequency Band (MHz)	Maximum Output Power (dBm)
BT LE	2402-2480	9
WLAN	2412-2472	20

EN

Wireless Specifications

Wireless Technology	Bluetooth BLE
Version	Supported BT5.1
Operation Frequency	2402MHz - 2480MHz
Modulation	GFSK
Receiver sensitivity	-96dBm

The wireless communication of the device is supported by BLE and Wi-Fi communication. The communication is encrypted through an exchange of a paired key and established between the device and the Withings Application for BLE and Wi-Fi.

The communication latency between the device and the Withings Application takes less than 10 seconds when the device and the smartphone are less than 16 feet (5 meters) apart.

The communication between the device and the Withings Application is not modified with sources of interference located within 5 meters.

General information

EN

Technical specifications

Product Name: Withings U-Scan Reader
Model: WPA02
Dimension: 93 x 33 mm
Weight: 134 grams
Distance between user and device during use: 10 cm
Power source: Powered by a 3.8 Vdc Lithium-ion battery (use the wireless charging U-Scan Station included).
 DC 5V 1A power adapter (not included)
 USB C input connector
 WPC Qi 1.2 charge
 112-200kHz operation (per Qi standard)
 TX power output 3W
Battery life: Up to 3 months after a full charge.
Operating Conditions:
 - indoor use;
 - altitude up to 2,000 m;
 - temperature 2°C to 30°C / 35,6°F to 86°F;
 - maximum relative humidity of 90% for temperatures up to 30°C
Water and splash resistant: Yes
Storage Conditions:
 - Temperature range: -25°C to 70°C / -13°F to 158°F
 - Humidity range: 20% to 90
 - %Pressure range: 700-1060hPa
Expected product life: 2 years
IP protection level: IP45

Withings U-Scan Station Usage Recommendations and Warnings

Product Name: Withings U-Scan Station
Model: SWPA01
Dimensions: 130 x 56 x 135 mm
Weight: 210 g
Power Source:
 - DC 5V 1A power adapter (not included).
 - USB-C input connector.
 - WPC Qi 1.2 charge.
 - 110-250kHz operation (per-Qi standard).
 - TX power output 3W.

Placement and Connection:

- Plug the Withings U-Scan Station in a dry place and ensure the surrounding environment remains dry.
 - Do not immerse the Withings U-Scan Station in water.
 Ensure that the USB-C port and base remain dry during filling and usage.

Before Plugging in the Withings U-Scan Station:

- Check that no water is present in the USB-C port.
 - Inspect the device for cracks, scratches, or signs of deterioration. If damaged, do not use the station.

Using the Withings U-Scan Station:

- Monitor the temperature: Use the Withings U-Scan Station in a room with an ambient temperature of 0-30°C.
 - Charging issues: If the Withings U-Scan Reader does not charge properly, remove it from the station, check for water ingress, and ensure it is properly closed. If no moisture is detected, unplug and replug the Withings U-Scan Station, then reinsert the Withings U-Scan Reader.
 - Unplug after use: Always disconnect the Withings U-Scan Station from the power source after use.

Safety Notice:

- Do not open or disassemble the Withings U-Scan Station. Attempting to do so may compromise functionality and pose safety risks.
 - If charging is still not available, you can remove the plastic cover next to the button on the inside of the product and connect your USB-C cable directly to your product - only if there are no signs of moisture and no alerts have been received.
 - In case of issues, contact customer support.
 - Failure to follow these recommendations may compromise the functionality of the Withings U-Scan Reader and Station. Withings will not be held responsible for any resulting consequences.

European Union - Declaration of Conformity

Withings hereby declares that the Withings U-Scan Reader and Station comply with the essential requirements of applicable EU Directives and Regulations.
 The full text of the Declaration of Conformity can be found at: [withings.com/compliance] (<https://www.withings.com/compliance>).

Disposal

In compliance with European Directive 2012/19/EU, the device must be disposed of responsibly to reduce environmental and health risks from electronic waste.

- The symbol on the device or packaging indicates that the product must not be disposed of with household waste.
 - At the end of its life cycle, the user must deliver the device to an electronic waste collection center or return it to the retailer when purchasing a new device.
 - Proper disposal prevents negative environmental and health impacts, conserves resources, and supports energy efficiency.
 - Illegal disposal may result in administrative penalties as per local regulations. The device and its parts must be disposed of in accordance with national and regional laws.

Need help?

→ withings.com/support

Federal Communication Commission interference statement

FCC ID Withings U-Scan Reader: XNAWPA02

FCC ID Withings U-Scan Station: XNASWPA01

This Transmitter must not be co-located or operating in conjunction with any other antenna or transmitter

1. The equipment complies with FCC RF radiation exposure limits set forth for an uncontrolled environment, under 47 CFR, 2.1093 paragraph (d)(2).
 2. The U-Scan has been tested to the FCC exposure requirements (Specific Absorption Rate).

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions:

1. This device may not cause harmful interference.
 2. This device must accept any interference received, including interference that may cause undesired operation.

FCC Caution: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment.

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in residential installations.

This equipment generates, uses, and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation.

If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try the following corrective measures:

- Reorient or relocate the receiving antenna.
 - Increase the distance between the equipment and the receiver.
 - Connect the equipment to an outlet on a different circuit than the one used by the receiver.
 - Consult the dealer or an experienced radio/TV technician for assistance.

EN

General information

EN

IMPORTANT NOTE:

Radiation Exposure Statement:

This equipment complies with FCC radio radiation exposure limits for an uncontrolled environment.

- The transmitter must not be co-located or operate in conjunction with any other antenna or transmitter.
- The Country Code selection feature must be disabled for products marketed in the U.S. and Canada.

Distance between user and device during use: 10 cm

Warranty

Withings Two (2) Years Limited Warranty - Withings U-Scan

Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, FRANCE ("Withings") warrants the Withings-branded product ("Withings Product") against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEARS from the date of the original retail purchase by the end-user ("Warranty Period").

Withings' published guidelines include, but are not limited to, the information provided in this guide or the Quick Start Guide.

Withings does not guarantee that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage resulting from failure to follow the instructions related to the product's use.

Australian Consumer Law Notice (AU) : Our goods come with guarantees that cannot be excluded under the Australian Consumer Law.

You are entitled to:

- A replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage.
- A repair or replacement if the goods fail to be of acceptable quality, provided the failure does not amount to a major failure.

Security

Withings recommends that you add a passcode (personal identification number [PIN]), Face ID or Touch ID (fingerprint) to your phone to add a layer of security. It is important to secure your phone since it will be storing personal health information.

Users should follow the password and two-factor authentication guidelines when logging into the Withings App. Users will also receive additional update notifications on the device via the Withings App, and updates are delivered wirelessly, encouraging rapid adoption of the latest security fixes. Users are able to see the currently installed firmware on the Withings App, under Devices > U-Scan.

This tab also indicates if an update is available.

- Do not install the device on a smartphone that you do not own.
- Do not use a public wifi network you are unfamiliar with. Use a trusted Wi-Fi network with your device.
- Withings also recommends upgrading your Withings App when an upgrade is available.
- The Withings Mobile App is not intended to be used on a computer. No anti-virus software is needed.
- Only use official app stores to download the Withings App. If in doubt, use the link go.withings.com.

Equipment symbol description

26 - 27

EN

Catalog number

Serial Number

Consult instructions for use or consult electronic instructions for use

Model Number

This product complies with the essential requirements of applicable EU Directives and Regulations

Regulatory Compliance Mark (RCM) with ACMA's labeling requirements

Humidity limitation

Temperature limit

Atmospheric pressure limitation

Direct current

IP45

Protected against solid foreign objects of 1,0 mm ϕ and greater
Protected against water jet from all directions

Do not dispose of this product as unsorted municipal waste, take it to electronic recycling.

Do not use if the package is damaged and consult instructions for use

Packaging recyclable

Packaging recyclable

Packaging recyclable

California Energy Commission (CEC)compliant

Contains Bisphenol A and Lead, substances recognized by the State of California as causing reproductive harm. www.P65Warnings.ca.gov

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2025 Withings. All rights reserved. April. 2025