EN

FR

DE

WITHINGS

PRODUCT GUIDE

Table of Contents

ENGLISH.....	3
FRANÇAIS.....	10
DEUTSCH	18

USER GUIDE

EN

This guide explains how to use your Withings BodyScan 2. It applies to users in the United-States, Europe, United Kingdom, Switzerland, Australia, New Zealand, Turkey and Hong Kong.

3

Important notice

Before using Withings BodyScan 2, review the information in this guide.

For more information on how to use the ECG functionality with your Withings BodyScan 2, please refer to the section "About your device".

Retain this documentation for future reference.

Installation instructions are available within the Quick Start Guide provided with this guide.

Please contact Withings when in need of assistance, setting up, using or maintaining the device or to report unexpected operations or events, or visit withings.com/support.

Withings BodyScan 2 is a wellness product. It is not a medical device, and is not intended for use in the diagnosis, cure, mitigation, treatment or prevention of any diseases or conditions. Always consult your doctor for any medical concern or advice. The data provided by this device is for informational purposes only and is not intended to replace professional medical diagnosis, advice, or treatment. The Withings BodyScan 2 is intended for lay users. Weight measurement is for all ages; all other measurements are for adults aged 18 and older.

WITHINGS,
2 rue Maurice Hartmann,
92130 Issy-les-Moulineaux,
FRANCE

About the device

Withings BodyScan 2 is a connected smart scale providing the following features without medical purposes:

- Measure weight
- Calculate Body Mass Index (BMI)
- Measure whole-body and segmental body composition
- Measure electrodermal skin conductance
- Measure standing pulse rate
- Estimate vascular age
- Estimate cardio efficiency and cardio responsiveness
- Measure blood oxygen level

Withings BodyScan 2 includes electrodes intended to acquire an ECG signal used by a Withings ECG software. This ECG software is marketed as **Withings ECG Monitor 2** in Europe, UK, Switzerland, Turkey, Australia, New Zealand, and Hong Kong ; and as **Withings ECG App** in the United-States.

Please refer to the Withings ECG Monitor 2 or Withings ECG App Instructions for Use, available at withings.com/guides and in the "settings" of the Withings App.

Withings BodyScan 2 is a general wellness product, specifically intended to support and promote a general state of health and encourage healthy lifestyle activities. The device's features and output results do not relate to, nor are they intended to address, any specific disease, injury, or medical condition. Withings BodyScan 2 does not claim, nor is it intended for, diagnosis, prevention, monitoring, prediction, prognosis, treatment, or alleviation of any disease, injury, or disability.

Warnings & safety notice

EN

- This device uses a low-level electrical current. As a precaution, people with **implanted medical devices** such as **pacemakers** or **implantable cardioverter defibrillators (ICDs)** should consult their doctor before use. Compatibility depends on the specifications of the implanted device.
- This device is for wellness tracking only and should not be used for medical purposes. Always consult your doctor for any health concerns.
- If you are pregnant, accuracy may be impacted for body composition analysis, Skin Response Score, Vascular Age, and ECG (feature available via Withings ECG Monitor 2). Consult your doctor during pregnancy.
- The use of medication with anticholinergic agents may impact the Skin Response Score results. Ask your doctor about the medications you take if you are not sure whether they have anticholinergic properties.
- Drinking alcohol 24 hours prior to taking a measurement may affect the results of the Skin Response Score.
- Misuse of the device can cause injury.
- Measurement results may be impacted in case of lower limb amputation bigger than toes.
- Do not step on the scale with wet feet or hands.
- The device is not intended to continuously monitor vital signs in critical conditions or where the nature of the variations is such that it could result in immediate danger to the user.
- The device does not provide alarms.
- Keep away from children and pets. Small parts may be inhaled or swallowed.
- Exposing the device to prolonged lint, dust, or light (including sunlight) may reduce its lifetime or damage it.
- Inspect electrodes for warping, surface damage or corrosion and check for any other form of damage.
- Do not use the device if it is damaged. Use of a damaged device could cause injury or equipment failure.
- Do not submerge the scale or any of the components in water.
- Do not try to repair or modify this equipment yourself. No service or repairs should be performed on the device other than cleaning and charging guidance.
- Do not use accessories, detachable parts, or materials not described in the product guide. Use of parts and components other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Do not interconnect this equipment with other equipment not described in the product guide.
- Follow operational and storage conditions as described in the technical specifications section of this guide. Otherwise, measurement results may be impacted.
- To recharge the battery, use a power cord that complies with the safety standards of the country in which it is used and that matches the voltage of the power outlet.
- Do not open or disassemble the device for battery replacement.
- The power cord of the charger and the handle cable may cause a strangulation effect due to excessive length. Keep it away from children and pets.
- In case of severe lesions on the feet (such as ulcers), consult your doctor prior to use, as the performance of the measurement may be impacted. Avoid using the device on feet with poor skin integrity, such as cuts, wounds or damaged skin.

How to use Withings BodyScan 2 and take a first measurement

- To set up your device, you need an iOS or Android smartphone. Compatibility and minimum OS requirements may vary over time.
- Follow the information in the Quick Start Guide and in the Withings application to set-up your device.
- To wake up your device for the first time, please press the power button for 3 seconds. If your product is not starting, consider charging it for a minimum of three hours and try again.
- Once your product is installed and configured, it will appear in your devices list in settings.
- You may have to follow some tutorials in the Withings application in order to activate some measurements. Withings ECG Monitor 2 and Withings ECG App require you to follow a tutorial for the activation of the metric.
- To take the first measurement, grab the handle and step on the scale with bare feet. Your name will appear on the scale's screen during the measurement: it means your scale has correctly identified you and the measurements sequence will follow. Once you step off the scale, the scale will automatically power off.
- If it is not the case, the measurement will most likely appear in "Unknown Measurements" in the application. Please assign it to your user in the Withings application so that the scale can automatically recognize the user during the next weigh-in.

Troubleshooting — Withings BodyScan 2

If you cannot fix the problem using the following troubleshooting instructions, please contact Withings or go to: withings.com/support

Problems

Solutions

Scale does not wake up when I step on it.

- Make sure your Withings BodyScan 2 has enough battery life.
- Recharge your scale if necessary as described in this user guide.

I'm having syncing issues with my scale.

- Make sure that your Withings BodyScan 2 appears in Devices in the Withings application. If not, please install it.
- Make sure your Withings BodyScan 2 has enough battery life.
- Recharge your scale if necessary using the included USB-C power cable. When plugged in, your Withings BodyScan 2 will display the percentage of battery left.
- If your Withings BodyScan 2 is configured with Wi-Fi, make sure your router is plugged in, that your Wi-Fi is operational and that the scale is in range of your router.
- If necessary, reboot your scale. Press and hold the button of your scale for 20 seconds.
- If necessary, reboot your mobile phone.

I want to change the units of my scale (lb, kg, st lb).

- Press the button located on the back of your scale.
- Press the button located on the back of your scale to make the arrow point towards the right unit.
- Press and hold the button until the progress bar reaches the right edge of the screen.

Technical specifications

Product Name: Withings BodyScan 2

Model: WBS15

ECG Sensors: 2 stainless steel electrodes (handle) and 1 ITO electrode (glass plate)

Battery Operated: 1 year typical use on a single charge

Power Source: Power internally by a 3.7 Vdc Lithium ion battery (use the USB-C charging cable included) and a DC 5V power adapter with a minimum power of 3W and a maximum power of 5W.

Display: Color 3.42inch LCD screen

Weight Withings BodyScan 2: 4.8kg (10.6lb)

Dimensions Withings BodyScan 2 : 342*356*5cm

Operation time: 60 seconds

Parts in contact with the skin: Glass plate of the scale and Handle with its electrodes

Weight Units: Kilograms (kg), Pounds (lb), Stone (st lb)

Weight range: 5kg - 200kg | 11lb - 440lb | 1st - 31st

Transport and storage conditions:  -10°C (14°F) — 50°C (122°F)

 10% — 90%

 86kPa — 106kPa

Max altitude: 2000m

Operation conditions:  5°C (41°F) — 35°C (95°F)

 20% — 80%

 86kPa — 106kPa

Max altitude: 2000m

Wireless Communication: BLE and Wi-Fi (encrypted communication)

IP Protection level: IP22

Mode of operation: Continuous Operation

Expected product life: 3 years

Wireless information

Wireless Specifications

Mode	Frequency Band (MHz)	Maximum Output Power (dBm)
BT LE	2402-2480	5 dBm, 2.4G
WLAN	2412-2472	16 dBm

The wireless communication of the device is supported by BLE and Wi-Fi communication. This communication is established between the device and the Withings application. The communication between the device and the Withings application is encrypted through an exchange of a paired key.

The communication latency between the device and the Withings application takes less than 10 seconds when the device and the smartphone are less than 16 feet (5 meters) apart.

The communication between the device and the Withings application is not modified with sources of interference located within 5 meters.

Cleaning, Maintenance and Storage

Cleaning the Withings BodyScan 2

- It is recommended to clean the glass plate and the handle electrodes of the scale before each measurement.
- Use only a cloth dampened with warm water to clean your scale. The use of any detergents or cleaners may cause irreversible damage to the conductive coating on top of the glass plate.
- Do not use the device while doing cleaning and maintenance operations.
- To recharge the battery, follow instructions in this guide.

Storage

- Store the device in a clean and safe location between use.
- If storage conditions are different from the operating conditions indicated in the technical specifications, wait 30 minutes before taking a measurement.

Europe - EU Declaration of Conformity

Withings hereby declares that the device Withings BodyScan 2 is in conformity with the essential requirements and other relevant requirements of the applicable EU Directives and Regulations. The full text of the EU declaration of conformity can be found at: withings.com/compliance

Safety and performance

RF Statement

Electrical equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the accompanying documents. Portable and mobile RF communications equipment can affect electrical equipment.

Guidance and manufacturer's declaration-electromagnetic emissions

The device is intended for use in the electromagnetic environment specified below. The customer or the user of the device should ensure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic environment - guidance
CE emissions CISPR11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RE emissions CISPR11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Not applicable	

Declaration: electromagnetic emissions and immunity for equipment and systems that are not life-supporting and are specified for use only in a shielded location.

Declaration - electromagnetic immunity

Withings BodyScan 2 is intended for use in the electromagnetic environment specified below.

The customer or the user of the Withings BodyScan 2 should ensure that it is used in such an environment.

Safety and performance

Immunity test	IEC 60601 Test Level	Compliance Level	Electromagnetic environment – guidance
Conducted RF IEC 61000-4-6	3 Vrms 150kHz to 80MHz	N/A	N/A
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the equipment or system including cables than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Interference may occur in the vicinity of equipment marked with the following symbol:  .
Electrostatic discharge (ESD) IEC 61000-4-2	Contact: ±8 kV Air: ±2 kV, ±4 kV, ±8 kV, ±15 kV	Contact: ±8 kV Air: ±2 kV, ±4 kV, ±8 kV, ±15 kV	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-8	2 kV for power supply lines 1 kV for input/output lines	N/A	The main power quality should be of the kind used in a typical commercial or hospital environment.
Surge IEC 61000-4-5	1 kV differential mode 2 kV common mode	line(s) to line(s): +/- 1kV	The main power quality should be of the kind used in a typical commercial or hospital environment.
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	-5% UT (95% dip in UT) for 0.5 cycles, -40% UT (60% dip in UT) for 5 cycles, -70% UT (30% dip in UT) for 25 cycles, -5% UT (95% dip in UT) for 5 sec	N/A	The main power quality should be of the kind used in a typical commercial or hospital environment. If the user of the equipment or system requires continued operation during power main interruptions, it is recommended that the equipment or system be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz	The power frequency magnetic field should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Federal Communication Commission interference statement

This device complies with part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

FCC Caution: Changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate the equipment

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to part 15 of the FCC Rules.

These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful

interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one or more of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC ID: XNAWBS15

Disposal

Actuation of European directive 2012/19/EU, for reduction in use of dangerous substances in the electric and electronic device and for garbage disposal. The symbol applied on the device or its packaging means that at the end of its useful life, the product should not be disposed of with domestic waste. At the end of the device's useful life, the user must deliver it to a collection center for electric and electronic garbage, or return it to the retailer when purchasing a new device. Disposing of the product separately prevents possible negative environmental and health consequences deriving from inadequate disposal. It also allows the recovery of materials it is composed of to save energy and resources and avoid negative effects to the environment and health. In case of abusive disposal of devices by the user, will be applied for administrative endorsements in compliance with current standards. The device and its parts must be disposed of as appropriate, in accordance with national or regional regulations.

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Warranty

Withings™ Two (2) Years Limited Warranty - Withings BodyScan 2

Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, («Withings») warrants the Withings-branded hardware product («Withings Product») against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser («Warranty Period»). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or the quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

AU: Our goods come with guarantees that cannot be excluded under the Australian Consumer Law. You are entitled to a replacement or refund for a major failure and compensation for any other reasonably foreseeable loss or damage. You are also entitled to have the goods repaired or replaced if the goods fail to be of acceptable quality and the failure does not amount to a major failure.

Need help?

→ withings.com/support

Security

Withings recommends that you add a passcode (personal identification number [PIN]), Face ID or Touch ID (fingerprint) to your phone to add a layer of security. It is important to secure your phone since you will be storing personal health information. Users will also receive additional update notifications on the device via the Withings application, and updates are delivered wirelessly, encouraging rapid adoption of the latest security fixes. Do not install the device on a smartphone that you do not own. Do not use a public Wi-Fi network you don't know. Use a trusted Wi-Fi network with your device. No indication of anti-virus software is needed. Only use official app stores to download the Withings application. In case of doubt, use the link go.withings.com.

Equipment Symbol Description

Symbol for Waste Electrical and Electronic Equipment (WEEE). Do not dispose of this product as unsorted municipal waste, take it to electronic recycling.	The CE labeling certifies that the product conforms with the essential requirements and other relevant requirements of the applicable EU Directives and Regulations	IP22 Ingress of water or particulate matter	Model	CADA and Ecoponto logos
Consult instructions for use	Device manufacturer	Ingress of water or Type BF-Applied Part (electrodes of the glass plate and of the handle)	Unique device identifier	Espalmar and Azul logos
Direct Current	Temperature range	Do not use if package is damaged and consult instructions for use	Serial number	Packaging recyclable
	Temperature range	Do not use if package is damaged and consult instructions for use	Packaging recyclable	Packaging recyclable
	Upper and lower limits of pressure		Reference	Packaging recyclable
			Packaging recyclable	Consult instructions for use

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 Symbol for Waste Electrical and Electronic Equipment (WEEE). Do not dispose of this product as unsorted municipal waste, take it to electronic recycling.

 Consult instructions for use

 Direct Current

 The CE labeling certifies that the product conforms with the essential requirements and other relevant requirements of the applicable EU Directives and Regulations

 Device manufacturer

 Temperature range

 Temperature range

 Upper and lower limits of pressure

IP22 IP22 Ingress of water or particulate matter

 Ingress of water or Type BF-Applied Part (electrodes of the glass plate and of the handle)

 Do not use if package is damaged and consult instructions for use

 Do not use if package is damaged and consult instructions for use

 Model

 Unique device identifier

 Serial number

 Packaging recyclable

 Reference

 Packaging recyclable

Packaging recyclable

Packaging recyclable

 Consult instructions for use

Product guide

WITHINGS

© 2026 Withings. All rights reserved. January 2026
support.withings.com 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, France