WITHINGS
BODY SEGMENT
PRODUCT GUIDE

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USER GUIDE

ENGLISH

This guide explains how to use your Withings Body Segment

Important notice

EN

Before using Withings Body Segment, review the information in this guide. You may also find this guide online at: <https://www.withings.com/guides>

Retain this documentation for future reference.

Installation instructions are available within the Quick Start Guide provided with this user guide. Please contact Withings when in need of assistance, setting up, using or maintaining the device or to report unexpected operations or events.

Any serious incident that has occurred in relation to the Withings Body Segment should be reported to Withings and competent authorities in your country of residence.

Disclaimer: Information in this guide may change without notice.

This product is not a medical device.

Intended Use

Withings Body Segment is a connected smart scale providing the following wellness features:

- measure weight
- calculate Body Mass Index (BMI)
- measure segmental body composition
- measure pulse rate
- estimate vascular age
- compute electrodermal activity score

Warnings & safety notice

- Self-diagnosis and self-treatment can be dangerous. Always consult your doctor in case of questions or doubt.
- Do not use the Withings Body Segment if you have a cardiac pacemaker, an implantable cardioverter-defibrillator (ICD), or other implanted electronic devices.
- Misuse of the device can cause injury.
- Do not step on the scale with wet feet or hands.
- The device is not intended to continuously monitor vital signs in critical conditions or where the nature of the variations is such that it could result in immediate danger to the user.
- The device does not provide alarms.
- Keep away from children and pets. Small parts may be inhaled or swallowed.
- Exposing the device to prolonged lint, dust or light (including sunlight) may reduce its lifetime or damage it.
- Inspect electrodes for warping, surface damage or corrosion and check for any other form of damage.
- Do not use the device if it is damaged. Use of a damaged device could cause injury or equipment failure.
- Do not submerge the scale or any of the components in water.
- Do not try to repair or modify this equipment yourself. No service or repairs should be performed on the Withings Body Segment other than the maintenance listed in the device's instructions for use.
- Do not use accessories, detachable parts, or materials not described in the instructions for use. Use of parts and components other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions.
- Do not try to repair or modify this equipment yourself. No service or repairs should be performed on the Withings Body Segment other than the maintenance listed in the device's instructions for use.

Warnings & safety notice

- Do not use accessories, detachable parts, or materials not described in the instructions for use. Use of parts and components other than those specified or provided by the manufacturer of this equipment could result in increased electromagnetic emissions or decreased electromagnetic immunity of this equipment and result in improper operation.
- Portable RF communications equipment (including peripherals such as antenna cables and external antennas) should be used no closer than 30 cm (12 inches) to any part of the device, including cables specified by the manufacturer. Otherwise, degradation of the performance of this equipment could result.
- Use of this equipment adjacent to or stacked with other equipment should be avoided because it could result in improper operation. If such use is necessary, this equipment and the other equipment should be observed to verify that they are operating normally.
- Do not interconnect this equipment with other equipment not described in the instructions for use.
- Follow operational and storage conditions as described in the technical specifications section of the device's instructions for use. Otherwise, measurement results may be impacted.
- To recharge the battery, use a power cord that complies with the safety standards of the country in which it is used and that matches the voltage of the power outlet.
- The power cord of the charger and the handle cable may cause a strangulation effect due to excessive length. Keep it away from children and pets.
- Do not open or disassemble the device for battery replacement.

How to use Withings Body Segment and take a first measurement

EN

- Follow the information in the Quick Start Guide and in the Withings application to set-up your device.
- To wake up your device for the first time, please press the power button for 3 seconds. If your product is not starting, consider charging it for a minimum of three hours and try again.
- Once your product is installed and configured, it will appear in your devices list in settings.
- To take the first measurement, grab the handle (optional) and step on the scale with bare feet. Your name, or trigram (as configured in the app), will appear on the scale's screen at the end of the weight measurement: it means your scale has correctly identified you and the measurements sequence (as defined in the app) will follow.
- If it is not the case, the measurement will most likely appear in "Unknown Measurements" in the application. Please assign it to your user so that the scale can automatically recognize the user during the next weigh-in.

Troubleshooting — Withings Body Segment

If you cannot fix the problem using the following troubleshooting instructions, please contact Withings or go to: withings.com/support

Problems	Solutions
Scale does not wake up when I step on it.	- Make sure your Withings Body Segment has enough battery life. Recharge your scale if necessary as described in this user guide. - Battery in sleep mode (before installation/configuration).
I'm having syncing issues with my scale.	- Make sure that your Withings Body Segment appears in Devices in the Withings Application. If not, please install it. - Make sure your Withings Body Segment has enough battery life. Recharge your scale if necessary using the included USB-C power cable. When plugged in, your Withings Body Segment will display the percentage of battery left. - If your Withings Body Segment is configured with Wi-Fi, make sure your router is plugged in, that your Wi-Fi is operational and that the scale is in range of your router. - If necessary, reboot your scale. Press and hold the button of your scale for 20 seconds. - If necessary, reboot your mobile phone.
I want to change the units of my scale (lb, kg, st lb).	- Press the button located on the front side of your scale. - Press the button again to make the arrow point towards the right unit. - Press the button once more and maintain it until the progress bar is complete.

Technical specifications

Product Name: Withings Body Segment

Model: WBS08

Battery Operated: 1 year typical use on a single charge

Power Source: Power internally by a 3.6 Vdc Lithium ion battery (use the USB-C charging cable included) and a DC 5V power adapter with a minimum power of 3W and a maximum power of 5W

Heart Rate Measurement Range: 30bpm to 230 bpm

Display: Color 3.2inch LCD screen

Weight Withings Body Segment: 4.8kg (10.6lb)

Operation time: 60 seconds

Parts in contact with the skin: Glass plate of the scale and Handle with its electrodes

Weight Units: Kilograms (kg), Pounds (lb), Stone (st lb)

Weight range: 5kg - 200kg | 11lb - 440lb | 1st - 31st

Transport and storage conditions:

↕ -10°C (14°F) — 50°C (122°F)

🌀 10% — 90%

🌀 86kPa — 106kPa

Maximum altitude: 2000m

Operation conditions:

↕ 5°C (41°F) — 35°C (95°F)

🌀 20% — 80%

🌀 86kPa — 106kPa

Maximum altitude: 2000m

IP Protection level: IP22

Mode of operation: Continuous Operation

Wireless information

Wireless Specifications

Mode	Frequency Band (MHz)	Maximum Output Conducted Power (dBm)
BT LE	2402-2480	5 dBm
WLAN	2412-2462	16 dBm

The wireless communication of the device is supported by BLE and Wi-Fi communication. This communication is established between the device and the Withings application. The communication between the device and the Withings application is encrypted through an exchange of a paired key.

The communication latency between the device and the Withings application takes less than 10 seconds when the device and the smartphone are less than 16 feet (5 meters) apart.

The communication between the device and the Withings application is not modified with sources of interference located within 5 meters. Wireless coexistence has been tested in accordance with the following standards: - ANSI C63.27:2017 and, - AAMITIR69:2017

Electromagnetic disturbances have been tested in accordance with the standard IEC 60601-1-2:2014.

Cleaning, Maintenance and Storage

Cleaning and maintenance of Withings Body Segment

- It is recommended to clean the glass plate and the handle electrodes of the scale before each measurement.
- Use only a cloth dampened with warm water to clean your scale. The use of any detergents or cleaners may cause irreversible damage to the conductive coating on top of the glass plate.
- Do not use the device while doing cleaning and maintenance operations.
- To recharge the battery, follow the instructions in this guide.

Storage

- After a measurement using the handle, lower slowly so it rolls back into the scale without any sudden drop.
- Store the device in a clean and safe location between uses.
- If storage conditions are different from the operating conditions indicated in the technical specifications, wait 30 minutes before taking a measurement

Safety and performance

RF Statement

Medical electrical equipment needs special precautions regarding EMC and needs to be installed and put into service according to the EMC information provided in the accompanying documents. Portable and mobile RF communications equipment can affect medical electrical equipment.

Guidance and manufacturer's declaration-electromagnetic emissions

The device is intended for use in the electromagnetic environment specified below.
The customer or the user of the device should ensure that it is used in such an environment.

Emissions Test	Compliance	Electromagnetic environment - guidance
CE emissions CISPR11	Group 1	The device uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in nearby electronic equipment.
RE emissions CISPR11	Class B	
Harmonic emissions IEC 61000-3-2	Not applicable	The device is suitable for use in all establishments, including domestic establishments and those directly connected to the public low-voltage power supply network that supplies buildings used for domestic purposes.
Voltage fluctuations/ Flicker emissions IEC 61000-3-3	Not applicable	

Safety and performance

Declaration: electromagnetic emissions and immunity for equipment and systems that are not life-supporting and are specified for use only in a shielded location.

Declaration - Electromagnetic immunity

Withings Body Segment is intended for use in the electromagnetic environment specified below. The customer or the user of the Withings Body Segment should ensure that it is used in such an environment.

Immunity test	IEC 60601 Test Level	Compliance Level	Electromagnetic environment - guidance
Conducted RF IEC 61000-4-6	3 Vrms 150kHz to 80MHz	N/A	N/A
Radiated RF IEC 61000-4-3	3 V/m 80 MHz to 2.5 GHz	N/A	Portable and mobile RF communications equipment should be used no closer to any part of the equipment or system including cables than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter. Interference may occur in the vicinity of equipment marked with the following symbol:  .
Electrostatic discharge (ESD) IEC 61000-4-2	Contact: ± 8 kV Air: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	Contact: ± 8 kV Air: ± 2 kV, ± 4 kV, ± 8 kV, ± 15 kV	Floors should be wood, concrete, or ceramic tile. If floors are covered with synthetic material the relative humidity should be at least 30%.
Electrical fast transient/burst IEC 61000-4-8	2 kV for power supply lines 1 kV for input/output lines	N/A	The main power quality should be of the kind used in a typical commercial or hospital environment
Surge IEC 61000-4-5	1 kV differential mode 2 kV common mode	line(s) to line(s): +/- 1kV	The main power quality should be of the kind used in a typical commercial or hospital environment
Voltage dips, short interruptions and voltage variations on power supply input lines IEC 61000-4-11	-5% UT (95% dip in UT) for 0.5 cycles, -40% UT (60% dip in UT) for 5 cycles, -70% UT (30% dip in UT) for 25 cycles, -5% UT (95% dip in UT) for 5 sec	N/A	The main power quality should be of the kind used in a typical commercial or hospital environment. If the user of the equipment or system requires continued operation during power main interruptions, it is recommended that the equipment or system be powered from an uninterruptible power supply or a battery.
Power frequency (50/60Hz) magnetic field IEC 61000-4-8	30 A/m 50 Hz or 60 Hz	30 A/m 50 Hz or 60 Hz	The power frequency magnetic field should be at levels characteristic of a typical location in a typical commercial or hospital environment.

Information to users in Canada

IC: 11411A-WBS08

This device complies with Industry Canada's license exempt RSSs. Operation is subject to the following two conditions:

- (1) This device may not cause interference; and
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

Radiation Exposure Statement

This equipment complies with Canada radiation exposure limits set forth for an uncontrolled environment. This equipment should be installed and operated with a minimum distance 20cm between the radiator & your body.

Disposal

The symbol applied on the device or its packaging means that at the end of its useful life, the product should not be disposed of with domestic waste. At the end of the device's useful life, the user must deliver it to a collection center for electric and electronic garbage, or return it to the retailer when purchasing a new device. Disposing of the product separately prevents possible negative environmental and health consequences deriving from inadequate disposal. It also allows the recovery of materials it is composed of to save energy and resources and avoid negative effects to the environment and health. In case of abusive disposal of devices by the user, will be applied administrative endorsements in compliance with current standards. The device and its parts must be disposed of as appropriate, in accordance with national or regional regulations.

Warranty

Withings™ Two (2) Years Limited Warranty - Withings Body Segment

Withings, 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, ("Withings") warrants the Withings-branded hardware product ("Withings Product") against defects in materials and workmanship when used normally in accordance with Withings' published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser ("Warranty Period"). Withings' published guidelines include but are not limited to information contained in technical specifications, safety instructions or the quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product's use.

Security

Withings recommends that you add a passcode (personal identification number [PIN]), Face ID or Touch ID (fingerprint) to your phone to add a layer of security. It is important to secure your phone since you will be storing personal health information. Users will also receive additional update notifications on the device via the app, and updates are delivered wirelessly, encouraging rapid adoption of the latest security fixes. Do not install the device on a smartphone that you do not own. Do not use a public Wi-Fi network you don't know. Use a trusted Wi-Fi network with your device. The Withings application is not intended to be used on a computer. No indication of anti-virus software is needed. Only use official app stores to download the Withings application. In case of doubt, use the link go.withings.com.

Need help?

→ withings.com/support

Equipment Symbol Description

Do not dispose of this product as unsorted municipal waste, take it to electronic recycling.

Read the Instruction Manual before use

Direct Current

Temperature range

Upper and lower limits of pressure

Upper and lower limits of relative humidity

Ingress of water or particulate matter

Caution

Serial number

Withings hereby declares Withings Body Segment is in conformity with the essential and relevant requirements of applicable EU Directives. The full text of the EU declaration of conformity is available at: withings.com/compliance

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Withings Body Segment | Proactive Health Tracking © 2024 Withings. All rights reserved. Aug. 2024