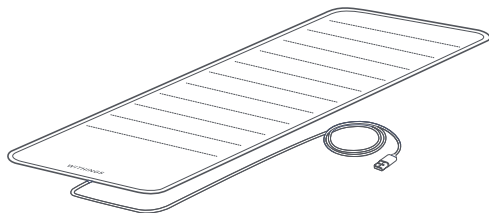
Withings Sleep Analyzer

QUICK INSTALLATION GUIDE

GUIDE D'INSTALLATION | INSTALLATIONSANLEITUNG | GUÍA DE INSTALACIÓN | GUIDA ALL'INSTALLAZIONE | GUIA DE INSTALAÇÃO

EN | FR | DE | ES | IT | PT

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How to install your Withings Sleep Analyzer

Comment installer votre Withings Sleep Analyzer | So installieren Sie Ihr Withings Sleep Analyzer |
Cómo instalar Withings Sleep Analyzer |

1

Place the entire sleep sensor under your mattress, so that it will be positioned underneath your chest. Only the cable should extend beyond the edge of your mattress.

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
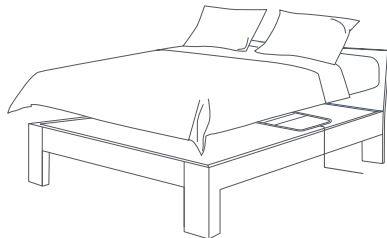
2

Plug the sensor into an outlet using the provided adapter.

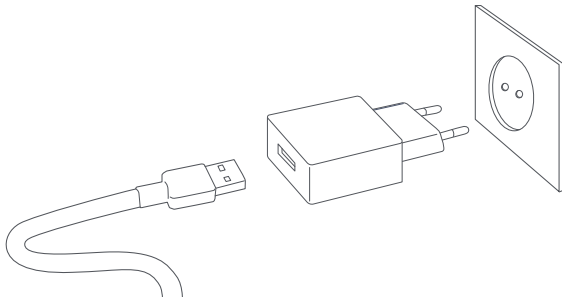- FR** Branchez le capteur à une prise murale en utilisant l'adaptateur fourni.
- DE** Verbinden Sie den Sensor anhand des mitgelieferten Netzteils mit einer Steckdose.
- ES** Utiliza el adaptador suministrado para enchufar el sensor a una toma de corriente.
- IT** Collegare il sensore a una presa di corrente utilizzando l'adattatore fornito in dotazione.
- PT** Ligue o sensor a uma tomada utilizando o adaptador fornecido.

How to install your Withings Sleep Analyzer

Comment installer votre Withings Sleep Analyzer | So installieren Sie Ihr Withings Sleep Analyzer |
Cómo instalar Withings Sleep Analyzer |

3

Visit go.withings.com from your smartphone or tablet, and follow the instructions to install the Health Mate app.

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4

In the Health Mate app, follow the instructions to install Withings Sleep Analyzer.

- FR** Dans l'app, suivez les instructions pour installer le Withings Sleep Analyzer.
- DE** Folgen Sie der Anleitung in der App, um Ihr Withings Sleep Analyzer zu installieren.
- ES** Desde la aplicación, siga las instrucciones de instalación de Withings Sleep Analyzer.
- IT** Nell'applicazione, seguire le istruzioni per installare Withings Sleep Analyzer.
- PT** Siga as instruções no aplicativo para instalar o Withings Sleep Analyzer.

Care and maintenance

Entretien et maintenance | Pflege und Wartung | Cuidado y mantenimiento | Cura e manutenzione | Cuidado e manutenção

To clean the textile cover of the sensor, gently remove the cover first. The cover can be machine washed, but do not tumble dry or iron it. Wait until the cover is completely dry before replacing it on the sensor.

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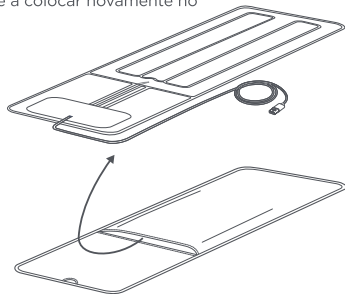
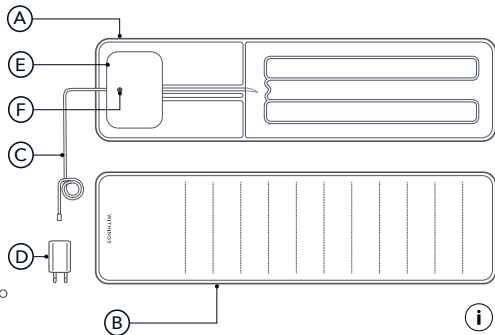
Federal Communication Commission Interference Statement

This equipment has been tested and found to comply with the limits for a Class B digital device, pursuant to Part 15 of the FCC Rules. These limits are designed to provide reasonable protection against harmful interference in a residential installation. This equipment generates, uses and can radiate radio frequency energy and, if not installed and used in accordance with the instructions, may cause harmful interference to radio communications. However, there is no guarantee that interference will not occur in a particular installation. If this equipment does cause harmful interference to radio or television reception, which can be determined by turning the equipment off and on, the user is encouraged to try to correct the interference by one of the following measures:

- Reorient or relocate the receiving antenna.
- Increase the separation between the equipment and receiver.
- Connect the equipment into an outlet on a circuit different from that to which the receiver is connected.
- Consult the dealer or an experienced radio/TV technician for help.

FCC Caution: Any changes or modifications not expressly approved by the party responsible for compliance could void the user's authority to operate this equipment.

This device complies with Part 15 of the FCC Rules. Operation is subject to the following two conditions: (1) This device may not cause harmful interference, and (2) this device must accept any interference received, including interference that may cause undesired operation.

Important note: Radiation Exposure Statement

The product complies with the US portable radio frequency (RF) exposure limit set forth for an uncontrolled environment and is safe for intended operation as described in this manual. Further RF exposure reduction can be achieved if the product is kept as far as possible from the user's body or if the device is set to lower output power if this function is available.

This transmitter must not be co-located or operating in conjunction with any other antenna or transmitter.

Country Code selection feature to be disabled for products marketed to the US.

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12 – 13

IC: 11411A-WSM02

This device complies with Industry Canada's licence exempt RSSs. Operation is subject to the following two conditions:

- (1) This device may not cause interference; and
- (2) This device must accept any interference, including interference that may cause undesired operation of the device.

Radiation Exposure Statement

The product complies with the Canada portable RF exposure limit set forth for an uncontrolled environment and is safe for intended operation as described in this manual. Further RF exposure reduction can be achieved if the product is kept as far as possible from the user's body or if the device is set to lower output power if such function is available.

Country Code selection feature to be disabled for products marketed to CANADA.

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US Withings Two (2) Years Limited Warranty – Withings Sleep Analyzer

Withings 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, France (“Withings”) warrants the Withings-branded hardware product (“Withings Product”) against defects in materials and workmanship when used normally in accordance with Withings’ published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser (“Warranty Period”). Withings’ published guidelines include but are not limited to information contained in technical specifications, safety instructions or quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product’s use.

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UK Withings Two (2) Years commercial guarantee – Withings Sleep Analyzer

Withings 2 rue Maurice Hartmann, 92130 Issy-les-Moulineaux, France (“Withings”) warrants the Withings-branded hardware product (“Withings Product”) against defects in materials and workmanship when used normally in accordance with Withings’ published guidelines for a period of TWO (2) YEARS from the date of original retail purchase by the end-user purchaser (“Warranty Period”). Withings’ published guidelines include but are not limited to information contained in technical specifications, safety instructions or quick start guide. Withings does not warrant that the operation of the Withings Product will be uninterrupted or error-free. Withings is not responsible for damage arising from failure to follow instructions relating to the Withings Product’s use.

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Sleep Apnea Detector - Instructions for Use

Rev. 1.4

1

INDICATIONS FOR USE

Sleep Apnea Detector is a medical device used in the management of the sleep apnea syndrome at home. It is not intended to monitor vital signs in critical conditions.

Sleep Apnea Detector is indicated for use on adults (18 years of age or older).

Sleep Apnea Detector may be used

- as a diagnostic test for the risk of sleep apnea syndrome
- as a monitoring tool in patients treated by alternative treatments to CPAP,
- as an educational tool for patients treated with CPAP,
- and by health-conscious individuals.

2

USING THE Sleep Apnea Detector

a. Set-Up

- Withings Sleep is a sleep tracking mat, distributed without Sleep Apnea screening function
- Sleep Apnea Detector is activated during the installation of WSA-SW1 with Withings Sleep
- Sleep Apnea Detector is only available for Withings Sleep, installed with a smartphone with iOS 10 or later, or Android 6 or later.

b. On-boarding

- Open Withings Health Mate app.
- In the Device tab, select your Withings Sleep, then select “Understand sleep apnea detection”
- Follow the instructions.
- You may exit on-boarding at any time by pressing the return button.

c. Track Sleep Apnea

- Check that the placement of your Withings Sleep is optimal for Sleep Apnea detection, as described in product installation. It should be:
 - Placed under the mattress
 - With the maximal contact surface between Withings Sleep and the bed, and between Withings Sleep and the bed box. If needed, a cardboard sheet can be used to maximize contact.
 - Placed at heart level
 - Covering at best your sleeping zone in the bed.
- Make sure your Withings Sleep is plugged to the power outlet using provided adapter. If not, plug the Withings Sleep and wait for 10 minutes before anyone enters the bed. Don't let heavy objects on the bed when you are not using it.
- Go to sleep for at least 5 hours
- Withings Sleep will synchronize the data to your Health Mate account once you get out of bed
- Results are readable in the Health Mate app

3

Sleep Apnea results analysis

- After a night tracked, you will receive one of the following classifications on your Sleep Apnea Detector along with the related Apnea Hypopnea Index (AHI) score :
 - Low / Mild: This means you experienced less than 15 apneic episodes per hour in average last night
 - Moderate: This means you experienced between 15 to 30 apneic episodes per hour in average last night.
 - Severe: This means you experienced more than 30 apneic episodes per hour in average last night.
 - Night too short: A minimum of 5 hours spent asleep are needed to compute Sleep Apnea results. A new night spent on Withings Sleep is needed to compute results
 - Noise sensor disabled: You need to activate this sensor in order to compute Sleep Apnea results. Once done, a new night spent on Withings Sleep is needed to compute results
- The Mild, Moderate and Severe classification are provided following the American Association of Sleep Medicine (AASM) recommendations for Sleep Apnea results analysis.
- The Sleep Apnea results are available on a detailed display in the Health Mate app.

-
- Classification of Sleep Apnea syndrome into these 3 categories may represent only potential findings. If you are experiencing any symptoms or have concerns, contact your physician. If you believe you are experiencing a medical emergency, you should contact emergency services.
 - A small percentage of people may have certain physiological conditions preventing the user from creating enough signal to produce a good recording.
 - For more information on sleep apnea and apneic episodes, please read the articles at the bottom of your sleep apnea report or in the Sleep Apnea guide.
 - All results are synced via Wi-Fi and can then be visualized on Health Mate on your Android or iOS smartphone. You may use the Health Mate app to share your sleep apnea results with a clinician.

4

SAFETY AND PERFORMANCE

The Sleep Apnea Detector software's ability to accurately classify a sleep recording into sleep apnea severity was studied in a multi-centric clinical investigation, on 118 subjects, against polysomnography scored according to AASM recommendations. The Sleep Apnea Detector demonstrated 88.0% sensitivity and 88.6% specificity on the AHI = 15 events/h threshold, and 86.0% sensitivity and 91.2% specificity on the AHI = 30 events/h threshold.

5

TROUBLESHOOTING

If you experience difficulties in operating your Sleep Apnea Detector, refer to the troubleshooting guide below.

Problem 5.1: My sleep data is not available in Health Mate

Solutions:

- Follow the troubleshooting procedure [here](#)

Problem 5.2: I can see my sleep data, but no sleep apnea results

Solution:

- Check if your Withings Sleep is up to date with the latest firmware, in the Devices > Settings section. Information about the latest firmware releases is available [here](#).

6

CAUTIONS

The Sleep Apnea Detector cannot provide information for other sleep breathing disorders than sleep apnea. If you believe you're having a medical emergency, call emergency services.

DO NOT take into account recordings performed outside of the operational temperature range (10°C - 35°C) , humidity range of 15% to 85% relative humidity and atmospheric pressure range 700-1060hPa.

DO NOT take into consideration results from data tracked when another person than the main user lays on the top of Withings Sleep.

DO NOT use with water beds, air beds, or mechanically articulated mattresses.

DO NOT use Sleep Apnea Detector when the Withings Sleep is in close vicinity to strong electromagnetic fields (e.g. electromagnetic anti-theft systems, metal detectors).

DO NOT change your medication or treatment without talking to your doctor. Not intended for use by individuals under age 18. You should talk to your doctor if you experience symptoms or concerns. Interpretations made by this software are potential findings, not a complete diagnosis of neither sleep apnea, nor other sleep breathing disorders. The user is not intended to interpret or take clinical action based on Sleep Apnea Detector output without consultation of a qualified healthcare professional.

Sleep Apnea Detector is not indicated for use on children (< 18 years).

The Sleep Apnea Detector does not have any residual risk after mitigations.

Sleep Apnea Detector is not indicated for use on patients with arrhythmia, cardiac insufficiency, asthma, or chronic obstructive pulmonary disease.

For all instructions specific to the Withings Sleep, please refer to the user manual accompanying the Withings Sleep Product.

CAUTION: Withings does not guarantee that you are not experiencing sleep apnea or other health conditions when the Sleep Apnea Detector labels your sleep data with mild sleep apnea.

You should notify your physician if you detect possible changes in your health.

CAUTION

EN -Only the power supply that was included with the device should be used. Withings warranty does not cover defects due to the use of parts not provided by WITHINGS in the original packaging.

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Read instructions before use.

Australian Sponsor:

Emergo Australia, Level 20, Tower II
Darling Park, 201 Sussex Street- Sydney, NSW 2000
Australia

WITHINGS

Withings Sleep Analyzer - Model: WSM02 - V 2.0

© 2021 Withings All rights reserved.